


---

---

---

---

---

---

---

---

---

---

---

---

### Oticon Medicals solution to this!

Ponto Streamer & Connect Line

- Offers help in many of the previously mentioned examples
- You can control the many functionalities through one device
- Easy to use
- Grows with the user
- Wireless Bluetooth 2,4 GHz

oticon MEDICAL

---

---

---

---

---

---

---

---

---

---

---

---

### All Processors still available

Ponto Plus & Ponto Plus Power  
Power

Ponto, Ponto Pro & Ponto Pro  
Power

oticon MEDICAL

---

---

---

---

---

---

---

---

---

---





---

---

### Ponto Plus Processors

► Ponto Plus processors offer:

- New transducer
  - More powerful
  - Improved efficiency
- Improved Dynamic Feedback Cancellation
- Inium Wireless Platform
  - Developed in-house
  - The latest from Oticon
  - Developed for power aids and bone anchored device
- Wireless Connectivity
  - Streamer
  - ConnectLine


---

---

---


---

---

---

---

---

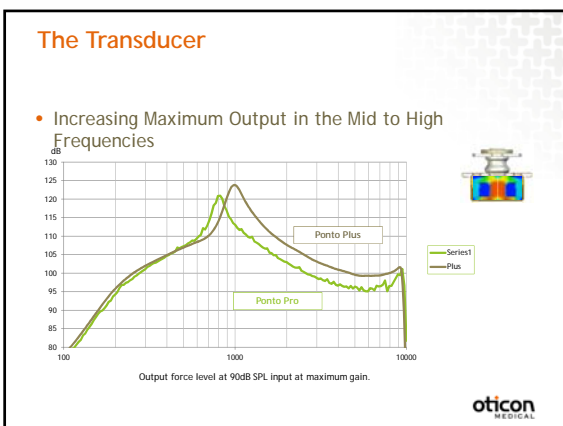


---

---

---


---

---

---

---

---

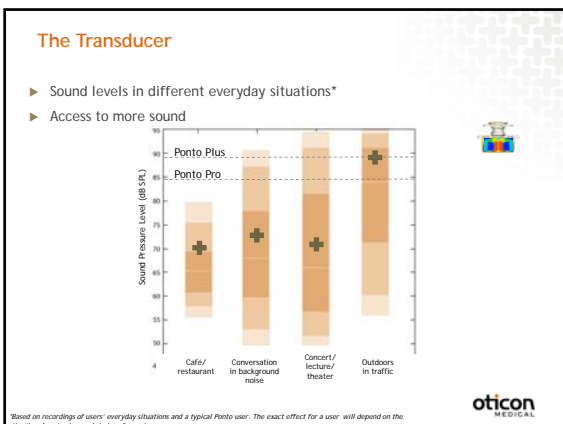


---

---

---

---

---

---

---

---

## Connectline & Ponto Streamer





---

---

---

---

---

---

---



---

### What is Ponto Streamer?

- It works as a remote control
- Oticon Medical branded
- It's worn around the neck

Focus on making a wireless solution with little power consumption in the sound processor, to create least possible limitations on the sound processors output. 10 hours of constant streaming

- Open Ponto Streamer - Doesn't need to pair in Genie Medical
- Wireless - Bluetooth 2.4GHz
- User centric solution - One device to control the different features with dedicated buttons
- Telecoil included


---

---

---

---

---

---

---

---


### Using the Ponto Streamer without the neck loop

The Ponto Streamer can also be used without the neck loop, e.g. with the optional belt clip. The distance to the sound processor should then be less than one meter. You may experience drop-outs in sound, which is why we recommend using the neck loop. If the Ponto Streamer is used without the neck loop, then the Ponto Streamer functionalities depend upon whether the Ponto Streamer is open or linked; see the table below.

Functionality	Works in an open Ponto Streamer	Works in a linked Ponto Streamer
As a remote control	No	Yes
Hear the ring from an incoming call	Yes	Yes
Streaming of sound from e.g. TV, or phone calls	No	Yes

If the Ponto Streamer is used with the neck loop then it has all functionalities activated no matter whether the Ponto Streamer is open or linked.

**IMPORTANT NOTICE**  
A defective neck loop should be replaced immediately.




---

---

---

---

---

---

---


---

### Ponto Streamer Fitting

► Optional linking Ponto Streamer and Ponto Plus

1. Attach the Ponto Streamer to the PC the ConnectLine / Settings tool change the the status to 'Linked'
2. Enter the Streamer's serial number

See quick programming guide for detailed instructions



oticon MEDICAL

---

---

---

---


---

---

---

---

When using the Ponto Streamer without the neck loop, more power will be used – consequently maximum streaming time is reduced to 5 hours on one full charge.

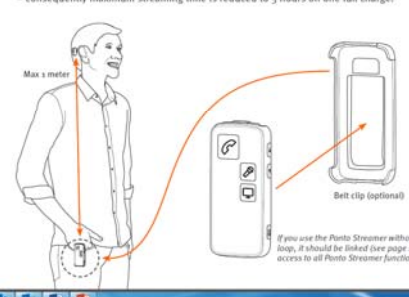

Max 4 meter

Belt clip (optional)

If you use the Ponto Streamer without the neck loop, it should be linked (see page 27) to get access to all Ponto Streamer functions.

oticon MEDICAL

---

---

---

---

---

---

---

---

Battery capacity and charge time	
Battery capacity using neck loop	Up to 10 hours typical daily use: 2 hours phone, 6 hours TV and 2 hours standby (after 2 years the capacity will be approx. 8 hours typical daily use).
Battery capacity not using neck loop	Up to 5 hours typical use: 1 hour phone, 3 hours TV and 1 hour standby.
Battery charge time	2 hours for complete charge, using enclosed power supply.
Remote	If Ponto Streamer is turned off the remote functionality of the sound processor will still work; it will work for up to 6 months on a full charge.
Battery life expectancy	The battery in Ponto Streamer will last for more than 2 years if fully charged once a day. The battery can be replaced by your hearing care professional.

28

---

---

---

---




---

---

---

---

### Ponto Streamer

Ponto Streamer status	sends signal to	to be used
Open Streamer as it comes from factory	any Ponto Plus within 1 m 	- with neck loop
Linked Streamer (linking takes place in Genie Medical)	only to the linked Ponto Plus 	- with or without neck-loop
Demo Streamer (for demonstration purposes, cannot be linked)	any Ponto Plus within 1 m 	- with neck loop

**oticon**  
MEDICAL

---

---

---


---

---

---

---

---

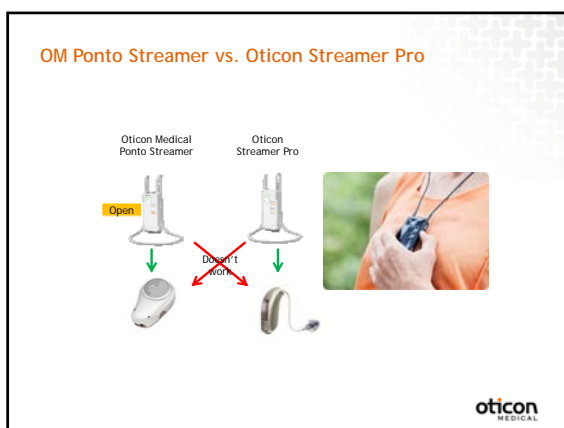


---

---

---

---

---

---


---


---

### The Ponto Streamer


The most connectivity opportunities in one system






The Ponto Streamer gives you all these communication possibilities / functionalities.



**oticon**  
MEDICAL

---

---

---

---

---

---

---

---

### Wireless Connectivity

► Ponto Streamer

Built-in telecoil  
Remote control function  
Wireless communication possibilities  
Including mini jack cable  
Contact for FM

oticon MEDICAL

---

---

---

---

---

---

---

---

### Ponto Streamer buttons/plugs

oticon MEDICAL

---

---

---

---

---

---

---

---

### Instrument view

11

---

---

---


---

---

---

---

---

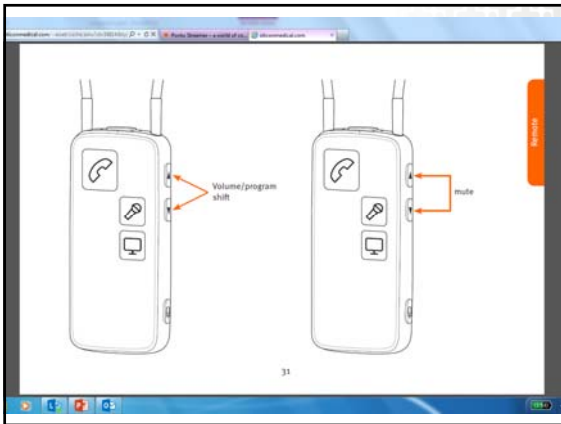

---

---

---


---

---

---

---

---

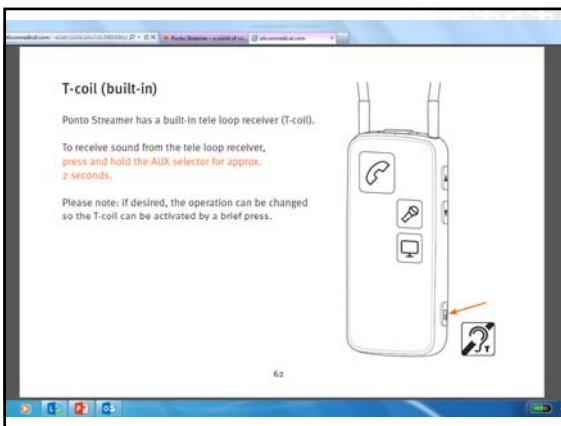

---

---

---


---

---

---

---

---

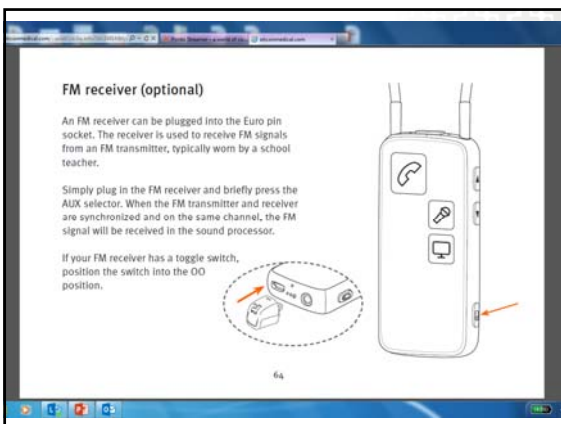

---

---

---

---

---

---

---

---



Replacement neck loops  
Replacement neck loops are available in three sizes: Short (51 cm), medium (66 cm) and long (81 cm) from your hearing care professional.

Belt clip  
A belt clip is available from your hearing care professional.

Charger Base  
A Charger Base for charging the Ponto Streamer is available from your hearing care professional.



69

---

---

---

---

---

---

---

---

### The Ponto Streamer

The most connectivity opportunities in one system

ONE DEVICE CONTROLS ALL NEEDS



oticon MEDICAL

---

---

---

---

---

---

---

---

### What is Connectline?

- A series of devices that helps users connect to a wide variety of communication and entertainment applications
- It's the one solution which offers most connectivity opportunities in one system
- Must be paired to the Ponto Streamer
- Not Oticon Medical branded



oticon MEDICAL

---

---

---

---

---


---

---


---

### ConnectLine and accessories

Extending the communication possibilities



The ConnectLine products are the same as for Oticon Streamer and hearing aids.



---

---

---

---

---

---



---

---

### Accessories - Connect Line TV adapter

The ConnectLine TV adapter is connected to the TV and transmits sound directly to the Ponto Plus sound processor through Ponto Streamer. To listen to the TV, all you have to do is to push the TV button on the Ponto Streamer.

- Oticon product (Identical with Oticons)
- Transmission range up to 100 feet (30 meters)



---

---

---

---



---

---

---

---

### Instruction video for TV adaptor - Install



---

---

---

---

---

---

---

---

Instruction video for TV adaptor - How To Use



---

---

---

---

---

---

---

---

Accessories - Connect Line microphone

The discreet microphone is worn by the conversation partner and connects wirelessly to Ponto Streamer, allowing you to hear the other person's voice directly in the Ponto Plus sound processor.

- Oticon product
- Transmission range up to 50 feet



---

---

---

---

---

---

---

---

Instruction video for Companion Mic & Orientation - How to Pair



---

---

---

---

---

---

---

---

### Instruction video for Companion Mic & Orientation - Wearing a CL Mic



---

---

---

---

---

---

---

---

### Accessories - Connect Line phone adapter

The ConnectLine phone adapter makes it possible to use the Ponto Streamer with a traditional analog landline phone. The solution supports both incoming and outgoing phone calls. During a phone call, the sound processor is used as a headset and the microphone on body worn Streamer enables hands-free operation.

- Oticon product
- Transmission range up to 100 feet (30 meters)



---

---

---

---

---

---

---

---

### Instruction video for land line phone adaptor



---

---

---

---

---


---

---

---

### Connecting an FM receiver to Ponto Plus

On Ponto Plus/ Ponto Plus Power there isn't a connection socket on the sound processor. Instead you connect it through the connection socket in the bottom of the Ponto Streamer. The FM plug used for Ponto Streamer looks like this:



Known FM receivers which can be used together with Ponto Plus and Ponto Plus Power are:

- Oticon Amigo R2 & Oticon Amigo R1
- Phonak MLxi universal receiver & Phonak Roger X universal receiver
- Widex universal receiver
- Comfort Audio DT-20

oticon MEDICAL

---

---

---

---

---


---

---

---

### Connecting an FM receiver to Ponto / Ponto Pro

On Ponto / Ponto Pro /Ponto Pro Power you attach the FM receiver directly to the sound processor through the DAI socket in the sound processor itself. The FM plug used for Ponto / Ponto Pro / Ponto Pro Power looks like this:



FM receivers which can be used together with Ponto, Ponto Pro and Ponto Pro Power are:

- Oticon Amigo R2BA
- Phonak MLxi BAHHA
- Phonak MLxS BAHHA

oticon MEDICAL

---

---

---

---

---

---

---

---

### Making an FM program with the Ponto Pro

oticon MEDICAL

---

---

---

---

---

---


---

---

### FM solution through Ponto Plus / Ponto Streamer

**Binaural solution - Only 1 FM receiver**  
- An FM receiver connected to the Ponto Streamer transmits to both Ponto Plus sound processors at the same time

**Connectivity in the classroom**  
- Attending classes is often not just a matter of listening to the teacher's voice. Sometimes students also need to listen to sound on PC's, or to TV programs or music.



40 **oticon** MEDICAL

---

---

---

---

---

---


---

---

### FM solution through Ponto Plus / Ponto Streamer

**Possible benefits for parents**  
- Choosing a solution is not only about focusing on the child's life in school.  
- Parents may find it beneficial that their child could connect the Ponto Streamer to their cell phone.

**Other advantages outside school**  
- In other cases outside of school, it could also be relevant for a child to use the Ponto Streamer, e.g. when he or she is watching TV, listening to music, using Skype, etc.



41 **oticon** MEDICAL

---

---

---

---

---

---

---

---

### FM solution through Ponto Plus / Ponto Streamer

**Battery usage in the sound processor**  
- When the FM receiver is connected to the Ponto Streamer, it uses battery from the Ponto Streamer instead of using the battery from the sound processor.

**Changing among different programs**  
- For some children it could possibly be easier to handle switching between FM listening and other programs using the controls on the Ponto Streamer.

**Wider selection of FM receivers**  
- More FM receivers on the market have the plug type used in Ponto Streamer



42 **oticon** MEDICAL

---

---

---

---


---

---

---

---

Instruction video of FM orientation and set up



oticon  
MEDICAL

---

---

---

---

---

---

---

---

Accessories – Sennheiser Communication UI Box\* for office phone



\* Together with a phone adaptor it can also work as a wireless solution

oticon  
MEDICAL

---

---

---

---

---

---

---

---

Ponto Plus Fitting and  
Genie Medical 2013.1



oticon  
MEDICAL

---

---

---

---

---

---

---

---

**Genie Medical 2013.1**

New In Genie Medical



- Instructional Videos for Streamer and ConnectLine
- ConnectLine Tool
- Ponto Streamer Settings Tool
- Individualized DFC Prescription
- Max Force Output Control



---

---

---

---


---

---

---

---

**Connecting Ponto Streamer to Genie Medical**



---

---

---

---

---

---

---

---

**EXPRESSlink programming device**



- EXPRESSlink is an alternative programming device to the HiPro and NearLink
- Faster programming with EXPRESSlink
- Can be used with all Ponto sound processors



---

---

---

---

---

---



---

---



### Connect line and the SSD patient

- Remote control
- Companion mic for car / restaurant (improve s/n ratio)
- Hands free phone calls to Ponto Plus keep the good ear open
- Option of extra year warranty if not interested in Streamer


---

---

---

---

---

---

---

---

---

---

---

---

### Ordering Ponto Plus - Add Streamer or warranty

130-000-230-000	Ponto Plus, left, chrome barge	1M 200 Hz	Distortion soft band barge
130-000-230-000	Ponto Plus, right, chrome barge	1M 200 Hz	Distortion soft band chrome barge
130-000-230-000	Ponto Plus, left, white silver	1M 200 Hz	Distortion soft band steel
130-000-230-000	Ponto Plus, right, white silver	1M 200 Hz	Distortion soft band chrome


**SELECT PLUS PROCESSOR**

**ADDITIONAL PROCESSORS**

**EXTRA YEAR WARRANTY**

**PONTO STREAMER**

*Select one option*




---

---

---

---

---

---

---

---

---

---





---

---

### Genie Medical 2013.1

► Summary

- The Ponto Streamer and Connect line were designed to help the Ponto user maximize their hearing in challenging listening situations.
- The fitting procedure for Ponto Plus is similar to the procedure for previous Ponto sound processors
- The Ponto Streamer comes out of the box ready to use with Ponto Plus
- It is possible to fine tune a Ponto Streamer and ConnectLine, and individualize the Ponto Streamer in Ponto Settings
- Streamer is included with every new order, or opt for extra year repair warranty


---

---

---

---

---

---

---

---

---

---

---

---

List Pricing and price to customers calling OM

Description	Part #	List price	Patient Pay
Connect Line TV**	124771	\$ 150.00	\$ 300.00
Connect Line Phone**	128812	\$ 100.00	\$ 200.00
Connect Line Microphone**	108402	\$ 175.00	\$ 350.00
Digital Office Phone**	129740	\$ 100.00	\$ 200.00
USB Blue Tooth Dongle	123815	\$ 50.00	\$ 100.00
Take 10% off Connect Line Items if two items are ordered			
Take 15% off Connect Line Items if three or more are ordered			




---

---

---

---

---

---

---

---

---

---

Father and son..... Which one has the tattoo?




---

---

---

---

---

---

---

---

---

---

Questions?

Please feel free to contact me offline with any additional questions

Rebecca Cihocki Au. D.  
[rcih@oticonmedical.com](mailto:rcih@oticonmedical.com)

Cell (732) 289-1900




---

---

---

---

---

---

---

---

---

---