



Bionix Lighted Ear Forceps



Item #: 2750
Description: Bionix Lighted Ear Forceps

WHAT IS IT?

LED light source illuminates a disposable forceps for use during cerumen removal.

HOW DOES IT WORK?

Single-use, clear forceps is inserted into the LED light source. When locked in place, the LED light source is activated, illuminating the forceps. The tip of the forceps may be open and closed by manipulating that spring grip. When the spring grip is squeezed, it is forced to slide down toward the forceps tip, causing the tip to close while providing gripping strength. When the spring grip is released, the forceps tip will open. Magnification piece attaches near the base of the light source.

HOW IS THE PRODUCT SET UP?

- Insert end of forceps into light source
- Push down and rotate forceps clockwise to lock forcep in place; this will activate the LED light and illuminate the forceps
- You are now ready to use the lighted forceps to remove cerumen.

Oaktree Products, Inc.

CALL US AT 800.347.1960 FOR MORE INFORMATION; OTP@oaktreeproducts.com