

## GSI Suite: Expand Your Data Options

December 9<sup>th</sup>, 2015  
Karen Morris, MS, CCC/A



---

---

---

---

---

---

---

---

### Karen Morris

- Experience as a Clinical Audiologist in hospital and neurotology settings for many years.
- Past 10 years worked for audiology equipment manufacturing in education and support role



---

---

---

---

---

---

---

---

### Course Objectives

- Discuss the new graphic display options for acoustic reflexes, decay and tympanometry.
- Utilize the new features of GSI Suite 2.3 to build a custom report.
- Merge other certified Noah modules to create a custom report using GSI Suite 2.3.



---

---

---

---

---

---

---

---

## Agenda

1. Overview of GSI Suite
2. What's New and How To?
3. Using Report Designer
  - Using the Template editor to create customized layout
  - Merging Reports with certified Noah Modules Sin, etc)
  - Sequential Audiogram Reporting
4. Review



---

---

---

---

---

---

---

---

## Overview

GSI Suite version: 2.3.xx or higher  
GSI TympStar Pro



---

---

---

---

---

---

---

---

## GSI Suite

Data management software program

Can be used stand-alone OR  
Installed within NOAH.



---

---

---


---

---

---

---

---

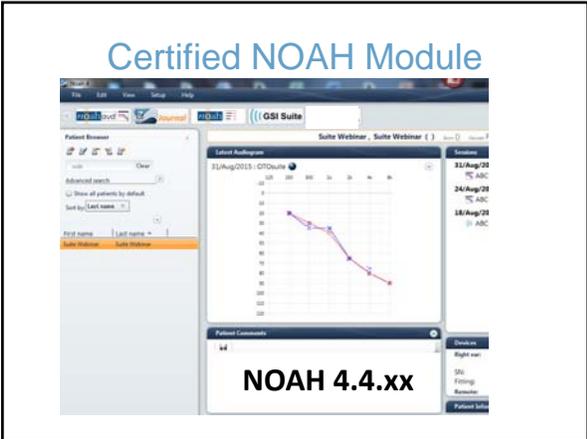

---

---

---

---

---

---

---

---

---

---



---

---

---

---

---

---

---

---

---

---



---

---

---

---

---

---

---


---

---

---

## Report (Print) to File

Configure → Templates Tab → Report File

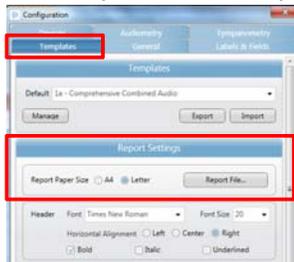

---

---

---

---

---


---

---

---

## Report (Print) to File

Configure → Templates Tab → Report File

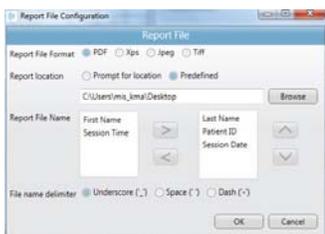

---

---

---

---

---

---

---

---

## Session Information Lists

Configure →  
General Tab →  
Manage Session Info  
Lists



---

---

---

---

---

---

---

---

### Session Information Lists

Examiner	License Number
Karen Morris, MS	147-000179

Examiner: Karen Morris, MS  
License Number: 147-000179

Add Edit Delete Close

---

---

---

---

---

---

---

---

### Session Information Lists

Session Information

8/18/2015 12:27 PM  
1 of 3

Examiner: Karen Morris, MS  
Location: South 1 ENT Clinic  
Reliability: Main Clinic  
Physician: Catalina Clinic

---

---

---

---

---

---

---

---

### Creating Predefined Comments

Configure →  
General Tab →  
Manage Predefined  
Comments

---

---

---


---

---

---

---

---

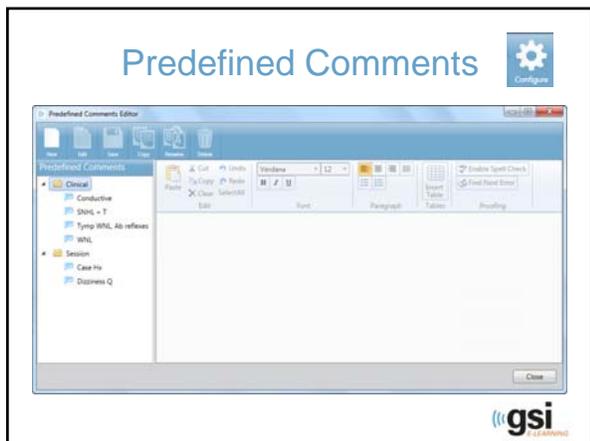


---

---

---


---

---

---

---

---

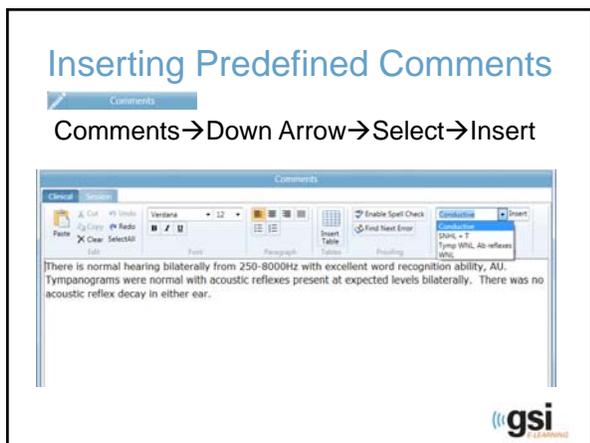


---

---

---


---

---

---

---

---

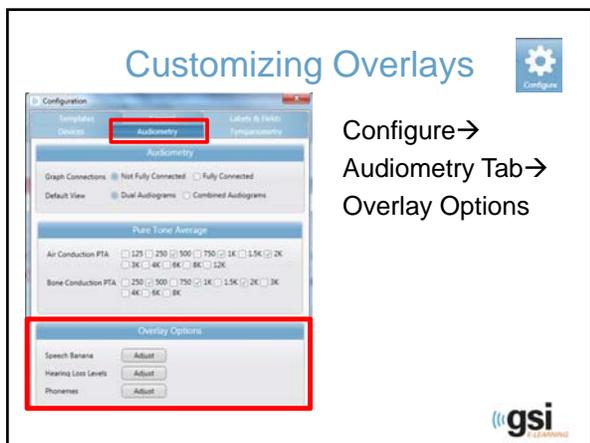


---

---

---

---

---

---

---

---

## Customizing Labels



Configure →  
Labels and Fields




---

---

---

---


---

---

---

---

## Merge Sessions

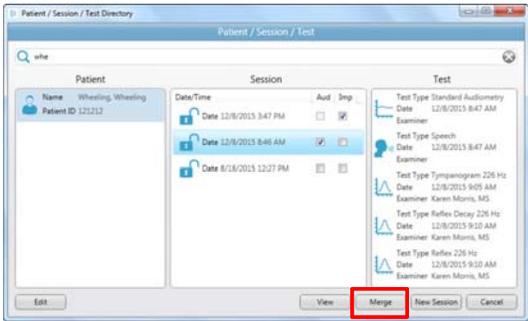


---

---

---

---

---

---

---

---

## Merge Sessions Tips

- Can be used with NOAH to combine audiometric and tympanometric results from other NOAH certified modules
- OR combining tests from different sessions
- Tympanometry/Reflex data from non-GSI devices appears in Screening Tymp Tab
- Reporting purposes only. Merged sessions are virtual




---

---

---

---


---

---

---

---

## Customizing or Creating Reports

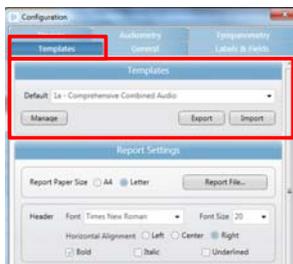

Configure →  
Templates Tab →  
Manage



---

---

---

---

---

---

---

---

## Customizing or Creating Reports



---

---

---

---


---

---

---

---

## Copy or New

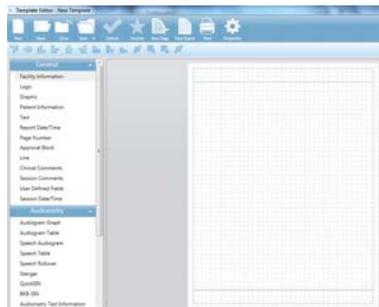

1. Drag Element onto Page
2. Re-size
3. Right Click
4. Edit Properties
5. Save As (Name)
6. Favorite?



---

---

---

---

---

---

---

---

Questions?



---

---

---

---

---

---

---

---

Thank You!

Visit the Grason-Stadler Website for additional CEU opportunities  
[www.grason-stadler.com/e-learning](http://www.grason-stadler.com/e-learning)

Sign-up for E-Learning Alerts  
<http://www.grason-stadler.com/e-learning/e-learning-alerts>



---

---

---

---

---

---

---

---

Contact Information

Karen Morris  
[krmr@grason-stadler.com](mailto:krmr@grason-stadler.com)

GSI Audiology  
[audiology@grason-stadler.com](mailto:audiology@grason-stadler.com)

GSI Marketing  
[marketing@grason-stadler.com](mailto:marketing@grason-stadler.com)



---

---

---

---

---

---

---

---