

rehAB Made Easy

Advanced Bionics rehAB on the Go!!

Karalee Needelman, MS-CCC-SLP, LSLS Cert AVT

<u>Karalee.Needelman@advancedbionics.com</u>

Cochlear Implant Consumer Specialist


Advanced Bionics


 $Any \ opinions \ of \ non-AB \ employees \ are \ their \ own \ and \ not \ those \ of \ the \ company. \ This \ we binar \ will \ be \ recorded \ for \ later \ viewing.$

Housekeeping

- You must stay logged in for the duration of this course in order to be eligible to earn CEU credit.
- This course is offered for Continuing Education Units (CEUs) from AudiologyOnline if you are a CEU Total Access member.
- Be sure to take the exam following course completion to earn credit.
- Need Technical Support?
 Contact AudiologyOnline at <u>1-800-753-2160</u>



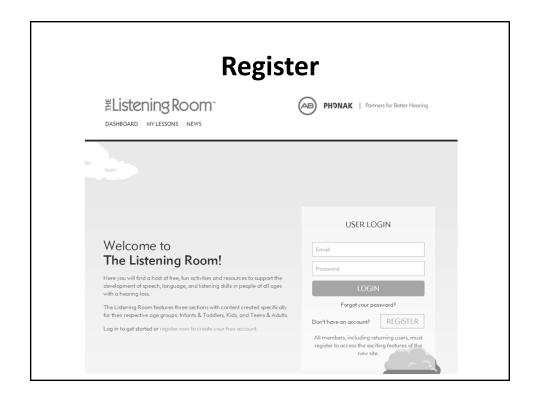

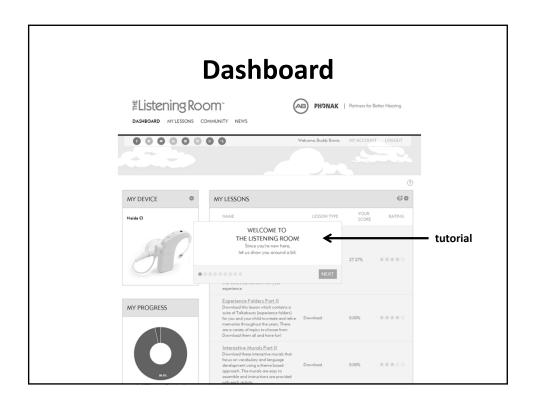

Lesson versus Activity


 <u>Lesson</u>: a group of activities that can be used to teach a therapy target


Interactive Murals Part II
Download these interactive
murals that focus on vocabulary
and language development
using a theme based approach.
The murals are easy to assemble
and instructions are provided
with each activity.


• Activity: individual exercises that are included in a Lesson





LITTLE RED RIDING HOOD: MURAL

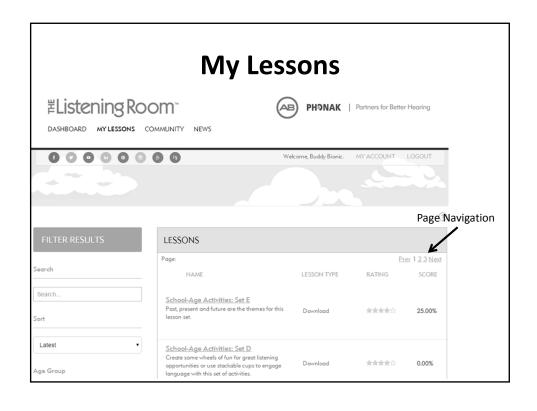

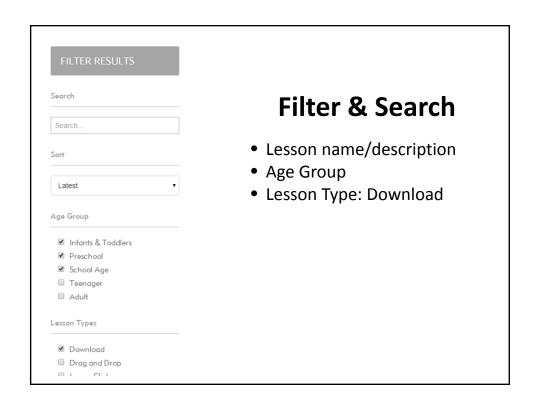

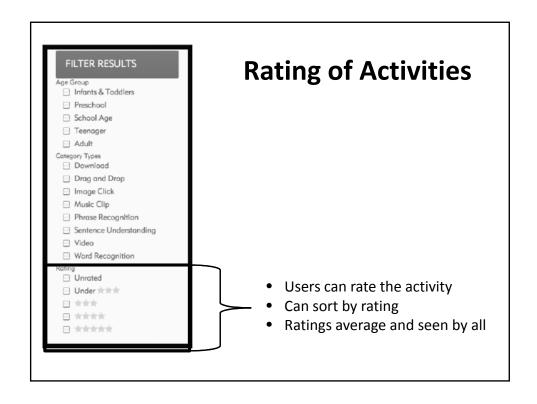

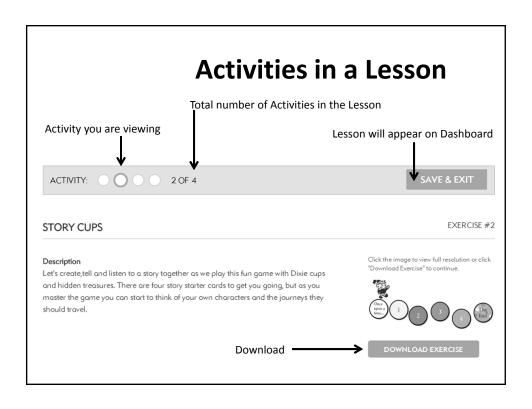

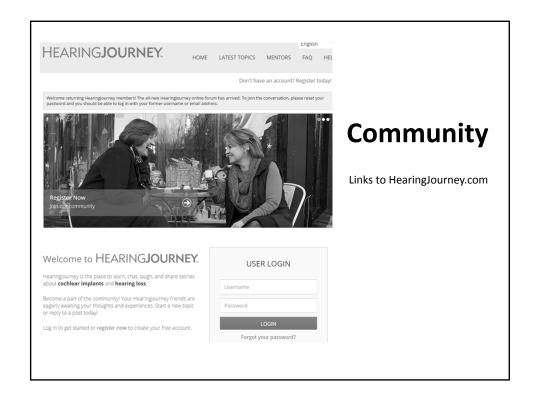


NEWS Welcome on board Phanok joins The Listening Room. All is proved to permer with Phanok to a goard the reach of The Listening Room rovine rehablishing who whose the Visiting Room who which dynamic plasterium is designed to support underly visiting and rehablishing for listenses of dogs, no matter who degree of hereing loss they experience or what bind of hearing device they use. Read Manue New Tools for Schools Troubleshooting Guides for the Nodds CI Q70 Videos and fire and fourhoodable troubleshooting guides are now a calledia. Read Manue Get Connected to AB – Telephone Tolk Connect with AB is an exceled colline velbor series for condidates, recipients and prefessionals to interest visit. All contents of the series to understand Aborace disnices heaving pick-indegrin the real world. Read Manue rehAB Mode Eary – Learn From An Expert rehAB On the COI is so notine velbane for professionals that will hely you learn here not access, neighbor, and use for professionals that will hely you learn here not access an inject, and use for professionals that will hely you learn here not accesses, neighbor, and use Table. Read Manue Loud & Clear Developed the en-event sizes of Loud & Clear is Advanced Binics' rehabilishing periodical. Read Manue FDA Approves the AppenCase Accessory for the Node CI Q70 Sound Protessor! The robust AppoCase* "accessory has been developed to provide the

Karalee Needelman, MS-CCC-SLP, LSLS Cert AVT <u>Karalee.Needelman@advancedbionics.com</u> Cochlear Implant Consumer Specialist Advanced Bionics







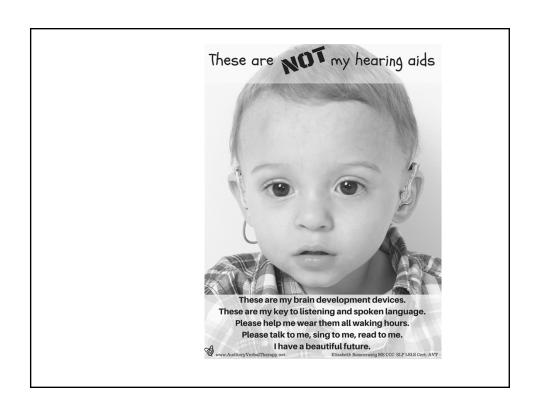

Research

Therapy Targets and Carry Over

- Exercises/activities to target skills may be some what arbitrarily derived
- To make a learned skill FUNCTIONAL, it must be integrated into REAL-LIFE situation
- Carry-over occurs when adult planned activity is more similar to every day life
- Infants, toddlers, and children learn language most efficiently through consistent and continual meaningful interactions in a supportive environment with significant caretakers

Research


Therapy Targets and Carry Over

- Working in partnership should result in the most effective intervention with very young children
- We must teach or give tools to the parents so they know to encourage and expect the child to demonstrate the same ability in the course of NORMAL every day events and activities
- We have to make carry-over easy and do-able for parents

Research

Therapy Targets and Carry Over



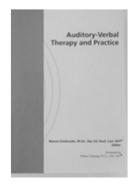

Listening and Spoken Language Knowledge Center and sold the professionals who support them

Core LSLS Principles by Doreen Pollack



- 1. Early diagnosis of hearing loss & immediate audiologic management and Auditory Verbal Therapy.
- 6. Guide & coach parents to integrate listening and spoken language into all aspects of the child's life.



- 2. Recommend use of appropriate, state-of- 7. Guide & coach parents to use natural the-art hearing technology.
 - developmental patterns of audition, speech, language, cognition, and communication.



3. Guide & coach parents in developing listening and spoken language by using hearing.

participate as primary facilitators in their child's listening and spoken language

4. Guide & coach parents to actively

development.

8. Guide & coach parents to help their child self-monitor spoken language through listening.



9. Administer ongoing diagnostic assessments to monitor progress and to evaluate the effectiveness of the plans for the child and family.



- 5. Guide & coach parents to create environments that support listening for the acquisition of spoken language throughout the child's daily activities.
- 10.Promote education in regular schools with peers who have typical hearing and with appropriate services from early childhood onwards.

www.ListeningandSpokenLanguage.org

The new Listening Room Carry over made easy!




- Activities for all ages and stages
- Free!
- · Sort/filter by age
- Sort/filter by type
- Rate activities

One theme... Infants and Toddlers through School Aged

Goldilocks and the Three Bears



Where do we start? Infants and Toddlers and New Listeners **Educational Articles:** Welcome to the EListening www.Hearing.journey.com Room Good and Not So **#Listening Listening Room Good Sound** by Dave Sindrey, M.Cl.Sc. Cert.AVT® The Learning EListening www.HearingJourney.com 置Listening www.HearingJourney.com Room **Tools for Schools** to Listen Sounds **Hearing Age** 置Listening www.HearingJourney.com Room 置Listening www.HearingJourney.com Room **Brothers and Sisters**

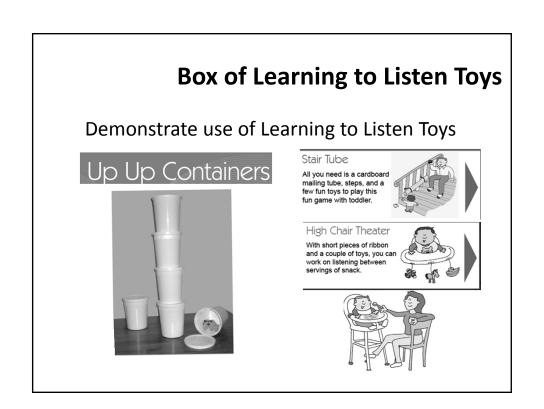




Moving From the Known to the UNKNOWN

What learning to listen sounds/ sound associations can we find?



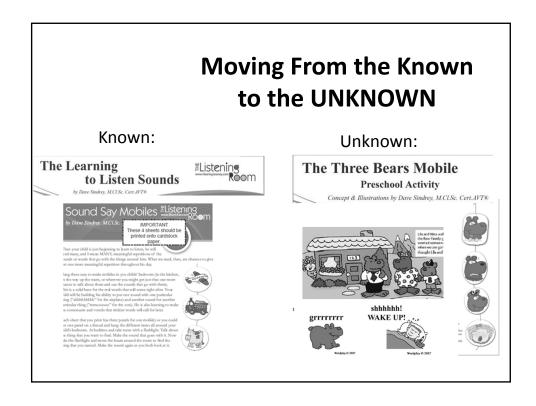






Preschool



Targets

- Responds appropriately to statements or questions
- Listens to a familiar story and answers questions
- Attempts to retell story in own words

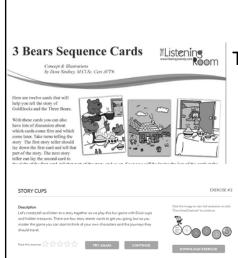

School Age

Targets

- Identifies an object or activity based on clues
- Sequences story
- IDs complex sentences in context
- Produces complex sentences with picture cues
- Tense, pronouns, prepositions

More Carry Over Activities/Themes

- Home:
 - In the Kitchen
 - Around the house





- Food: What's for Lunch?
- Forest Animals









DASHBOARD MY LESSONS COMMUNITY NEWS

Educational Articles

http://thelisteningroom.com/lessons/view/21

- Welcome to the Listening Room
- The Learning to Listen Sounds

ZListening Room[®]



DASHBOARD MY LESSONS COMMUNITY NEWS

Infants & Toddlers

Infant & Toddler Activities http://thelisteningroom.com/lessons/view/25

- Up UP containers
- Stair Tube
- Meal Mats
- High Chair Theatre





DASHBOARD MY LESSONS COMMUNITY NEWS

Preschool & School Age

Three Bears Theme Unit https://thelisteningroom.com/lessons/view/118

- Three Bears Mobile: https://thelisteningroom.com/lessons/view/113
- The Three Bears Mural: http://thelisteningroom.com/lessons/view/116
- Three Bears Sequence Cards: https://thelisteningroom.com/lessons/view/85

♯Listening Room[™]



DASHBOARD MY LESSONS COMMUNITY NEWS

School Age

Listening Ladder https://thelisteningroom.com/lessons/view/103

School Age Activities Inside and Outside https://thelisteningroom.com/lessons/view/92

• In the Kitchen Activity

School Age Activities Foodhttps://thelisteningroom.com/lessons/view/93

• What's for Lunch Activity

School Age Activities: Set D https://thelisteningroom.com/lessons/view/96


Story Cups Activity


Forest Friends: https://thelisteningroom.com/lessons/view/114


• Forest Friends Book Activity






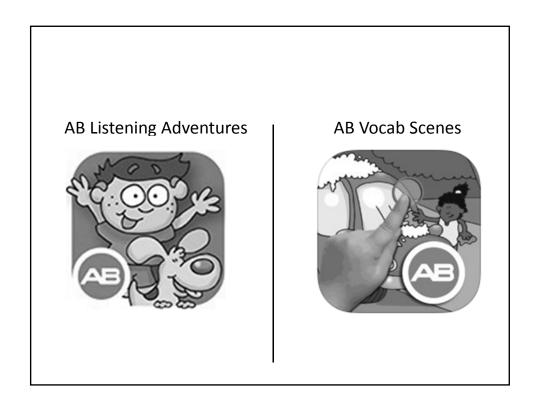

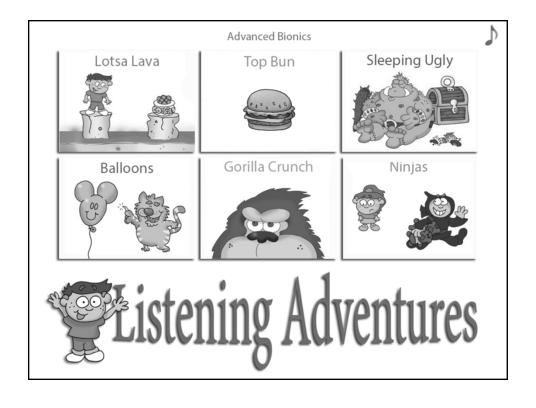


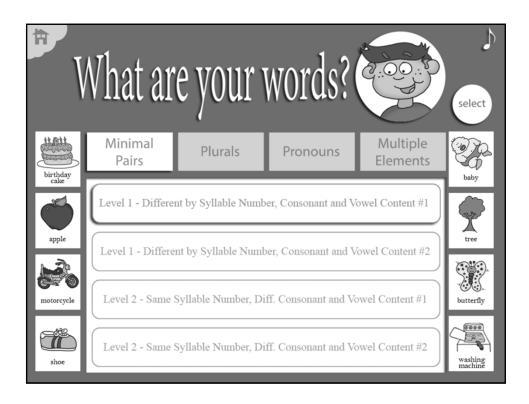




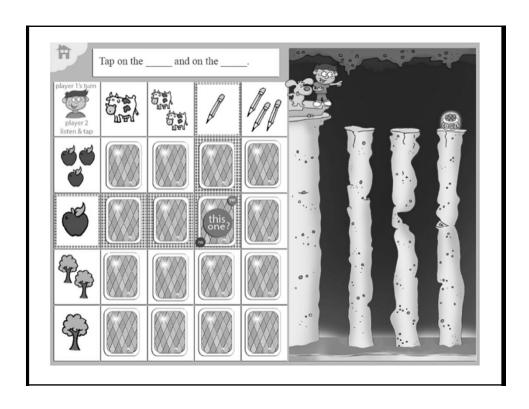



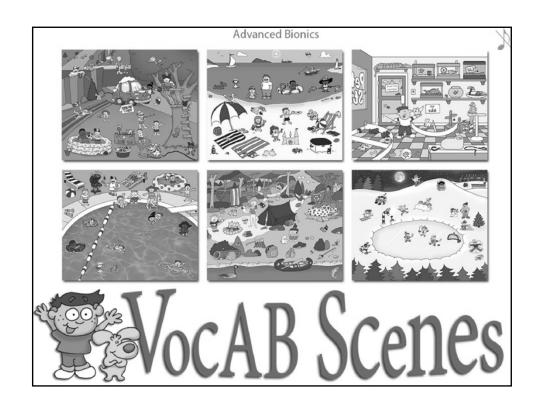

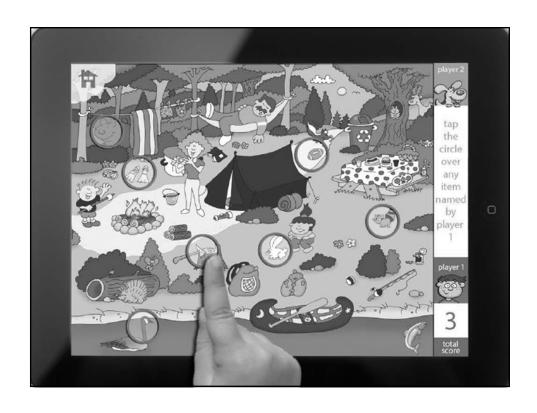







Join the Conversations Follow AB



facebook.com/advancedbionics

Product & techy tips, recipient stories, announcements, industry news



twitter.com/advancedbionics

Product & techy tips, recipient stories, announcements, industry news



youtube.com/advancedbionics

Recipient stories, techy animations, product how to's



linkedin.com/company/advanced-bionics

Career opportunities, company announcements, professional news



pinterest.com/advancedbionics

AB culture, rehab, industry awareness, recipient stories pins




instagram.com/advancedbionics

Buddy & Melody travel pics





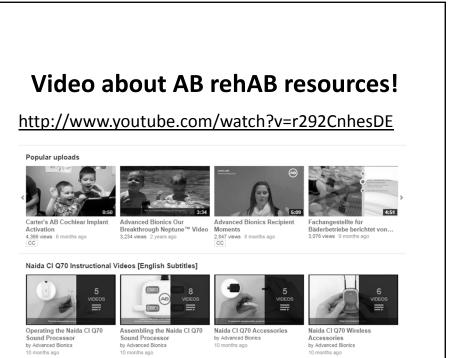

How to Videos



http://www.youtube.com/watch?v=szAtDQ9sJOU

Listening Checks are Essential!

Buddy makes Listening FUN and EXCITING





http://advancedbionics.com/content/ab/us/en/multiusedcontents/forms/banana_drop.html



Karalee Needelman, MS-CCC-SLP, LSLS Cert AVT
Cochlear Implant Consumer Specialist
Advanced Bionics
Karalee.needelman@advancedbionics.com

