

CONTINUING THE UNIQUE JOURNEY NEW PRODUCTS IN THE WIDEX UNIQUE FAMILY


Leslie Staverman, AuD
Director of Educational Development
Professional Education Manager
Widex USA



WIDEX UNIQUE
LIVE LIFE LARGER

LEARNER OUTCOMES

1. Participants will be able to list the additional models being released to the UNIQUE product line.
2. Participants will be able to describe basic changes made in Compass 2.1.
3. Participants will be able to discuss the benefits Unique will have to their patients.

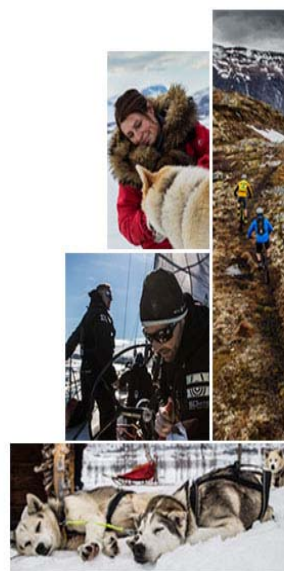

WIDEX UNIQUE
LIVE LIFE LARGER

EVERY SITUATION IS **UNIQUE**



WIDEX UNIQUE
LIVE LIFE LOUD

THE WIDEX LEGACY



WIDEX UNIQUE
LIVE LIFE LOUD

INTRODUCING WIDEX **UNIQUE**



WIDEX **UNIQUE**
LIVE LIFE LOUD

UNIQUE NUMBERS




U-PLATFORM

LINEAR INPUT LEVEL	113 dB SPL
SAMPLING FREQUENCY	33.1 kHz
BIT DEPTH	18 bit
LINEAR DYNAMIC RANGE	108 dB
PROCESSING POWER	600 Million Operations/sec
MEMORY	1Mbit
INTEREAR	21 Hz
DELAY	1.3-5 ms
CURRENT CONSUMPTION	< 1.00mA



WIDEX **UNIQUE**
LIVE LIFE LOUD

BETTER AND BROADER INPUT DYNAMIC RANGE

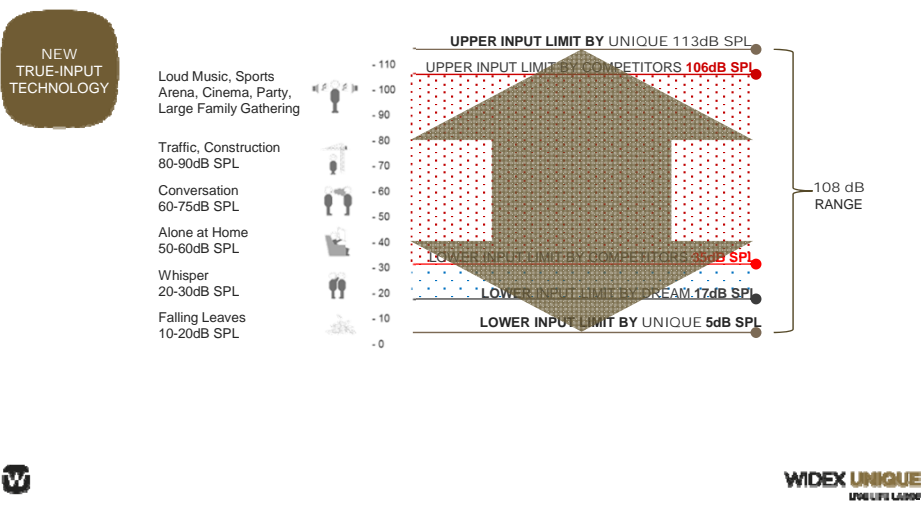

FULL FLEXIBILITY NO SACRIFICES



WIDEX UNIQUE
LIVE LIFE LOUD

WHAT'S NEW

NEW BTE
MODEL

EASYWEAR
THIN-TUBE

COM-DEX
UPDATE

COMPASS
GPS 2.1
UPDATE



WIDEX UNIQUE
LIVE LIFE LOUD

UNIQUE FASHION MINI



WIDEX UNIQUE
LIVE LIFE LOUD

NEW BTE STYLE UNIQUE FASHION MINI

- Small size BTE
- 4 performance levels
 - (440, 330, 220 110)
- Fitting range: 0-95dB HL
- 312 battery
- Configurable push button
- WidexLink & DEX compatible
- EASYWEAR Thin Tube compatible




WIDEX UNIQUE
LIVE LIFE LOUD

EASYWEAR THIN TUBES UNIQUE FASHION MINI (U-FM)



UNIQUE Fashion Mini

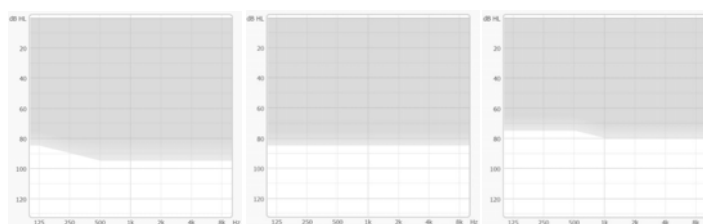

Standard hook and ear mold

THIN TUBE 1.4

THIN TUBE 0.9




WIDEX UNIQUE
LIVE LIFE LOUD

UNIQUE FASHION MINI OUTPUT COMPARISON



U-FM maximum output

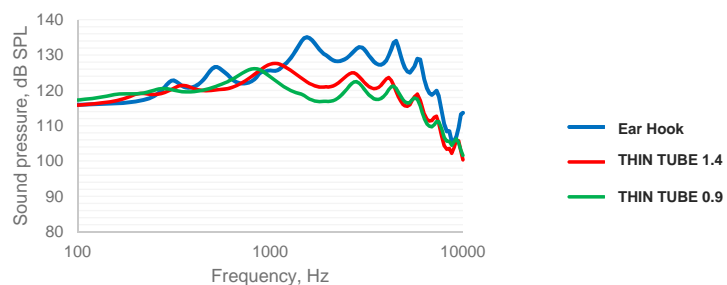

WIDEX UNIQUE
LIVE LIFE LOUD


UNIQUE FASHION MINI COLORS



WIDEX UNIQUE
LIVE LIFE LOUD

EASYWEAR THIN TUBES

- Two new EASYWEAR Thin Tubes:
 - EASYWEAR Thin Tube 0.9 mm diameter
 - EASYWEAR Thin Tube 1.4 mm diameter
- NanoCare Standard
- EASYWEAR instant and custom tips
- Five lengths (0 to 4) with right and left version

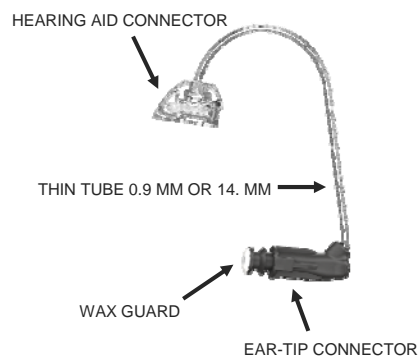

WIDEX UNIQUE
LIVE LIFE LOUD

EASYWEAR THIN TUBE 0.9 VS THIN TUBE 1.4



WIDEX UNIQUE
LIVE LIFE LOUD

EASYWEAR THIN TUBE VS RECEIVER



THIN TUBE 1.4 vs S-RECEIVER



THIN TUBE 1.4 vs M-RECEIVER



WIDEX UNIQUE
LIVE LIFE LOUD

EASYWEAR EAR-TIPS

- EASYWEAR Instant Ear-tips must be used
 - Not compatible with older Ear-tips



Open Domes



Tulip Domes



Double Domes



- New Custom Ear-tip for EASYWEAR THIN TUBE
 - Custom Ear-tip 0.9 / 1.4



WIDEX UNIQUE
LIVE LIFE LOUD

EASYWEAR THIN TUBE



1. Attach the "THIN TUBE REPLACER" as shown



2. Lift the "THIN TUBE REPLACER" in the direction of the arrow



2. Attach a new THIN TUBE and click the THIN TUBE on the hearing aid in the direction of the black arrow



WIDEX UNIQUE
LIVE LIFE LOUD

EASYWEAR THIN TUBE REMOVE CUSTOM EAR-TIP



1. Remove NanoCare

2. Use the special too to push out the THIN TUBE




WIDEX UNIQUE
LIVE LIFE LOUD

EASYWEAR THIN TUBE BENEFITS

- Patent-pending design as existing EASYWEAR receivers
- Compatible with the EASYWEAR ear-tips
- No sideways movement out of ear canal
- Fits people with a smaller ear canal
- Stays in the ear without anchor



WIDEX UNIQUE
LIVE LIFE LOUD

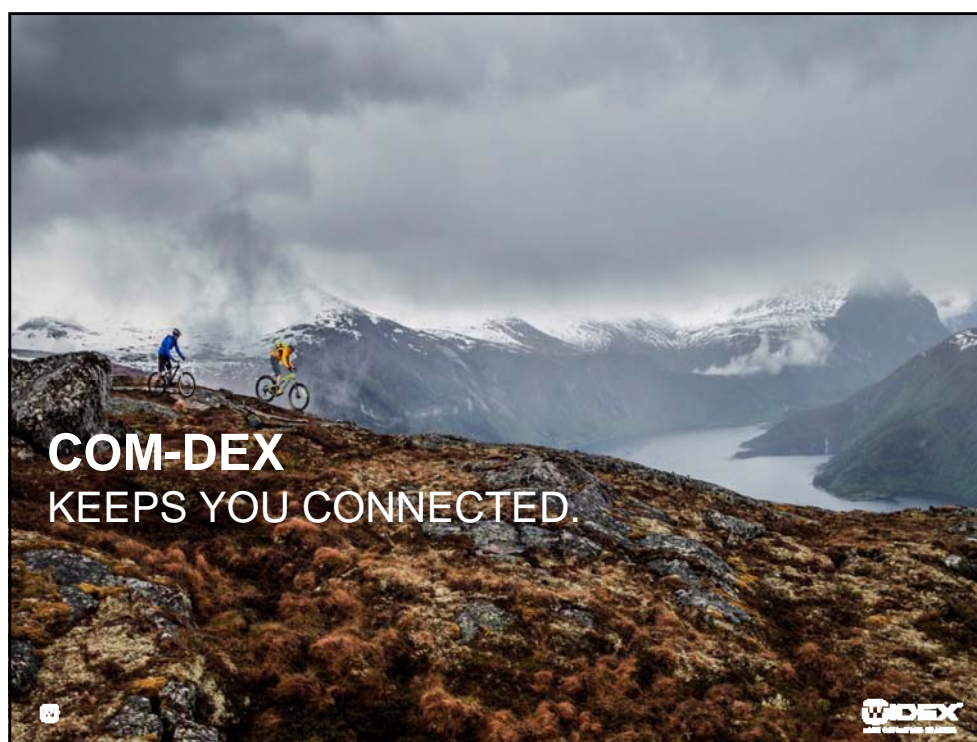

WIDEX
LIVE LIFE LOUD

COM-DEX – SEAMLESS CONNECTIVITY

- True hands-free solution with up to 8 hours streaming
- Attractive design - Stylish colours for more personal feeling
- Stream quality sound from all Bluetooth mobile devices
- Possible to connect work phone and private phone to one COM-DEX
- Use COM-DEX App with your iPhone or Android smartphone
- Compatible with CLEAR, SUPER, DREAM and UNIQUE



WIDEX UNIQUE
LIVE LIFE LOUD

COM-DEX END USER BENEFITS



WIDEX UNIQUE
LIVE LIFE LOUD

QUICK ACCESS – CONTROL YOUR HEARING AIDS

- Use App's to benefit more from your Widex hearing aids
- COM-DEX App (ver. 1.2.0) on Apple Watch
- Discreetly control your hearing aids right on your wrist
- Android widget gives you the COM-DEX App directly on your home screen
- Choose your own background – available in both iOS and Android




WIDEX UNIQUE
LIVE LIFE LOUD

COM-DEX APP – SCREEN SHOTS



Easy and attractive design

App help to connect and adjust

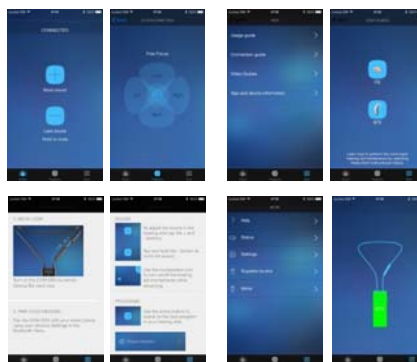

Help and more

Battery, about the app and setup



WIDEX UNIQUE
LIVE LIFE LOUD

COM-DEX "BOOST"



- Apple Watch App
- Android Widget
- Bluetooth connection w/ 2 phones (devices)
- Performance maintenance
 - Compatibility updates for Android
 - Improved microphone sensitivity
 - Com-Dex Firmware update tool
 - Extra MUTE Option



WIDEX UNIQUE
LIVE LIFE LOUD

WIDEX CROS



WIDEX UNIQUE
WIDEX DREAM




All Levels (440 / 330 / 220 / 110)



WIDEX UNIQUE
LIVE LIFE LOUD

WIDEX CROS PORTFOLIO

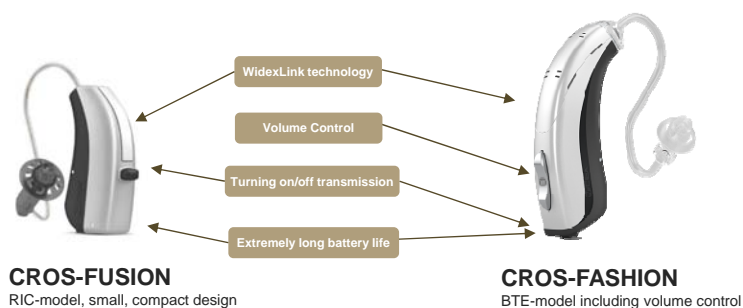


WIDEX UNIQUE
LIVE LIFE LOUD®

WIDEX CROS AND BATTERY LIFETIME


LOW BATTERY CONSUMPTION

- Uses three times less battery power than competitor's wireless CROS solutions

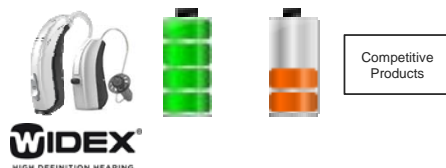

WIDEX UNIQUE
LIVE LIFE LOUD®


WIDEX CROS - EVIDENCE

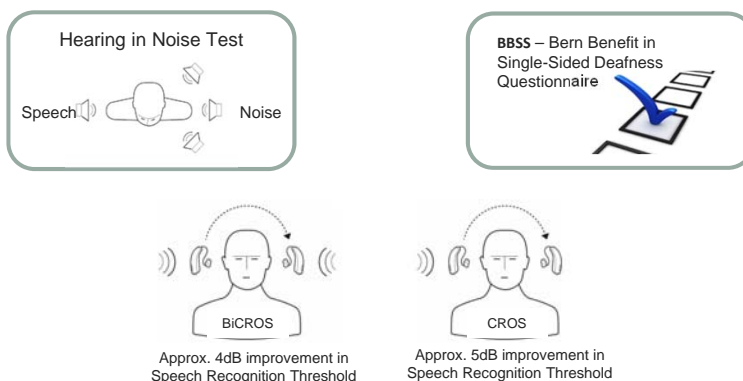

WIDEX UNIQUE
LIVE LIFE LOUD

WIDEX COMPASS™ GPS FITTING SIMPLIFIED

- Automatic Update – May 2nd
- Configurable Push-Button
 - Fashion Mini & Fusion
 - Unique Only
- Rename Programs
- Solution Guide
- Hearing Aid “User Guide”

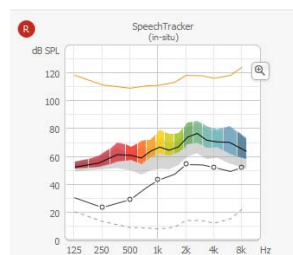

WIDEX UNIQUE
LIVE LIFE LOUD

UNIQUE FASHION MINI CONFIGURABLE PUSH BUTTON

- The UNIQUE FASHION MINI has a new configurable push button
- Always binaurally coordinated
- Choose between 5 different configurations
- Also available on the UNIQUE FUSION



Button functionality

- ☒ Program change
- ☐ Preference control
- ☐ Preference control and mute
- ☐ Preference control and program change
- ☐ No functionality



WIDEX UNIQUE
LIVE LIFE LARSEN

THANK YOU

Questions:

AOINQUIRY@widex.com

More Information:

- www.widexpro.com
- <https://learn.widex.pro/learn>



WIDEX UNIQUE
LIVE LIFE LARSEN