Practical Audiometry:

Comprehensive review of Astera in OTOsuite

AudiologyOnline Presentation

This presentation is brought to you by Audiology Systems and Ototmetrics in cooperation with AudiologyOnline

One hour of CEU has been submitted for AAA, IIHIS, BAA, CAA, SAC and the Kansas Speech-Language Hearing Association.

If at any time you have difficulties hearing or seeing this presentation please contact AO at: **800-753-2160**

Course Outline

- Brief Company Overview
- Brief Product Solutions Overview
- Introduction to OTOSuite
- Astera² Audiometry Basics
- Performing Pure Tone Audiometry with Astera²
- Performing Speech Audiometry with Astera²
- Special Tests
- Reporting
- Q&A

> otometrics & audiologysystems


Who Are We?

Otometrics – Develops, manufactures and markets computer based audiological, otoneurologic & vestibular instrumentation in more than 70 countries globally. Product brands include MADSEN, AURICAL, ICS, & OSCILLA

<u>Audiology Systems</u>— National partnership of industry professionals, audiologists and local audiology & vestibular experts who work together to distribute products, educate and serve as a resource to our customers



Our Product Line

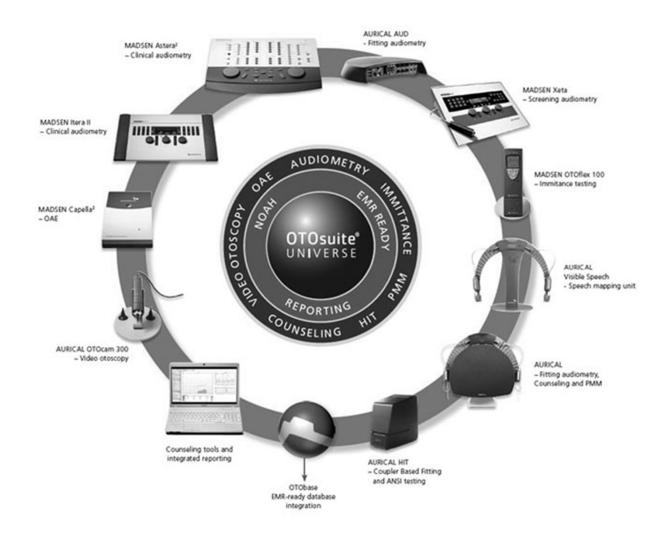

5

MADSEN Astera² Description

- The Astera² is a 2-channel, PC-based audiometer
- With Astera² you can perform all the standard audiometric tests including tone and speech testing as well as special tests such as tinnitus evaluation testing, high frequency tone testing and dedicated pediatric testing
- Astera² can be operated from your PC's keyboard or mouse or from the Audiometer Control Panel (ACP) with the OTOsuite[®] software acting as the display
- Integrates seamlessly with NOAH & is EMR compatible

www.audiologysystems.com



MADSEN Astera² Description

- Three Main Components of Astera²
- 1.




- 1. Audiometer Hardware
- Optional Audiometer Control Panel (ACP)

2.



3. OTOsuite Software



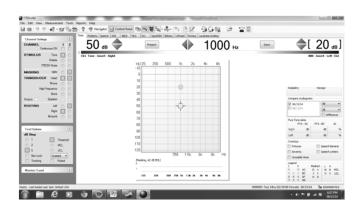


Basic Operation Preferences

Audiometer Control Panel (ACP) –
One button = one function

 Mouse – On screen controls. Easy functionality of point an click

 Keyboard -- Extensive shortcuts (see handout)



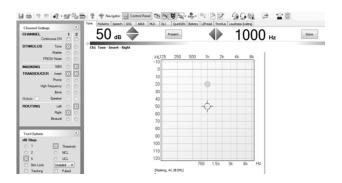




Basic Operation Preferences – ACP



Basic Operation Preferences – Keyboard Shortcuts

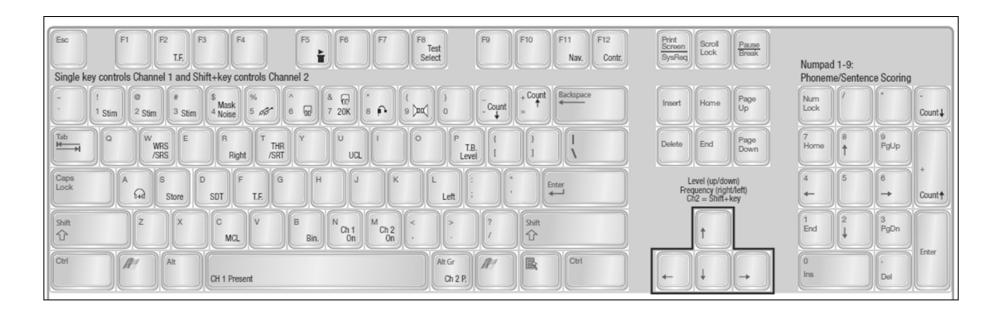



NAVIGATION OF AUDIOMETRY MODULE

We will now switch to a mode where you will see my screen. We will share a view of the screen so I can show you how to work with the Astera.