

# Pre-Operative Candidacy Evaluation

## Part 1: Hearing Aids and Bone Conduction Thresholds

### 1 Patient information

Name: \_\_\_\_\_

Date: \_\_\_\_\_ Age: \_\_\_\_\_

### 2 Candidacy Considerations

#### Type of Hearing Loss

- ☐ Conductive    ☐ SSD  
☐ Mixed        ☐ Other: \_\_\_\_\_

Duration of Overall Hearing Loss \_\_\_\_\_

Etiology \_\_\_\_\_

#### Conductive/Mixed Losses Medical Indications

- ☐ Cholesteatoma  
☐ Chronic mastoiditis/mastoid cavity  
☐ Chronic otitis media  
☐ Conditions precluding the use of conventional hearing aid  
☐ Congenital aural atresia  
☐ Draining ears  
☐ Ear canal stenosis  
☐ External otitis  
☐ Genetics  
☐ Ossicular disease  
☐ Otosclerosis  
☐ Other middle ear dysfunctions  
☐ Syndromic hearing losses

#### Single Sided Deafness Indications

- ☐ Acoustic neuroma  
☐ Genetics  
☐ Meniere's disease  
☐ Neurological degenerative disease  
☐ Ototoxic treatments  
☐ Sudden idiopathic deafness  
☐ Surgical

[www.Cochlear.com/US](http://www.Cochlear.com/US)

#### Cochlear Americas

13059 East Peakview Avenue  
Centennial, CO 80111 USA  
Telephone: 1 303 790 9010  
Support: 1 800 483 3123

#### Cochlear Canada Inc.

2500-120 Adelaide Street West  
Toronto, ON M5H 1T1 Canada  
Support: 1 800 483 3123

©Cochlear Limited 2017. All rights reserved. Hear now. And always and other trademarks and registered trademarks are the property of Cochlear Limited or Cochlear Bone Anchored Solutions AB.




BUN412 ISS4 APR17

### 3 Satisfaction with Hearing Aids

	Very Satisfied	Satisfied	Neutral	Dissatisfied	Very Dissatisfied
Noise Environments	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Quiet Environments	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Small Groups (3-5 people)	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Large Groups (5+ people)	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>
Listening at a distance (lectures, church, etc)	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>	<input type="radio"/>

If the majority of satisfaction ratings are "neutral" or "dissatisfied", proceed with the rest of evaluation and determine implant candidacy.

### 4 Candidacy Criteria

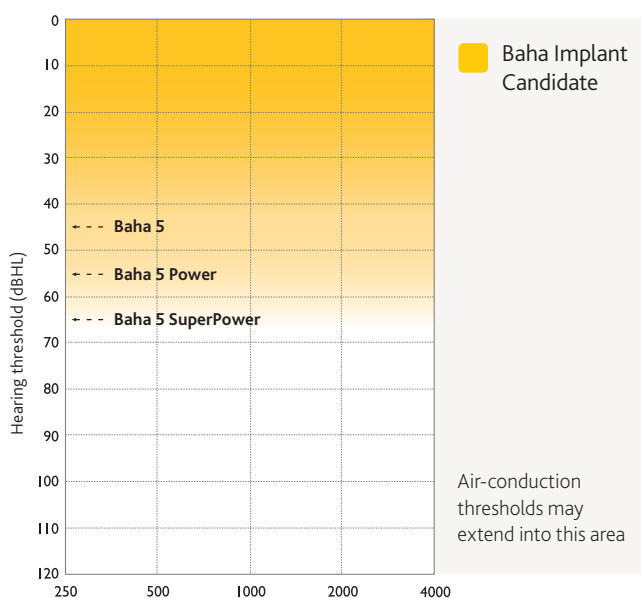

#### Please Check One

- ☐ Bone-conduction thresholds  $\leq 45$  dB HL averaged across 500, 1000, 2000, and 3000 Hz  
☐ Bone-conduction thresholds  $\leq 55$  dB HL averaged across 500, 1000, 2000, and 3000 Hz  
☐ Bone-conduction thresholds  $\leq 65$  dB HL averaged across 500, 1000, 2000, and 3000 Hz

Continued on reverse side ↩



Hear now. And always

Cochlear®

# Pre-Operative Candidacy Evaluation

## Part 2: Device Determination

### 1 Recommendations

	Hearing Aid	Baha 5	Baha 5 Power	Baha 5 SuperPower
Left Ear	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
Right Ear	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>

### 2 Device Benefits



#### Baha 5 Benefits

- Fully programmable, premium head-worn sound processor with Bluetooth® Smart technology
- Made for iPhone Hearing Device
- 2.4 GHz Wireless technology
- 17 channel sound analysis with 4 user-defined programs
- Scene Classifier
- Position Compensation
- Control Sync for bilateral synchronization
- Dual Track Feedback Manager
- Dedicated fitting rationales
- Wireless fitting with Airlink
- Compatible with FM and digital wireless ALD systems



#### Baha 5 Power Benefits

- Fully programmable, power head-worn sound processor with Bluetooth® Smart technology
- Made for iPhone Hearing Device
- 2.4 GHz Wireless technology
- 17 channel sound analysis with 4 user-defined program
- Volume rocker to adjust volume settings
- Visual indicator
- Scene Classifier II controlling:
  - Noise Manager II
  - Active Balanced Directionality
  - Active Gain
- Position Compensation II
- Control Sync for bilateral synchronization
- Dual Track Feedback Manager
- Dedicated fitting rationales for mixed loss, conductive loss and SSD
- Patient optimized fitting with Baha Fitting Software (5.2 or later)
- Wireless fitting with Airlink
- Compatible with FM and digital wireless ALD systems through the Cochlear Wireless Mini Microphone 2+



#### Baha 5 SuperPower Benefits

- Fully programmable, superpower head worn sound processor with Bluetooth® Smart technology
- Made for iPhone Hearing Device
- 2.4 GHz Wireless technology
- 17 channel sound analysis with 4 user-defined program
- Scene Classifier II controlling:
  - Noise Manager II
  - Active balanced directionality
  - Active gain
- Pinna compensation
- Control Sync for bilateral synchronization
- Dual Track Feedback Manager
- Dedicated fitting algorithms for mixed loss, conductive loss and SSD
- Patient optimized fitting with Baha Fitting Software (5.1 or later)
- Wireless fitting with Airlink
- Compatible with FM and digital wireless ALD systems by connecting a body-worn receiver of the listening system to the line input of the Cochlear Wireless Mini Microphone

### 3 Candidate Evaluation



#### Pediatric Candidate Test\*

- Use age appropriate tests to evaluate audibility and speech understanding.
- Use a Baha Softband to determine the benefit of Baha.

#### Pediatric Candidate Considerations

- We recommend fitting a Baha Softband prior to surgery and until sound processor fitting.
- When demonstrating with a Baha Softband, consider using a stronger head worn sound processor than indicated by the audiogram. This provides an experience that more closely demonstrates the benefits of the implanted device.

\*The Baha must be fitted/used on a softband for recipients ages 5 and under.



#### Adult Candidate Test

- Pure tone audiometry
- Speech audiometry
- Sound field test with testband/test rod/Softband

#### Adult Candidate Considerations

- Ensure the patient's expectations are realistic.
- When demonstrating with a headband, consider trying a stronger head worn sound processor than indicated by the audiogram. This provides an experience that more closely demonstrates the benefits of the implanted device.

Hear now. And always

