



Education
The Science of Hearing Aid Fitting

Frequency Lowering Fitting & Verification

Presenter — David J. Smriga, M.A.
Senior Audiology Consultant
Audioscan Education

Audioscan® Education
© 2019 Audioscan

 **audioscan**®
Professional Verification

Learning Outcomes

After this course learners will be able to...

- Describe clearly the different methods manufacturers use to create a frequency lowering result.
- Identify when frequency lowering should be employed to enhance patient understanding of speech.
- Explain how to implement a step-by-step fitting procedure that programs frequency lowering for maximum patient benefit.

 Audioscan® Education

©2018 Audioscan

Outline For Today's Webinar

- An overview of frequency lowering and when to use it
- Types of frequency lowering approaches
- Goals of frequency lowering verification
- Frequency lowering fitting and verification protocol
- Frequency lowering and the SII score




Frequency Lowering: When to use it



Frequency Lowering

- Process of shifting a range of input frequencies into a lower range of output frequencies
- Designed to improve the audibility of sounds for listeners with high frequency hearing loss

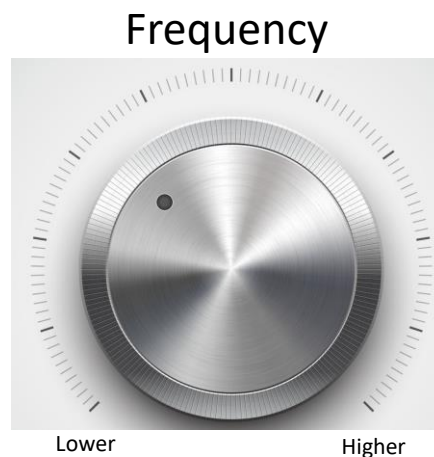


Audioscan® Education
© 2019 Audioscan.



The Concept Behind Changing Output Frequency Content

- Some hearing losses have un-aidable regions where important speech information exists
- Re-positioning input energy in these regions to regions that are aidable can provide access to these important speech cues

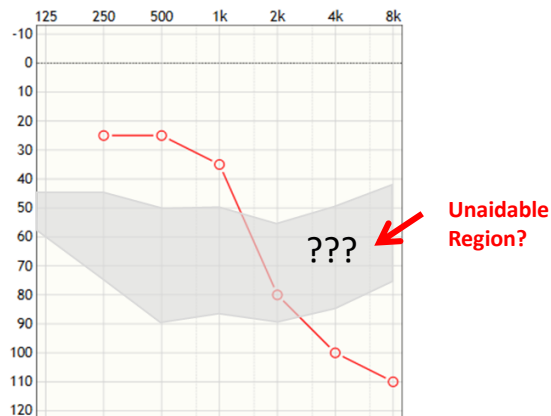


Audioscan® Education
© 2019 Audioscan.




When To Consider Frequency Lowering

- The area in the green circle has **limited audibility** (and may contain important speech cues)
- If Audibility cannot be restored
 - By adjusting gain, or
 - By selecting a different hearing aid
- This might be a time to consider frequency lowering

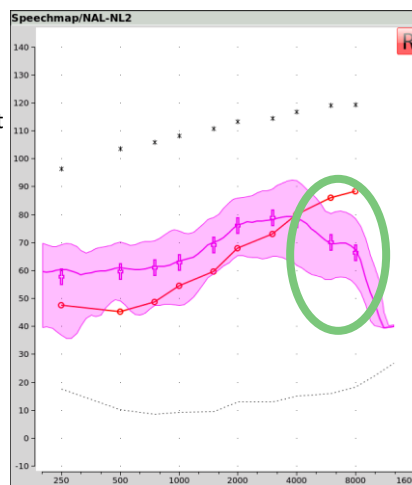

Frequency Lowering: Technological Approaches

Approach #1: Frequency Transposition

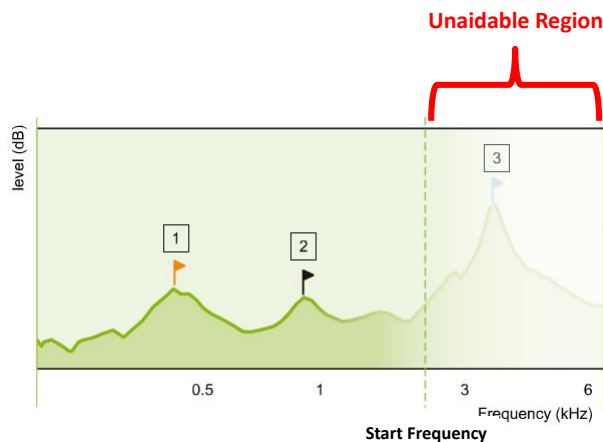


With permission from Christine Jones, Au.D., VP of Audiology for Phonak USA

Myriell Nyffeler, Speech Study Coordinator, Phonak Hearing Instruments, Switzerland, Phonak Virtual Audiology Conference, May, 2009

Audioscan® Education
© 2019 Audioscan.



Approach #1: Frequency Transposition

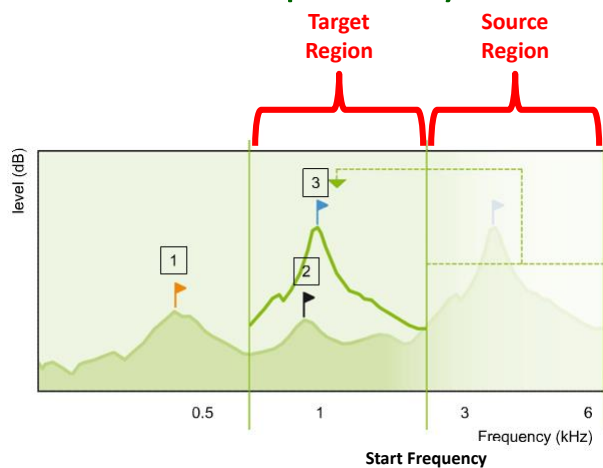

“Cuts” or “Copies” the input frequencies in the Source Region and “Pastes” them as output frequencies within the Target Region

With permission from Christine Jones, Au.D., VP of Audiology for Phonak USA


Myriell Nyffeler, Speech Study Coordinator, Phonak Hearing Instruments, Switzerland, Phonak Virtual Audiology Conference, May, 2009

Audioscan® Education
© 2019 Audioscan.



How Might This Look In A Test Box?

Consider the following input condition:

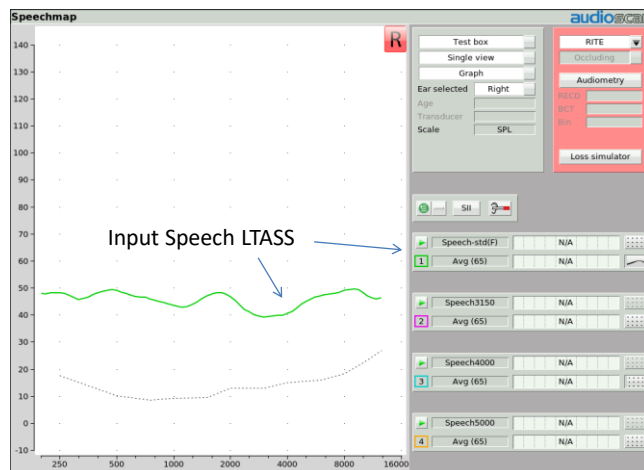


Audioscan® Education
© 2019 Audioscan.



How Might This Look In A Test Box?

Consider the following input condition:

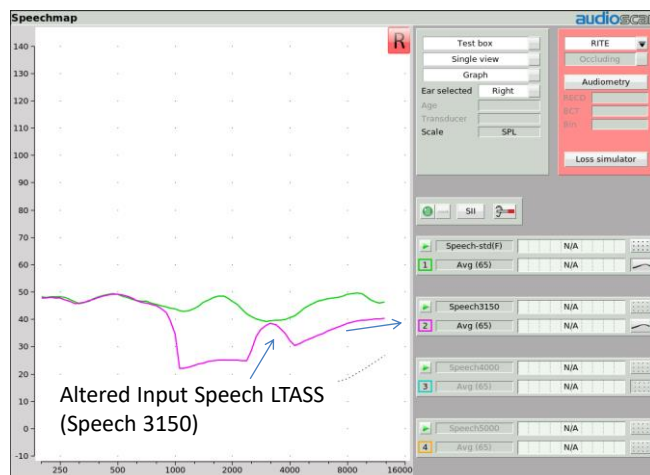


Audioscan® Education
© 2019 Audioscan.



How Might This Look In A Test Box?

Consider the following input condition:

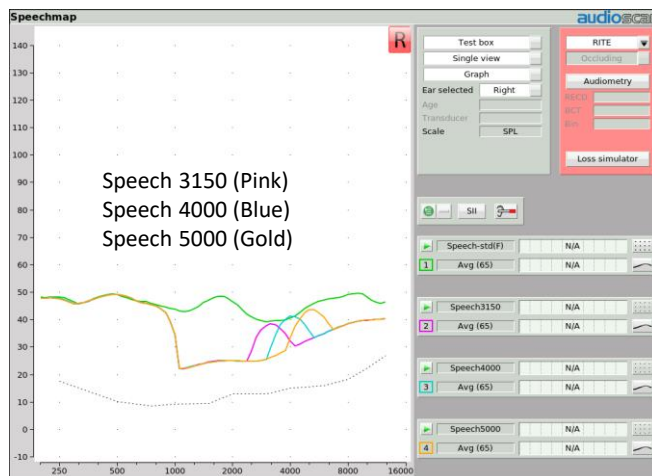


Audioscan® Education
© 2019 Audioscan.



How Might This Look In A Test Box?

Consider the following input condition:

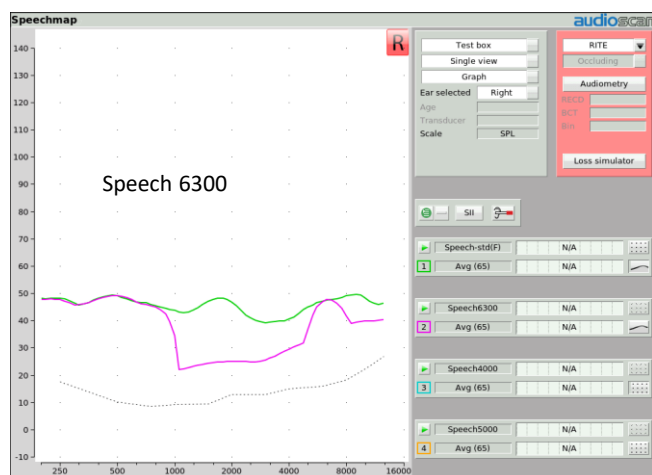


Audioscan® Education
© 2019 Audioscan.



How Might This Look In A Test Box?

Here is how these stimuli can be used to “display” frequency transposition:

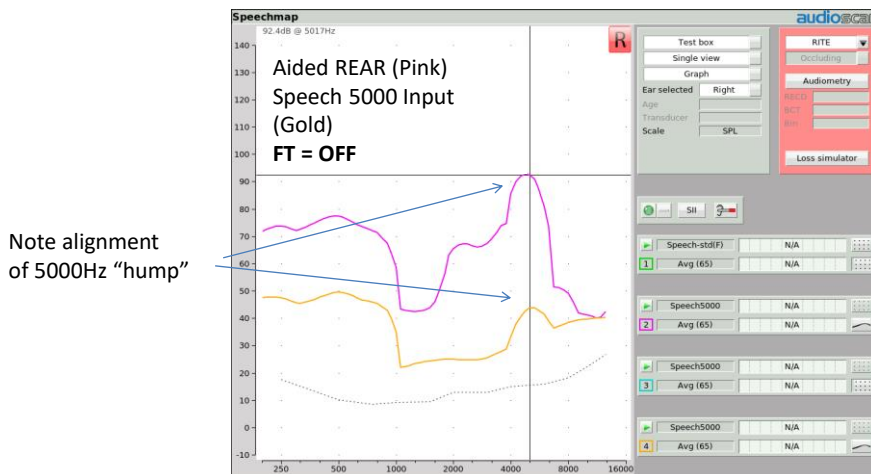


Audioscan® Education
© 2019 Audioscan.



How Might This Look In A Test Box?


Here is how these stimuli can be used to “display” frequency transposition:

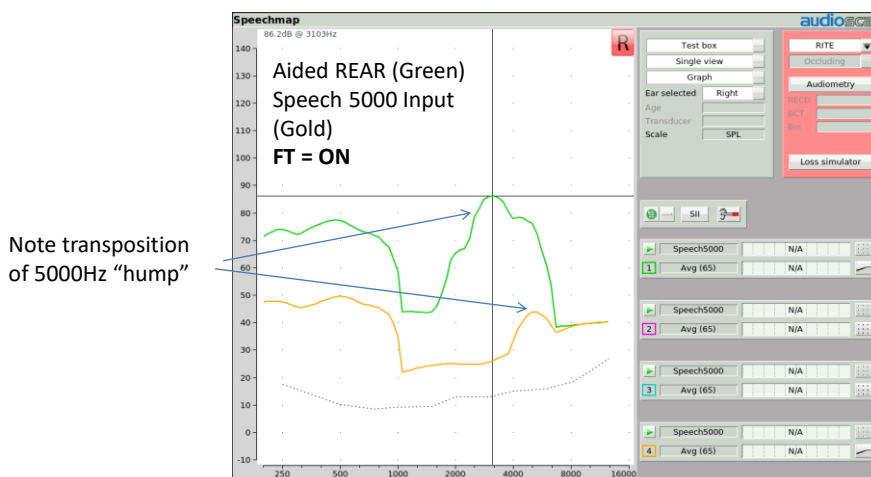

Audioscan® Education
© 2019 Audioscan.



Another Stimulus Option For FL Verification

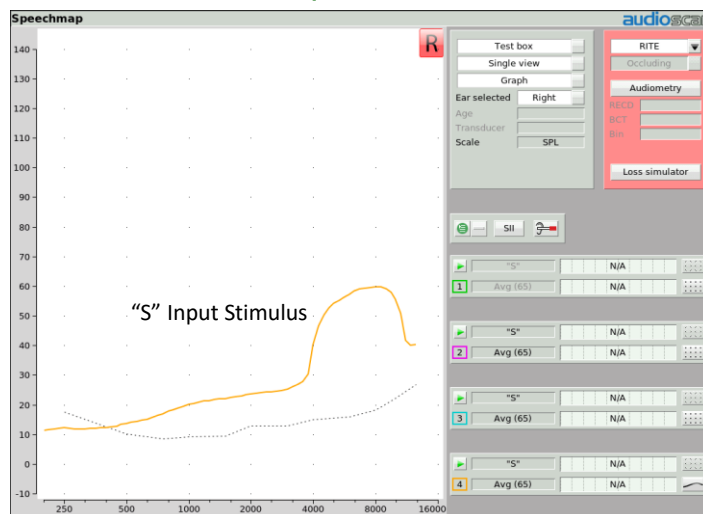


Glista, D., & Scollie, S. (2009). Modified verification approaches for frequency lowering devices. *AudiologyOnline*, Article 871. Retrieved from www.audiologyonline.com

Audioscan® Education
© 2019 Audioscan.



How Might This Look In A Test Box?

Here is how the “S” stimulus can be used to “display” frequency transposition:

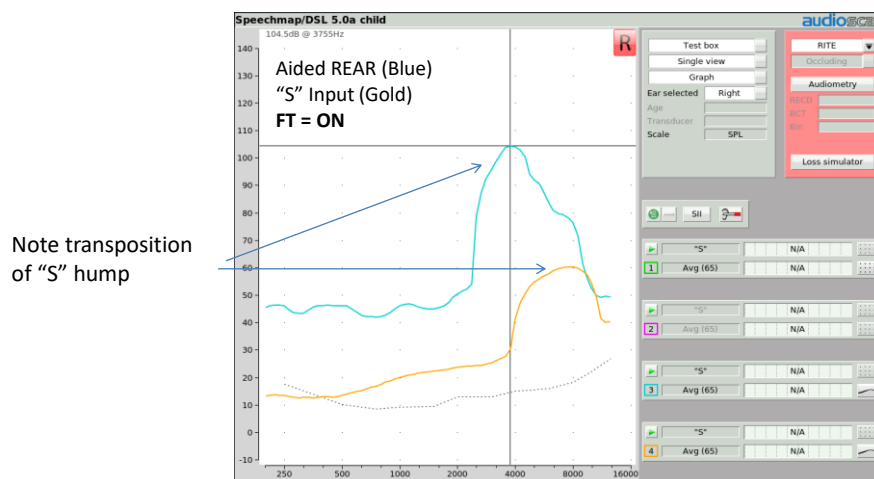

Note transposition
of “S” hump

Audioscan® Education
© 2019 Audioscan.




Frequency Transposition Can Be Found In

- Widex (Audibility Extender)
 - Use a “Cut” and “Paste” Approach
- Oticon (Speech Rescue)
 - Uses a “Copy” and “Paste” Approach



Approach #2: Frequency Compression

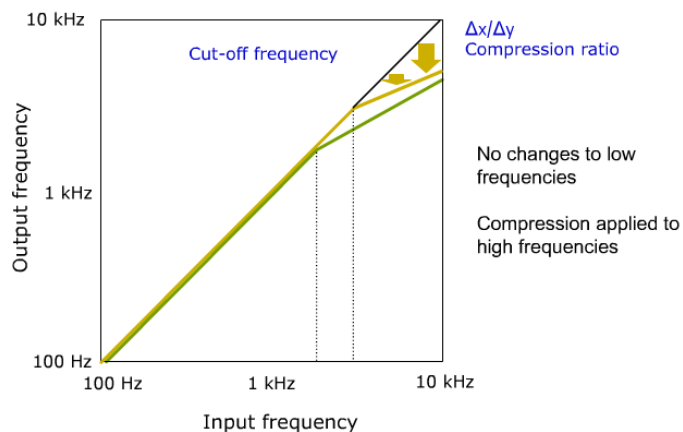


With permission from Christine Jones, Au.D., VP of Audiology for Phonak USA

Myriell Nyffeler, Speech Study Coordinator, Phonak Hearing Instruments, Switzerland, Phonak Virtual Audiology Conference, May, 2009



Approach #2: Frequency Compression

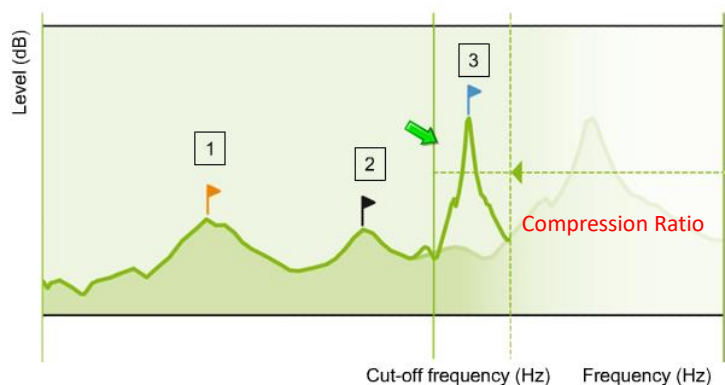


With permission from Christine Jones, Au.D., VP of Audiology for Phonak USA

Myirel Nyffeler, Speech Study Coordinator, Phonak Hearing Instruments, Switzerland, Phonak Virtual Audiology Conference, May, 2009

Audioscan® Education
© 2019 Audioscan.



Approach #2: Frequency Compression

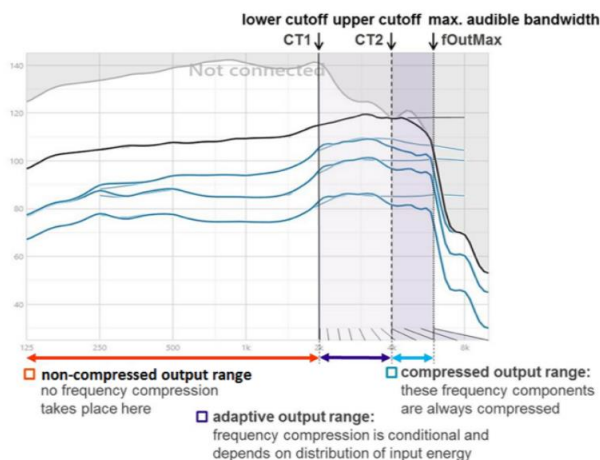

With permission from Christine Jones, Au.D., VP of Audiology for Phonak USA


<https://www.hearphonak.com/soundrecover-2-help-children-teens/>

Audioscan® Education
© 2019 Audioscan.



How Might This Look In A Test Box?

Here is how the “S” stimulus can be used to “display” frequency compression:

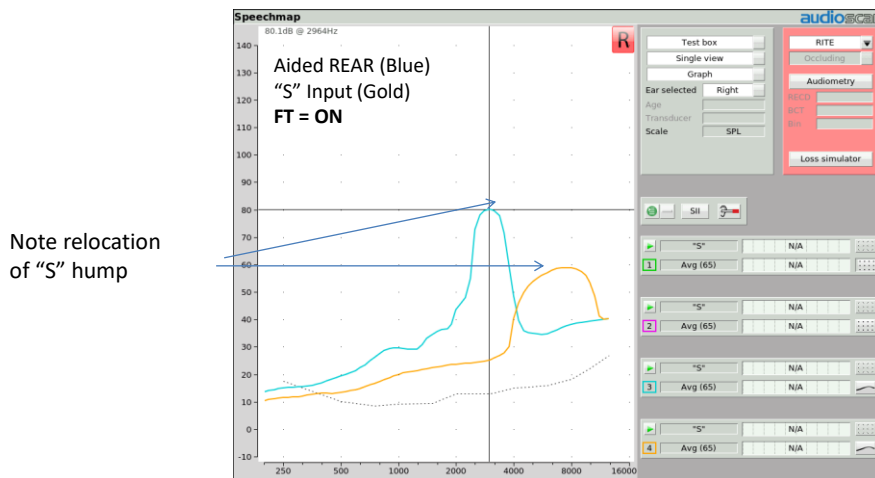

Audioscan® Education
© 2019 Audioscan.



Frequency Compression Can Be Found In

- Phonak (Sound Recover)
- Sivantos (FC)
- Unitron


Audioscan® Education
© 2019 Audioscan.



Approach #3: Frequency Translation

- Let's start with the following illustration:

- Imagine that this image represents hearing aid channels
- The higher channel numbers represent increasing frequencies

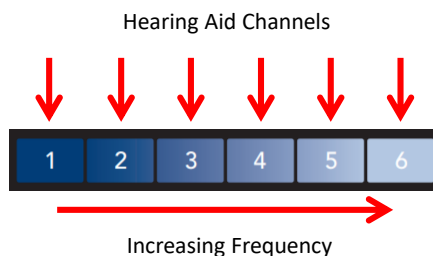

https://starkeypro.com/pdfs/technical-papers/Spectral_iQ_Technical_Paper.pdf


With permission from Jeff Crukley, Ph.D. Senior Research Scientist, Starkey Labs

Audioscan® Education
© 2019 Audioscan.



Approach #3: Frequency Translation

- Then, this is how frequency transposition could be represented

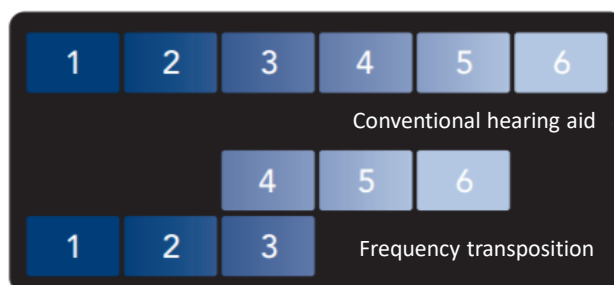

https://starkeypro.com/pdfs/technical-papers/Spectral_iQ_Technical_Paper.pdf


With permission from Jeff Crukley, Ph.D. Senior Research Scientist, Starkey Labs

Audioscan® Education
© 2019 Audioscan.



Approach #3: Frequency Translation

- And, this is how frequency compression could be represented

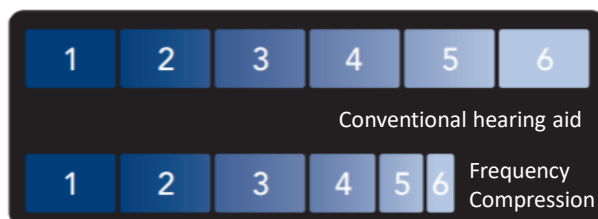

https://starkeypro.com/pdfs/technical-papers/Spectral_iQ_Technical_Paper.pdf


With permission from Jeff Crukley, Ph.D. Senior Research Scientist, Starkey Labs

Audioscan® Education
© 2019 Audioscan.



Approach #3: Frequency Translation

- Based on these representations, here is how frequency translation is designed to function

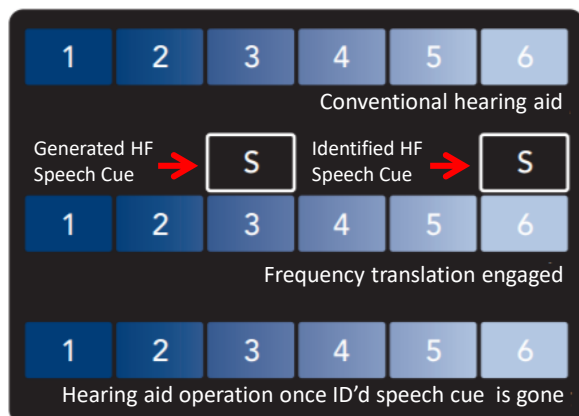

https://starkeypro.com/pdfs/technical-papers/Spectral_iQ_Technical_Paper.pdf

With permission from Jeff Crukley, Ph.D. Senior Research Scientist, Starkey Labs


Audioscan® Education
© 2019 Audioscan.



How might this look in a test box?

Here is how the “S” stimulus can be used to “display” frequency translation:

Note relocation of “S” hump AND maintenance of original “S” energy (within the H.A.’s output bandwidth)

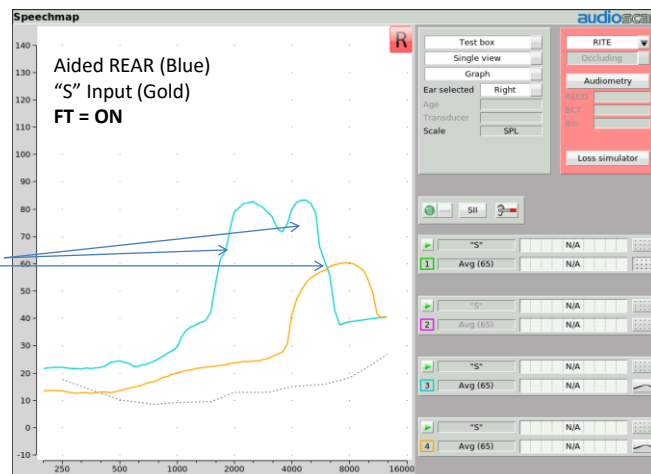

Audioscan® Education
© 2019 Audioscan.



Frequency Translation Can Be Found In

- Starkey (Spectral IQ)

Audioscan® Education
© 2019 Audioscan.



Frequency Lowering: Verification Protocol

Audioscan® Education
© 2019 Audioscan.



Goals of Frequency Lowering Verification

- To objectively determine frequency lowering candidacy



Audioscan® Education
© 2019 Audioscan.



Goals of Frequency Lowering Verification

- Confirm restoration of audibility for missing target speech cues
- Confirm restoration accomplished with minimal impact on the original audible bandwidth
- Accomplish the above two goals regardless of frequency lowering approach used



Audioscan® Education
© 2019 Audioscan.



Stimulus Options

- Live “S” and “SH” produced by testing clinician:
 - Advantages:
 - Represents critical phonemes important to capture when frequency lowering is being used
 - Easily produced
 - Disadvantage:
 - Not presented at a calibrated spectrum and level, thus subject to inter-test variability



Audioscan® Education
© 2019 Audioscan.



Stimulus Options


- The Audioscan “filtered” carrot passage input options:

- Advantage:

- Calibrated and pre-stored in system, eliminating issue of inter-test variability

- Disadvantages:

- Bandwidth narrower than the frication bandwidth of naturally produced “S” and “SH”
- Based on LTASS level rather than peak level
- May underestimate delivered audibility

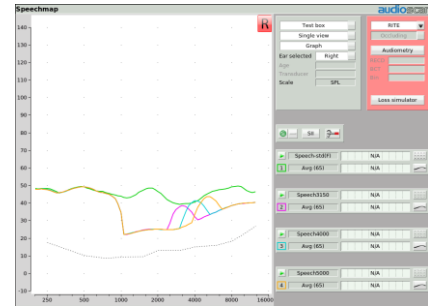

Stimulus Options

- Calibrated and stored “S” and “SH” stimuli
 - Advantages
 - Delivers a more representative bandwidth of these fricatives
 - Captures peak energy level components that contribute to overall audibility
 - Stored and calibrated for solid test-retest reliability

Method For Creating Calibrated Fricative Stimuli


- Excised fricatives from the ISTS signal
 - Multiple talkers and running speech
- Compared these fricatives to other clinically produced signals
 - Single talker and isolated fricative production
- This led to a solid definition of the frequency range and energy levels of interest for each phoneme
- White noise was then shaped to fabricate these targeted bandwidth and energy level components.

Scollie, S., et. al., (2016) Fitting Frequency-Lowering Signal Processing Applying the American Academy of Audiology Pediatric Amplification Guideline: Updates and Protocols – J Am Acad Audiol 27:219-236

Audioscan® Education
© 2019 Audioscan.



The “S” and “SH” Input Stimulus Spectra

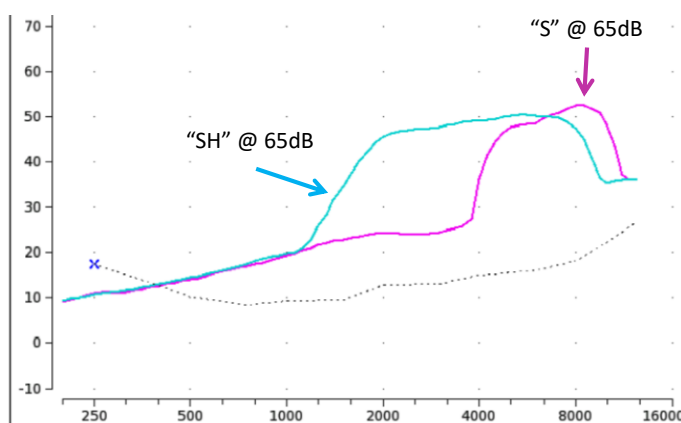

Scollie, S., et. al., (2016) Fitting Frequency-Lowering Signal Processing Applying the American Academy of Audiology Pediatric Amplification Guideline: Updates and Protocols – J Am Acad Audiol 27:219-236


Audioscan® Education
© 2019 Audioscan.



Clinical Procedure

Step One:

- Complete a Speechmap verification fitting
 - Ensure that any frequency lowering feature has been turned OFF during this process
 - Isolate and display the 65dB REAR image

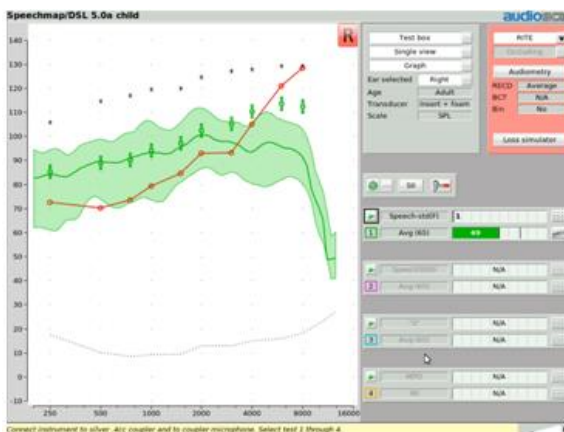


Audioscan® Education
© 2019 Audioscan.



Clinical Procedure

Step Two:

- With noise reduction OFF, determine if this fitting restores audibility for the “S” stimulus at 65dB SPL input level
 - If not, your options are:
 - Add more high frequency gain until “S” audibility has been restored
 - Switch to a different hearing aid that may be capable of delivering this additional gain
 - Activate frequency lowering to move the “S” stimulus into an audible frequency area

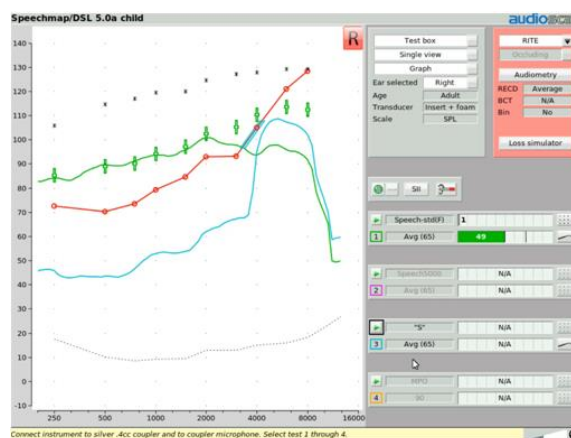


Audioscan® Education
© 2019 Audioscan.



Clinical Procedure

Step Three:

- Identify the MAOF Range
 - (Maximum Audible Output Frequency)
 - Low frequency end of the MAOF Range = frequency where aided LTASS crosses under threshold
 - High frequency end of the MAOF Range = frequency where the peaks of speech (99th percentile line) crosses under threshold

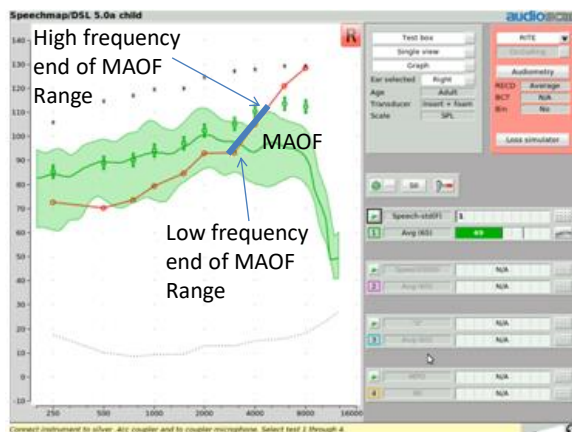


Audioscan® Education
© 2019 Audioscan.



Activating the Audioscan MAOF “Highlighter”

- Select a test and activate the “S” stimulus option (as before)
- While test is running, open the “Show MAOF” pull-down menu
- Select the 65 dB REAR you wish to use to identify the target MAOF Range
- This will automatically display a highlighted area on the audiogram that is the MAOF Range

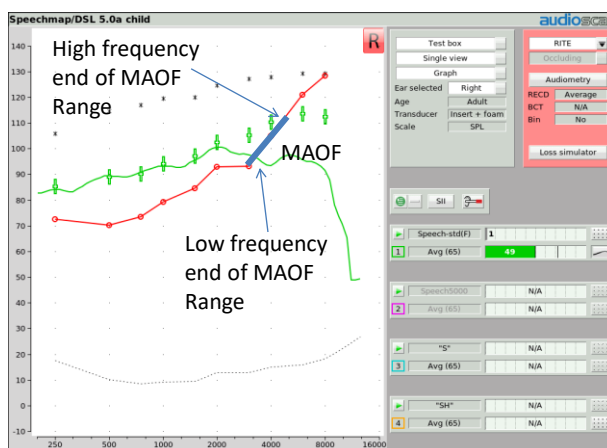


Audioscan® Education
© 2019 Audioscan.



Clinical Procedure

Step Four:

- Turn frequency lowering feature ON at its default settings
- Check for audibility of “S” stimulus
 - Target positioning the upper shoulder of the “S” stimulus within the MAOF Range
- If default settings don’t achieve target positioning, adjust the FL feature until target positioning has been acquired

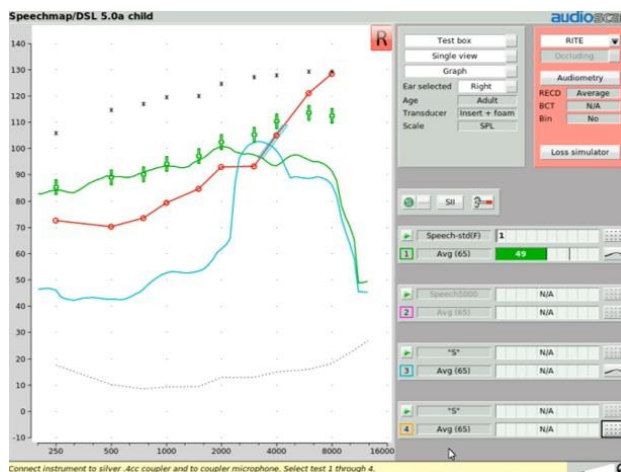


Audioscan® Education
© 2019 Audioscan.



Clinical Procedure

Step Five:

- Activate another test, now using the “SH” stimulus instead of the “S” stimulus
- Check for the following:
 - That the “SH” stimulus is also audible
 - That the “SH” audibility spectrum is different than the “S” audibility spectrum

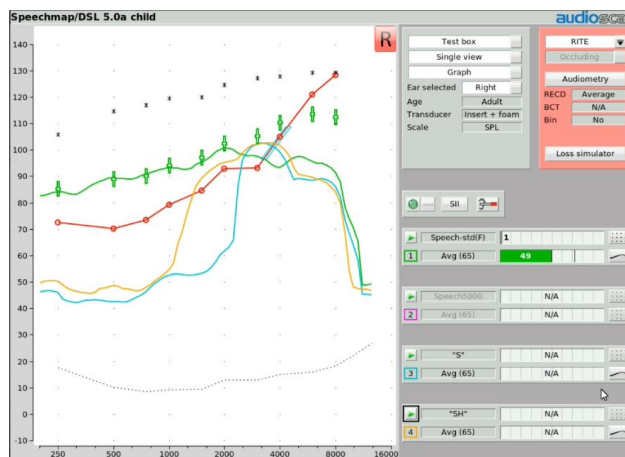

Audioscan® Education
© 2019 Audioscan.



Clinical Procedure

Step Six:

- Perform a listening check
 - Determine whether the sound quality and distinctiveness of speech are acceptable



Clinical Procedure

Step Seven:

- Re-enable noise reduction if this is a component of the fitting

Fitting and Post Fitting:

- At Fitting and Follow up, consider listener's responses, and/ or outcome measures re: perceived sound quality & ability to distinguish "S" and "SH".
- Inform caregivers and/or therapists of the altered sound quality that has now been engaged




Frequency lowering and the SII score

Audioscan® Education
© 2019 Audioscan.



What Is SII?

- Quantifies likely intelligibility based on the audibility of various speech cues

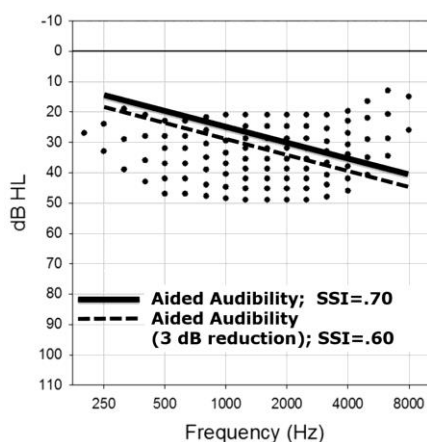


Killion MC, Mueller HG. [Twenty years later: A NEW Count-The-Dots method](#). *Hear Jour.* January 2010; 63(1):10-17.

Audioscan® Education
© 2019 Audioscan.



Aided and Unaided SII Inside of Speechmap

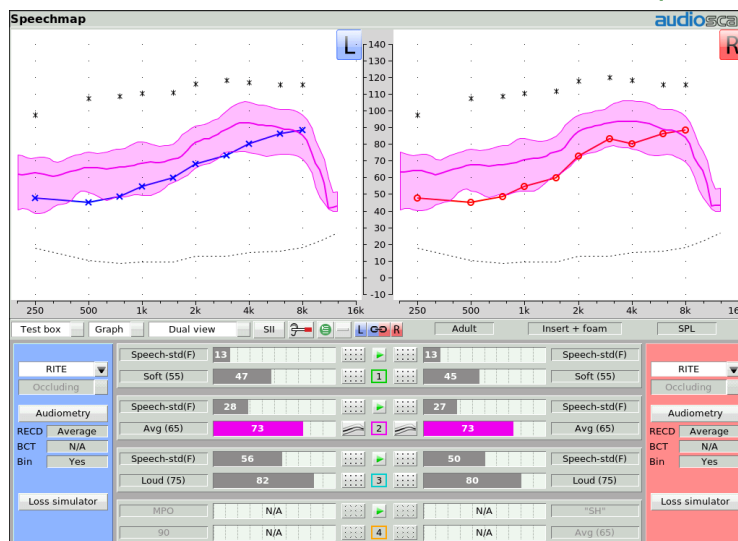


Audioscan® Education
© 2019 Audioscan.



SII Scores Do Not Appear With “S” and “SH”

- Main Reason:
 - These stimuli do not produce an aided broadband speech result - which is what the SII calculation is designed to quantify

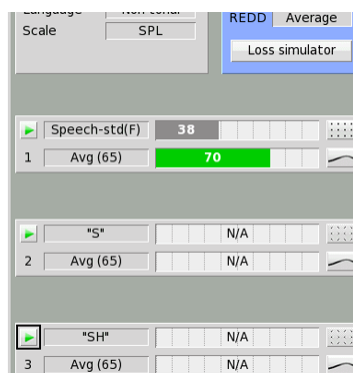

Audioscan® Education
© 2019 Audioscan.



The SHARP Audibility Estimates with Nonlinear Frequency Compression

- The “Situational Hearing Aid Response Profile”
- Developed at the Boys Town National Research Hospital
- Added a frequency lowering intelligibility estimate feature in 2014




https://journals.lww.com/thehearingjournal/Fulltext/2014/03000/SHARP_Updates_Enable_Audibility_Estimates_with.7.aspx

Audioscan® Education
© 2019 Audioscan.




Conventional SHARP Display

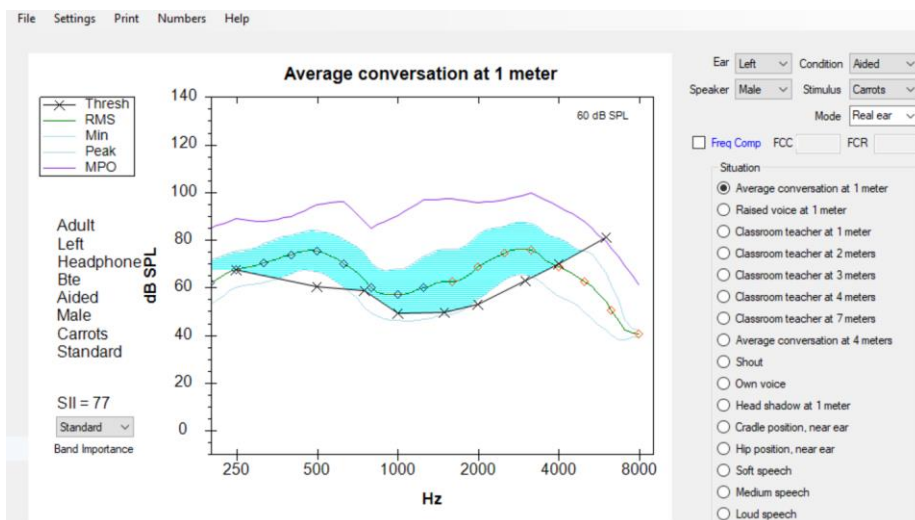

Audioscan® Education
© 2019 Audioscan.



SHARP Display With “Freq Comp” Selected

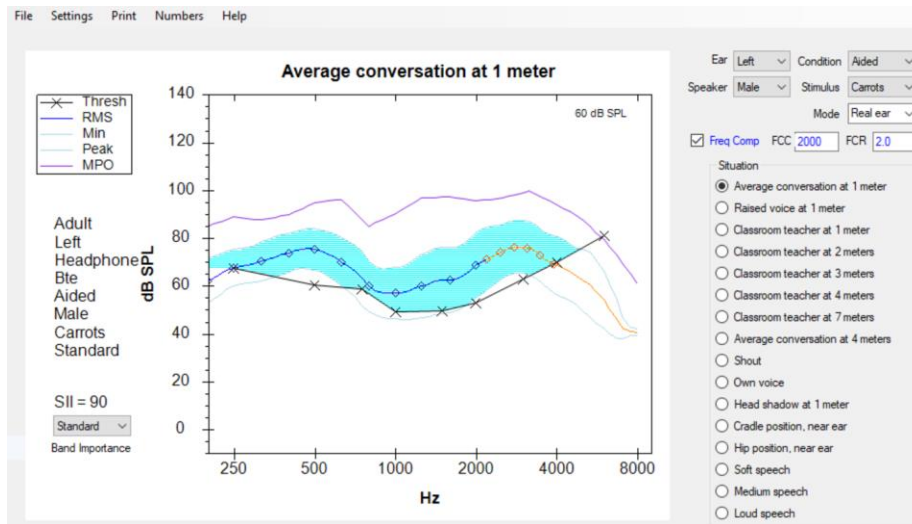

Audioscan® Education
© 2019 Audioscan.



To Summarize:

- Frequency lowering can be activated to restore audibility for speech sounds that may fall outside of a patient's aidable frequency range
- Several methods of applying frequency lowering are available
 - Frequency transposition
 - Frequency compression
 - Frequency translation

Audioscan® Education
© 2019 Audioscan.





To Summarize:

- Verification technology can be used to:
 - Objectively determine frequency lowering candidacy
 - Adjust frequency lowering settings to restore audibility for important speech cues with minimal interference with the existing aided bandwidth
 - Verify sound quality and distinctiveness
 - Aid in counseling



Follow-Up

- For additional information on frequency lowering verification:
 - Visit the education section of the Audioscan website (www.audioscan.com)
 - Consult the User Guide of your Audioscan verification equipment

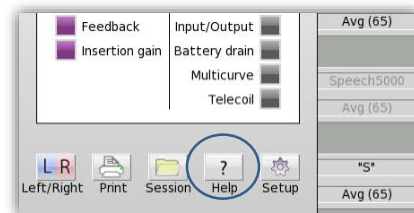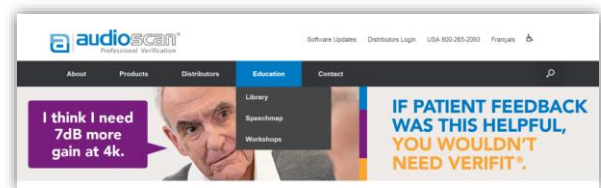



Education
The Science of Hearing Aid Fitting

Thank You!

info@audioscan.com

Audioscan® Education
© 2019 Audioscan

 **audioscan®**
Professional Verification