


Connecting to your cell phone

Discover hearing instruments

1 Bluetooth menu

Open the Bluetooth® menu on your phone (often found in “Settings” or “Tools”). Make sure Bluetooth is on.

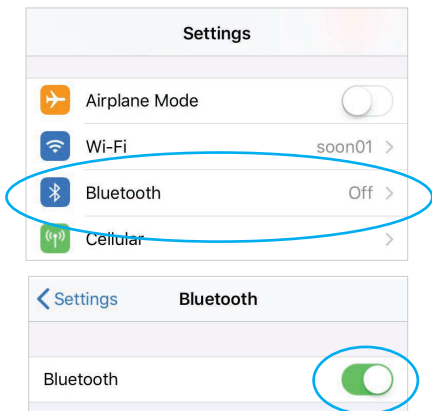


2 Pairing

Your hearing aid name: _____

For hearing aids with a battery door, restart the (R or L) hearing aid by opening and closing the battery door.

For hearing aids without a battery door, turn the (R or L) hearing aid off by pressing the lower part of the button until the LED turns red (4 sec). Then turn on the (R or L) hearing aid by pressing the same button until the LED blinks green (2 sec).


The name of your hearing aid will appear on the phone. Tap to select, and click “pair” if prompted.

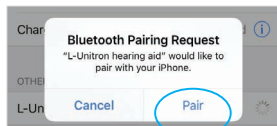

3 Phone calls


Short push (<2 seconds) on upper or lower button, either ear, to accept.

Long push (>2 seconds) on upper or lower button, either ear, to reject.

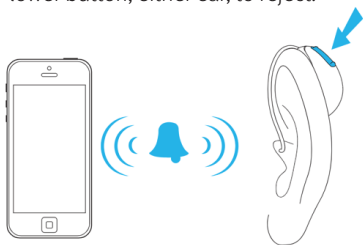

4 Adjust

Press the top button to increase and the bottom button to decrease Bluetooth® streaming volume. This can also be done with the volume buttons on the side of your phone.


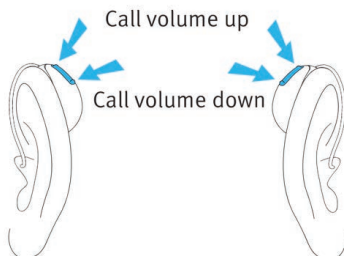


Download the Unitron Remote Plus app


Discover hearing instruments

This app provides easy control of hearing aid settings using your smartphone. It also lets you share in-the-moment experiences with hearing aid performance.

Visit the App Store (Apple™) or Google Play (Android™) and search for “Unitron Remote Plus” app.

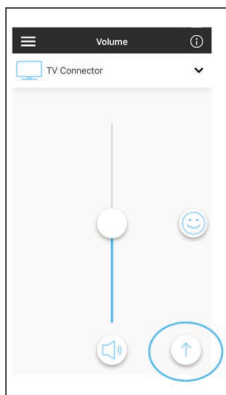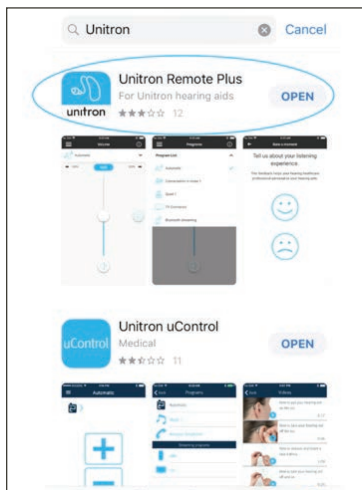

Advanced program settings

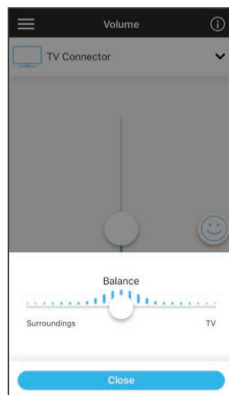

Environmental balance

Note: iOS 10.2 or Android 6.0 and Bluetooth 4.0 or newer are required to use the Unitron Remote Plus app.

Looking for more information? Visit unitron.com/us

- Step-by-step pairing guides and videos
- Troubleshooting tips
- Frequently Asked Questions
- Information on the TV Connector and Remote Plus app accessories

The Bluetooth® word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc. Apple, the Apple logo, iPhone, and iOS are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Android, Google Play, and the Google Play logo are trademarks of Google Inc.

sonova
HEAR THE WORLD