



**EXPRESSfit® Pro 2019.2:
Not Your Mama's Fitting
Software**

Scott Bunnell, Au.D.


www.sonici.com

CELEBRATING
20
YEARS OF SOUND

SONIC
Everyday Sounds Better

1

Learner Outcomes



- After this session, participants will be able to do the following things
 - Program Sonic hearing aids on the EXPRESSfit® Pro 2019.2 software platform
 - Transfer hearing instrument settings from one style of hearing aid to a different style hearing aid using the Transfer Settings tool in the EXPRESSfit® Pro fitting software
 - Update firmware on Sonic hearing aids, using the Firmware Updater tool.

www.sonici.com

CELEBRATING
20
YEARS OF SOUND

SONIC
Everyday Sounds Better

2

New! in EXPRESSfit® Pro 2019.2




- Support for new Trek products:
 - 2 Technology levels –
 - Trek 80 | 40
 - 2 form factors
 - Super Power and Ultra Power BTEs
- Updated Transfer Fitting feature
- Bimodal fitting support
- Firmware update

www.sonici.com



3

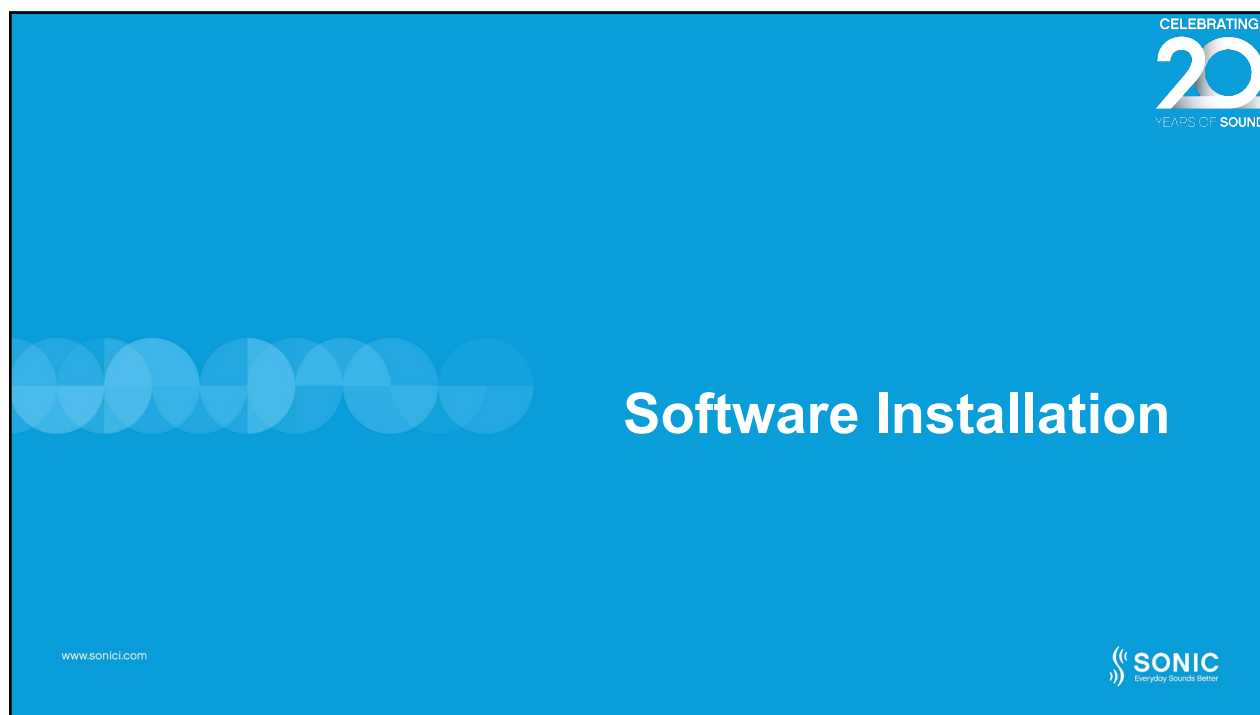


4

Website Download

CELEBRATING
20
YEARS OF SOUND

- Download and install from the website:
 - www.sonici.com
 - Go to Professional Center

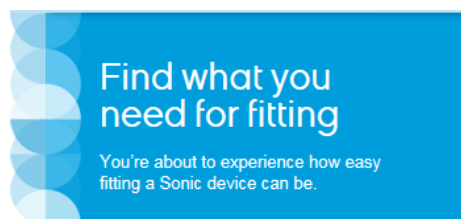

www.sonici.com

SONIC
Everyday Sounds Better

5

Thumb Drive Installation

CELEBRATING
20
YEARS OF SOUND



- If prompted, choose “open files and folders” or
- Choose “CD Drive (D:) EXPRESSfit”
- Click on “set up”

www.sonici.com

SONIC
Everyday Sounds Better

6

Sonic Software Updater



- Open in task bar or All Programs
 - Cannot be opened from EXPRESSfit® Pro
- Downloads in the background
- OK to turn off PC, download will resume when powered up again
- Install prompt will appear when download completes
- Download is deleted after install

www.sonici.com





7

Sonic Software Updater




- Changing the settings of the updater
 - Automatically download updates
 - Automatically install updates
 - Check for interval updates
 - Cancel, install and delete download from this menu

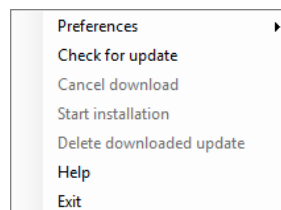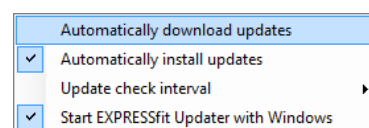

www.sonici.com



8

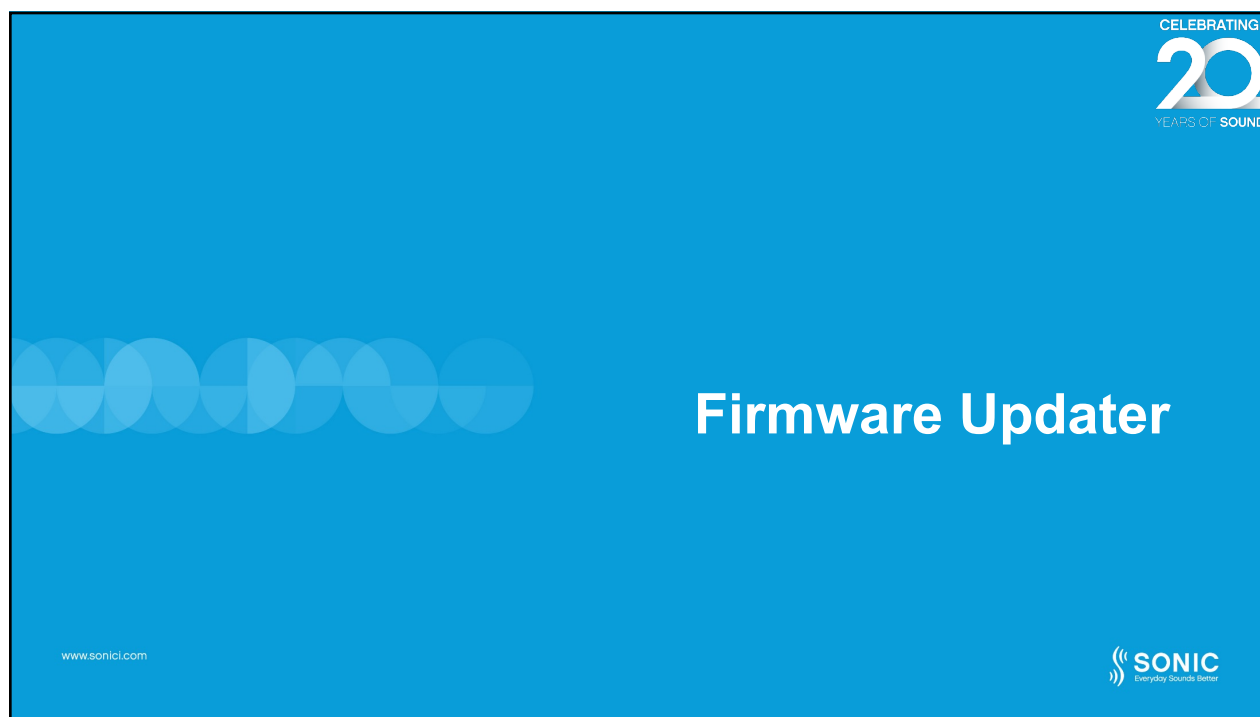

9

Firmware Updater – What Is It?

- Updates hearing instruments and accessories with
 - New features
 - Security updates
 - Fixes so repairs can be made in the clinic




www.sonici.com

CELEBRATING 20 YEARS OF SOUND

SONIC
Everyday Sounds Better

10

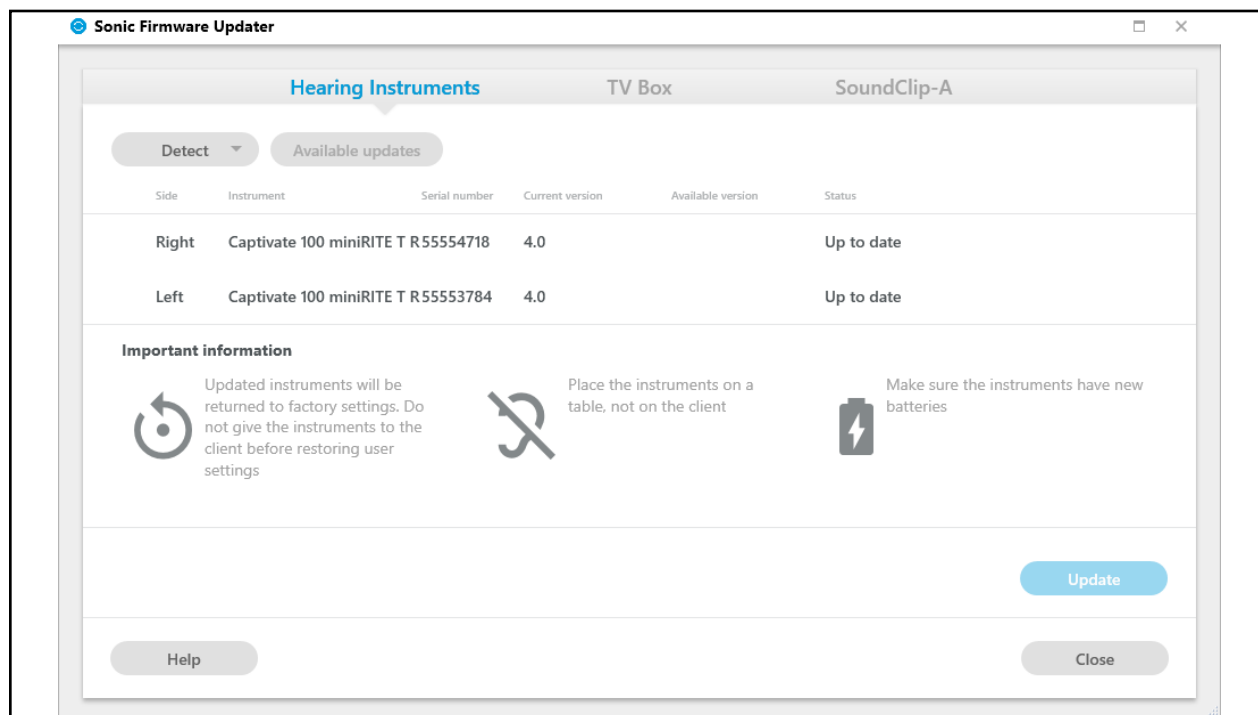

11

Firmware Update – **New!** In 2019.2

- Enhanced reliability of connection to Android devices
- Security improvements
- SmartMusic performance improvements
- Improved handling of multiple Bluetooth™ devices
- Bluetooth™ performance, stability and speed improvements
- Safety feature in rechargeable aids
- Support to TV adapters and EduMic connections

www.sonici.com



12

Firmware update

CELEBRATING
20
YEARS OF SOUND

- Enchant firmware
 - Version 3.0 ➡ 3.1

- Captivate firmware
 - Version 4.0 ➡ 4.1



www.sonici.com

SONIC
Everyday Sounds Better

13

CELEBRATING
20
YEARS OF SOUND

Firmware Updater

What to do before you update

www.sonici.com

SONIC
Everyday Sounds Better

14

Before You Update

CELEBRATING
20
YEARS OF SOUND

- The following scenarios must be met
 - Computer is online
 - Use a hardwire connection
 - Captivate miniRITE T R requires red pin removal tool to open battery drawer for hardwire connection
 - Use fresh/fully charged batteries
 - Ensure > 11% charge for the Captivate miniRITE T R
- It is recommended that programming is saved before doing a firmware update

www.sonici.com

SONIC
Everyday Sounds Better

15

Removal Tool for MNR T R

CELEBRATING
20
YEARS OF SOUND

Tools needed: Service tool (red),
part no: 890-22-2270-00



Look for the battery drawer release hole on the bottom of the instrument.



Insert the service tool in the battery drawer release hole to open the battery drawer.




The battery drawer has been released.


www.sonici.com

SONIC
Everyday Sounds Better

16

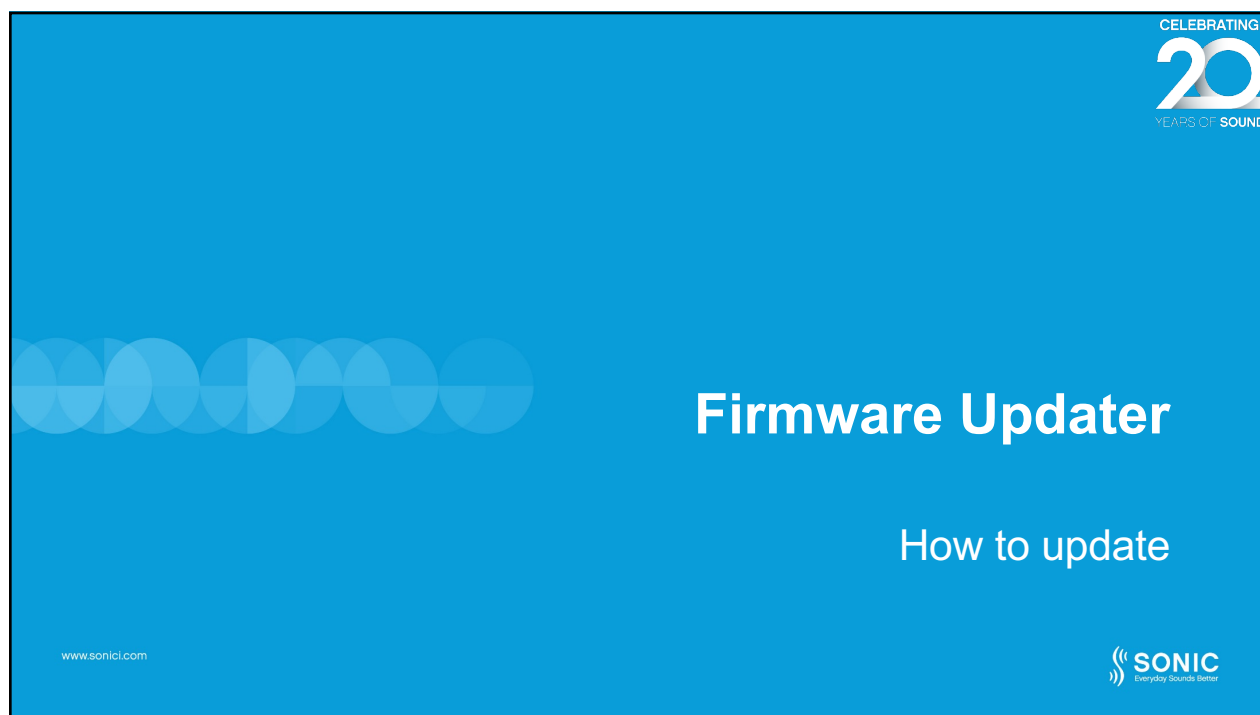

17

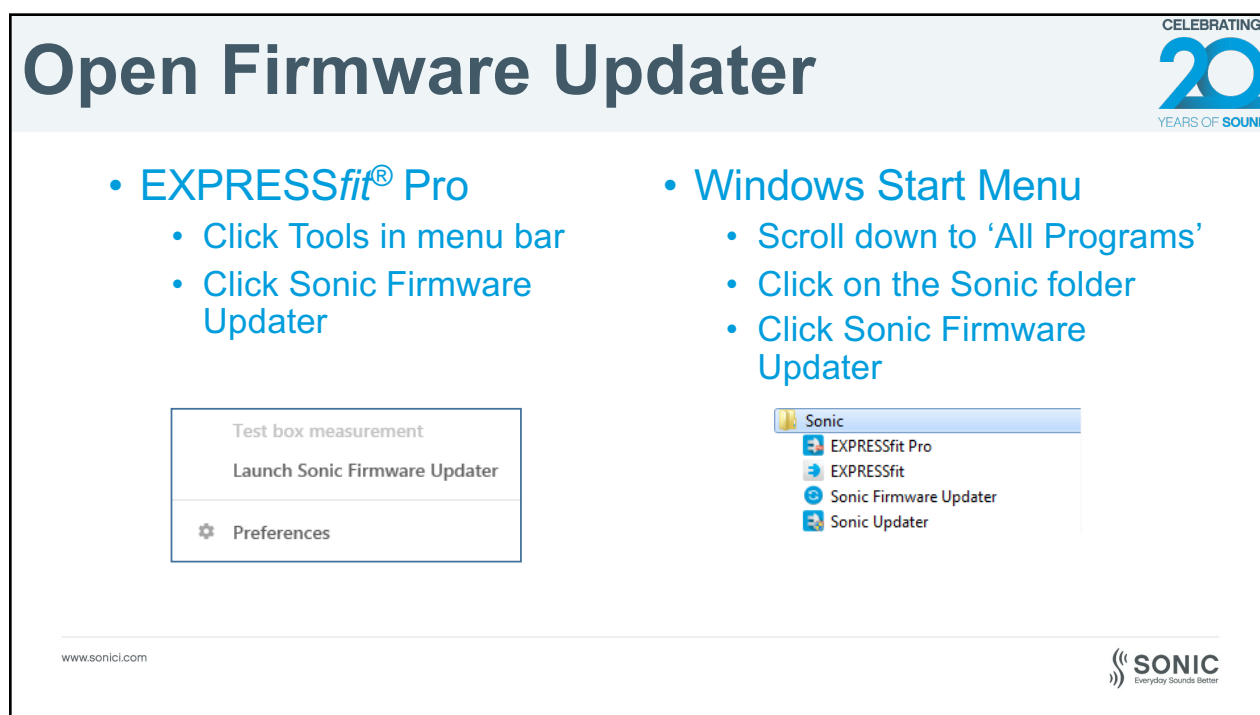


18

Hardwire Connection

CELEBRATING
20
YEARS OF SOUND

- You must have a hardwire connection
 - HiPro 2 or EXPRESSlink 3 recommended

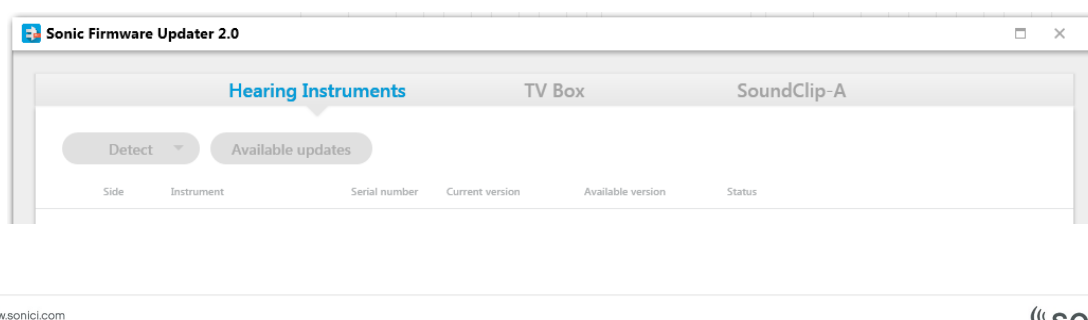


19

Detect in Firmware Updater

CELEBRATING
20
YEARS OF SOUND

- Select a programming device
- Click Detect

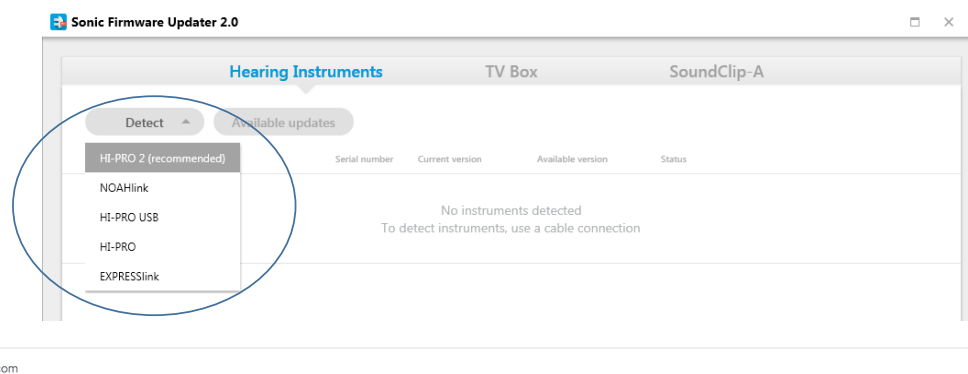

20

Select Hearing Instruments

CELEBRATING
20
YEARS OF SOUND

- By default, all detected instruments are selected for the update
 - If you do not want to update an instrument, uncheck box
 - Total update time is shown next to Update button

Side	Instrument
<input checked="" type="checkbox"/> Right	Enchant 100 BTE
<input checked="" type="checkbox"/> Left	Enchant 100 BTE

Estimated time: 3:21 min. Update

www.sonici.com

SONIC
Everyday Sounds Better

21

Update Hearing Instruments

CELEBRATING
20
YEARS OF SOUND


Status
<input checked="" type="checkbox"/> Updated successfully
<input checked="" type="checkbox"/> Updated successfully

- Click Update
- You will see progress bars showing the status of the update process
- When the update is successful, a green check mark will appear

www.sonici.com

SONIC
Everyday Sounds Better

22

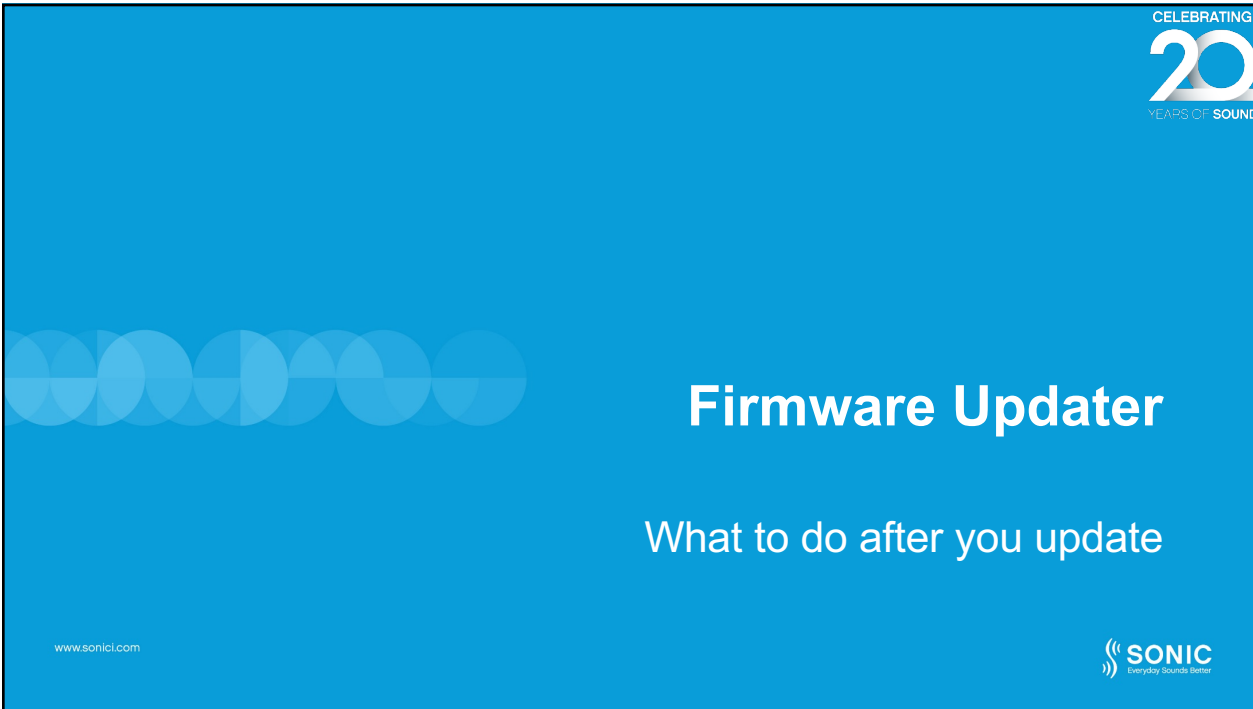A blue slide with a decorative pattern of overlapping circles on the left. The title 'Firmware Updater' is in large white font, and the subtitle 'What to do after you update' is in smaller white font below it. The Sonic logo is in the bottom right, and the website 'www.sonici.com' is in the bottom left. A 'CELEBRATING 20 YEARS OF SOUND' logo is in the top right corner.

CELEBRATING 20 YEARS OF SOUND


Firmware Updater

What to do after you update

www.sonici.com

SONIC
Everyday Sounds Better

23

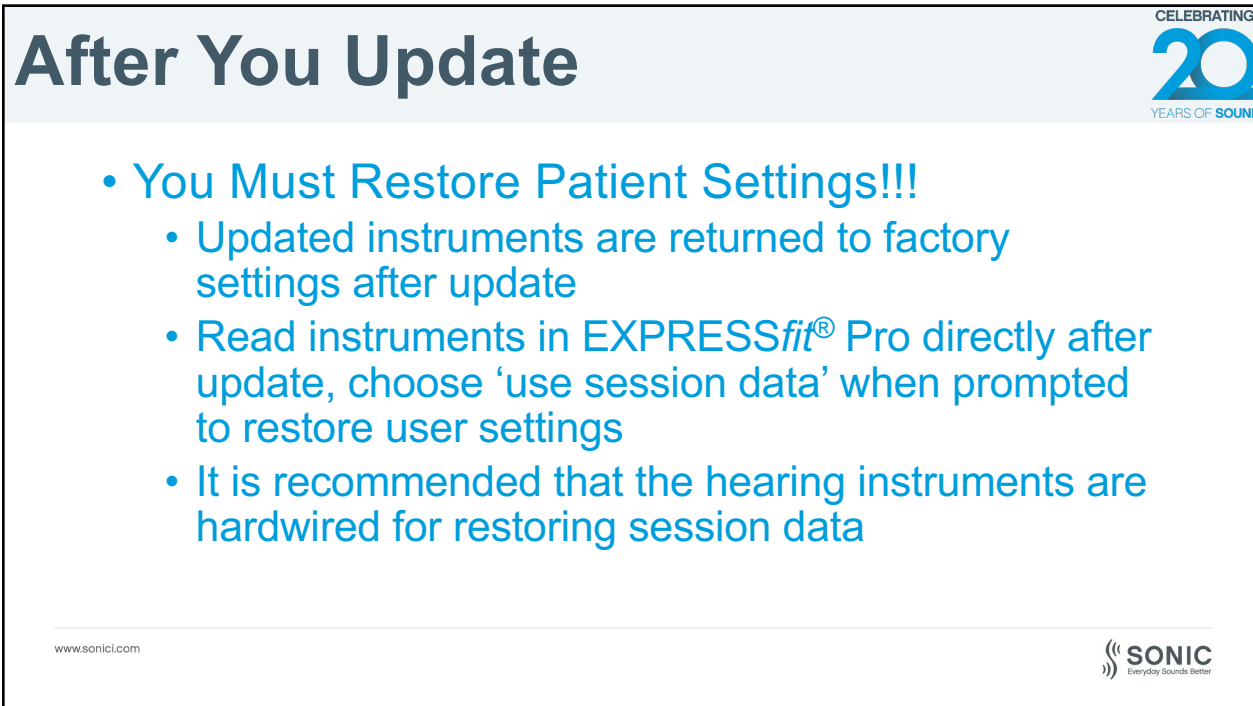A light blue slide with a grey header bar containing the title 'After You Update'. The main content area is white and contains a bulleted list of instructions. The Sonic logo is in the bottom right, and the website 'www.sonici.com' is in the bottom left. A 'CELEBRATING 20 YEARS OF SOUND' logo is in the top right corner.

CELEBRATING 20 YEARS OF SOUND

After You Update

- You Must Restore Patient Settings!!!
 - Updated instruments are returned to factory settings after update
 - Read instruments in EXPRESSfit® Pro directly after update, choose 'use session data' when prompted to restore user settings
 - It is recommended that the hearing instruments are hardwired for restoring session data

www.sonici.com

SONIC
Everyday Sounds Better

24

After You Update

- Provide patient with a new 'Instructions for Use' booklet



www.sonici.com

SONIC
Everyday Sounds Better

25

CELEBRATING
20
YEARS OF SOUND

Transfer Settings

What's new in EXPRESSfit® Pro 2019.2

www.sonici.com

SONIC
Everyday Sounds Better

26

Transfer Settings

CELEBRATING
20
YEARS OF SOUND

- Transfer settings from one style and/or model of hearing instrument to another on the SoundDNA platform
- **New in 2019.2!** – transfer volume steps, volume control range and all feature settings



www.sonici.com



SONIC
Everyday Sounds Better

27

Transfer Settings

CELEBRATING
20
YEARS OF SOUND

- When you connect an instrument in a patient record that has a saved session from a different style or technology level on the SoundDNA platform, there will be a prompt with an option to use the Transfer Settings tool to transfer patient settings

 <p>CV100 MNR T R 85 55554718</p> <p>Fitting data conflict The current fitting session data differs from the detected instrument settings. Choose how to proceed:</p> <ul style="list-style-type: none"> <input checked="" type="radio"/> Use hearing instrument data <input type="radio"/> Start new fitting <input type="radio"/> Use session data (Transfer Fitting) 	 <p>CV100 MNR T R 85 55553784</p> <p>Fitting data conflict The current fitting session data differs from the detected instrument settings. Choose how to proceed:</p> <ul style="list-style-type: none"> <input checked="" type="radio"/> Use hearing instrument data <input type="radio"/> Start new fitting <input type="radio"/> Use session data (Transfer Fitting)
<p>Retry Continue Cancel</p>	

www.sonici.com

SONIC
Everyday Sounds Better

28

Transfer Settings

CELEBRATING
20
YEARS OF SOUND

- A screen will appear next, explaining what will be transferred and what will be removed and prescribed

Will be transferred

- All settings available with the same values on source and target instruments

Will not be transferred

- Feedback measurements
 - In-situ audiometry results
 - Accessories pairing
 - Settings that are not available in the target instruments
 - Settings which cannot be transferred correctly.
- These settings will be set to default, or they will be prescribed**

www.sonici.com

SONIC
Everyday Sounds Better

29

Transfer Settings

CELEBRATING
20
YEARS OF SOUND



Source Instruments

Target Instruments

www.sonici.com

SONIC
Everyday Sounds Better

30

Transfer Settings

CELEBRATING
20
YEARS OF SOUND

Summary

Transferred

All settings which are available both on source and target instruments, covering:

- Fine Tuning
- Adaptation
- Manage Programs
- Adaptive Feedback Canceller
- Frequency Transfer
- Tinnitus SoundSupport
- Speech Variable Processing
- SmartCompress
- Directionality
- SPN Noise Reduction
- Impulse Noise Reduction
- Soft Noise Management
- Wind Noise Reduction

Not transferred



- Feedback measurements
- In-situ audiometry results
- Accessories pairing
- Settings that are not available in the target instruments
- Settings which could not be transferred correctly.

These settings have been set to default, or they have been prescribed

Target Instruments

TK80 BTE UP

Hook/Thin Tube
Undamped
Tubing
2 mm tube
Vent
Occluded

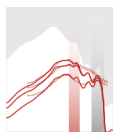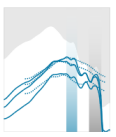TK80 BTE UP

Hook/Thin Tube
Undamped
Tubing
2 mm tube
Vent
Occluded

After the settings have been successfully transferred, a summary is provided

www.sonici.com

SONIC
Everyday Sounds Better

31

CELEBRATING
20
YEARS OF SOUND

Navigating EXPRESSfit® Pro 2019.2

www.sonici.com

SONIC
Everyday Sounds Better

32

Supported Programming Interfaces

CELEBRATING
20
YEARS OF SOUND

• Hardwire

- HI-PRO
- HI-PRO 2
- USB HI-PRO
- NOAHlink
- EXPRESSlink 3



• Wireless

- FittingLink 3.0
- NOAHlink wireless



www.sonici.com

SONIC
Everyday Sounds Better

33

Navigating in EXPRESSfit® Pro

CELEBRATING
20
YEARS OF SOUND

Navigation
Steps

Sub Tasks

test, test

File Edit Tools Help Select sound

Client Data

Instrument Acoustics

Fit Instrument

Features Selection

Finish Session

Personal data

Client information

First name: test

Last name: test

Gender: Male

Date of birth:

Age group: 18 - 70 years

Client language

None

Recently selected: None

All languages:

- Abkhazian
- Afar
- Afrikaans
- Akan
- Albanian
- Amharic
- Arabic

Fitting rationale

Best Fit Fast SE

Best Fit Fast

NAL-NL2

NAL-NL1

DSL v5.0 adult

DSL v5.0 pediatric

Navigation Steps

1. Select sound

2. Select client

3. Select language

4. Select fitting rationale

5. Detect

SONIC
Everyday Sounds Better

www.sonici.com


SONIC
Everyday Sounds Better

34

Navigating in EXPRESSfit® Pro

CELEBRATING
20
YEARS OF SOUND


- 5 navigation screens
 - Client Data
 - Instrument Acoustics
 - Fit Instrument
 - Feature Selection
 - Finish Session

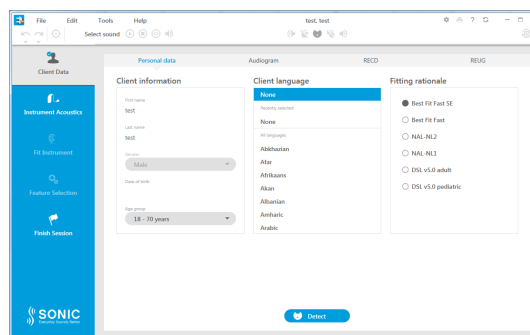

www.sonici.com

SONIC
Everyday Sounds Better

35

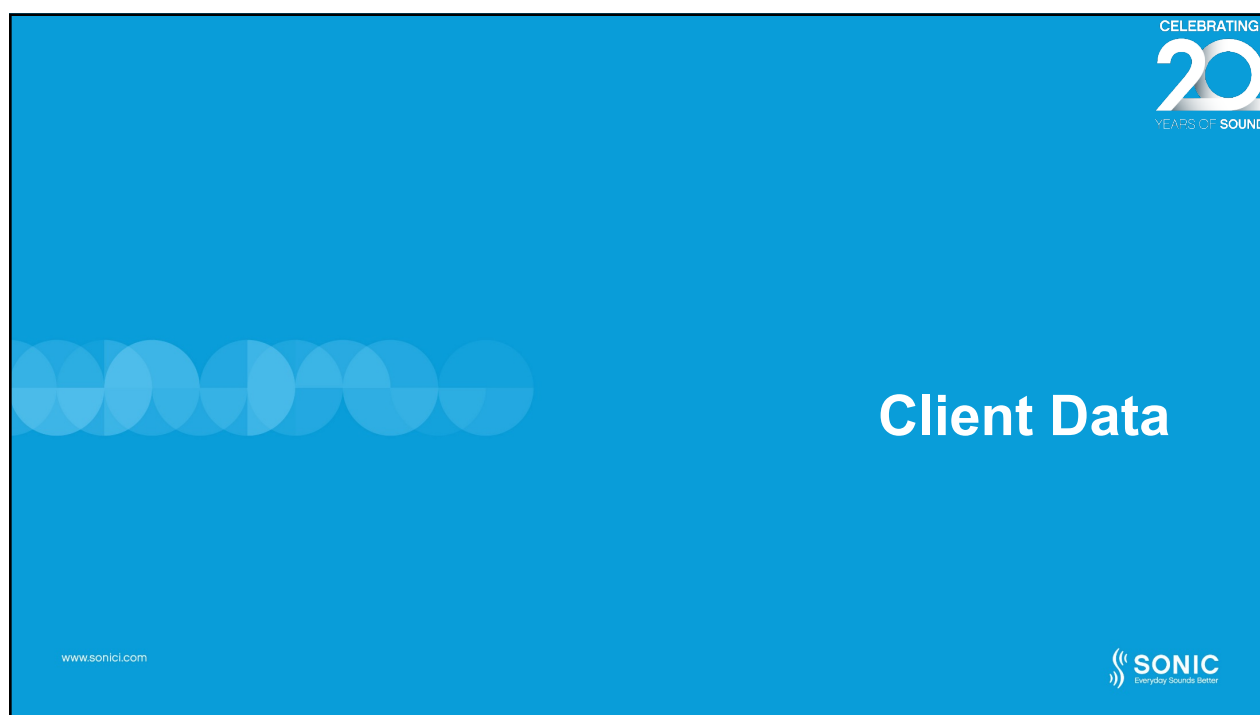

36

Client Data



- Check demographic data
 - Add gender and age if needed
- Select appropriate language
- Tabs for Audiogram, RECD and REUG information
- In-Situ Audiometry available (Audiogram)

www.sonici.com



37

Supported Fitting Algorithms



- Best Fit Fast
 - Designed to quickly achieve a good fitting result
 - Emphasizes speech intelligibility while still providing comfort
- Best Fit Fast SE
 - Emphasizes speech intelligibility for experienced users
 - Great choice for patients who prefer fittings with more gain

Fitting Rationale

- ☒ Best Fit Fast
- ☐ Best Fit Fast SE
- ☐ NAL-NL1
- ☐ NAL-NL2
- ☐ DSL [i/o]
- ☐ DSL 5 Adult
- ☐ DSL 5 Child

} Default for Trek!

www.sonici.com




38

Personal data	Audiogram	RECD	REUG
Client information <div> First name test </div> <div> Last name test </div> <div> Gender Male </div> <div> Date of birth </div> <div> Age group 18 - 70 years </div>	Client language <div> None </div> <div> Recently selected </div> <div> None </div> <div> All languages </div> <div> Abkhazian </div> <div> Afar </div> <div> Afrikaans </div> <div> Akan </div> <div> Albanian </div> <div> Amharic </div> <div> Arabic </div>	Fitting rationale <div> <input checked="" type="radio"/> Best Fit Fast SE </div> <div> <input type="radio"/> Best Fit Fast </div> <div> <input type="radio"/> NAL-NL2 </div> <div> <input type="radio"/> NAL-NL1 </div> <div> <input type="radio"/> DSL v5.0 adult </div> <div> <input type="radio"/> DSL v5.0 pediatric </div>	


39

CELEBRATING
20
YEARS OF SOUND



Instrument Acoustics

www.sonici.com




40

Instrument Acoustics

CELEBRATING
20
YEARS OF SOUND

- Set Acoustics
 - Drop down menu
 - Includes customizable Fitting Range graphs

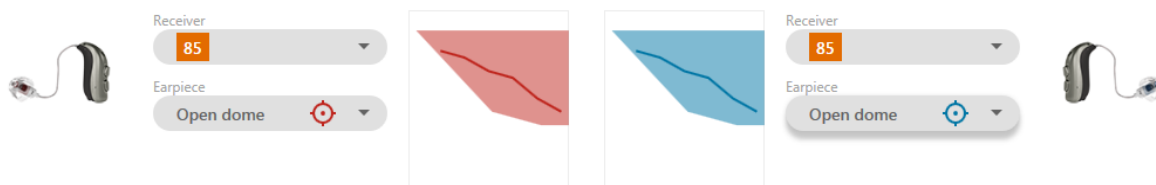

www.sonici.com

SONIC
Everyday Sounds Better

41

Instrument Acoustics - Simulate

CELEBRATING
20
YEARS OF SOUND

Family

Captivate 100
Captivate 80
Captivate 60
Enchant 100
Enchant 80
Enchant 60
Enchant 40
Enchant 20
Trek 80
Trek 40

New in 2019.2!

Style

TK80 BTE UP
TK80 BTE SP

Other Styles:

miniRITE
miniRITE T
miniRITE T R
BTE 105


Customs

- ITE FS
- ITE HS
- ITC
- CIC
- IIC


www.sonici.com

SONIC
Everyday Sounds Better

42

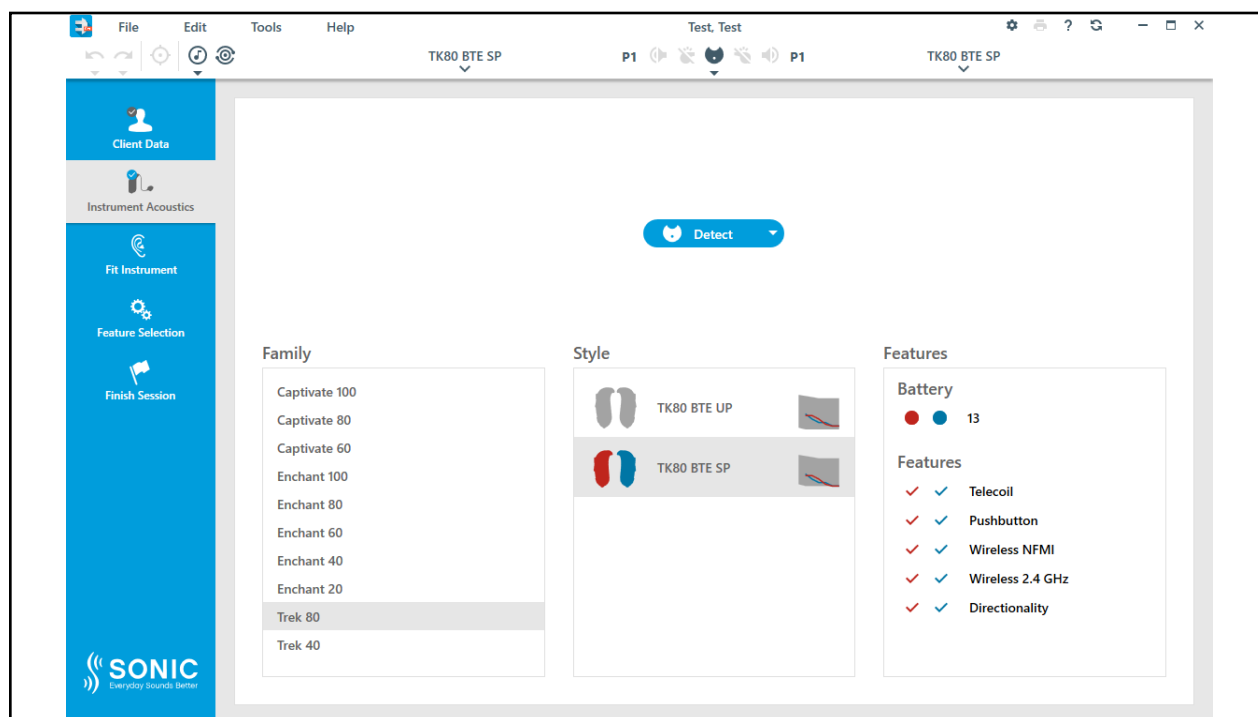

43

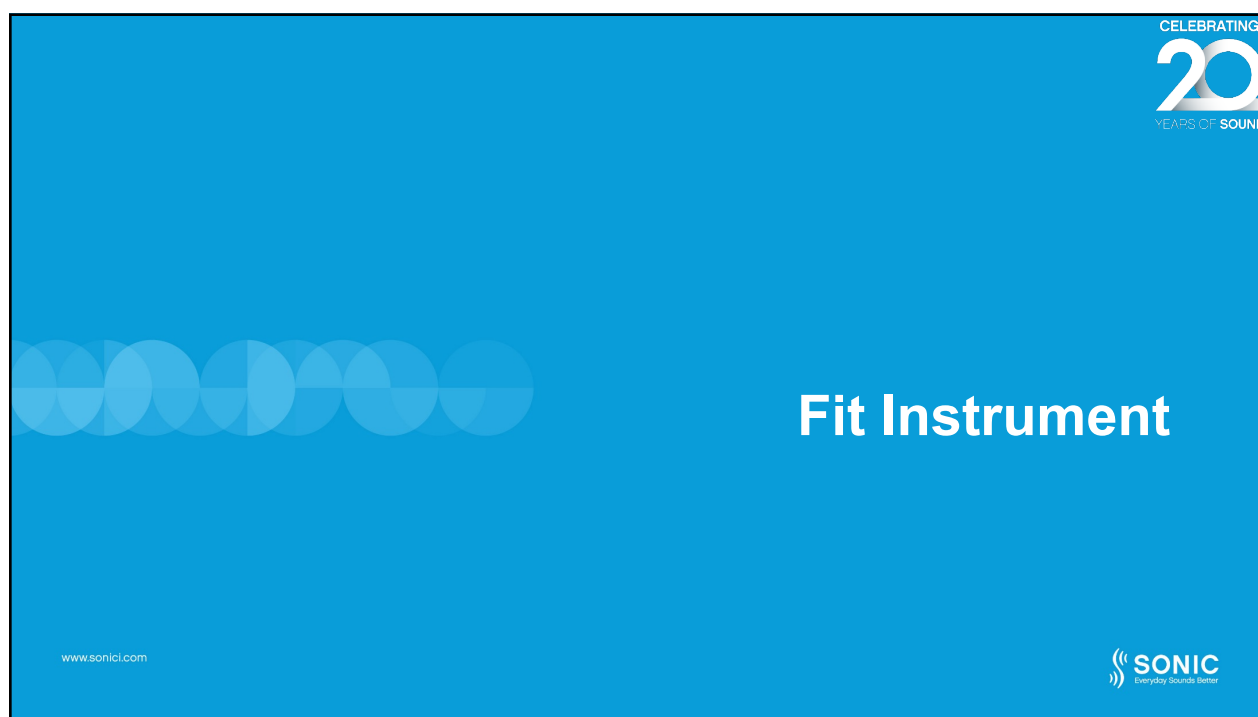

44

Fit Instrument

CELEBRATING
20
YEARS OF SOUND

- Adaptation Manager
- Fine Tuning
- Programs
- FB Manager
- Frequency Transfer
- Tinnitus SoundSupport

www.sonici.com


SONIC
Everyday Sounds Better

45

Adaptation Manager

CELEBRATING
20
YEARS OF SOUND

- Automatic or manual gain increases
 - 80% - 90% - 100%
- Duration of automatic increase:
 - One week to 4 months

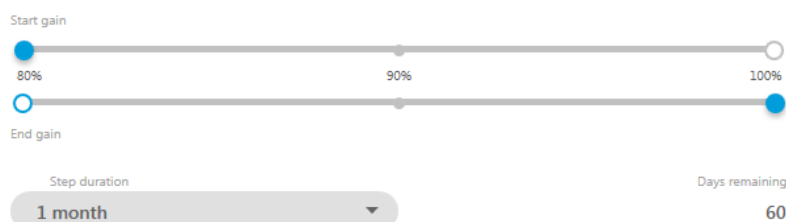


www.sonici.com

SONIC
Everyday Sounds Better

46

Fine Tuning

CELEBRATING
20
YEARS OF SOUND

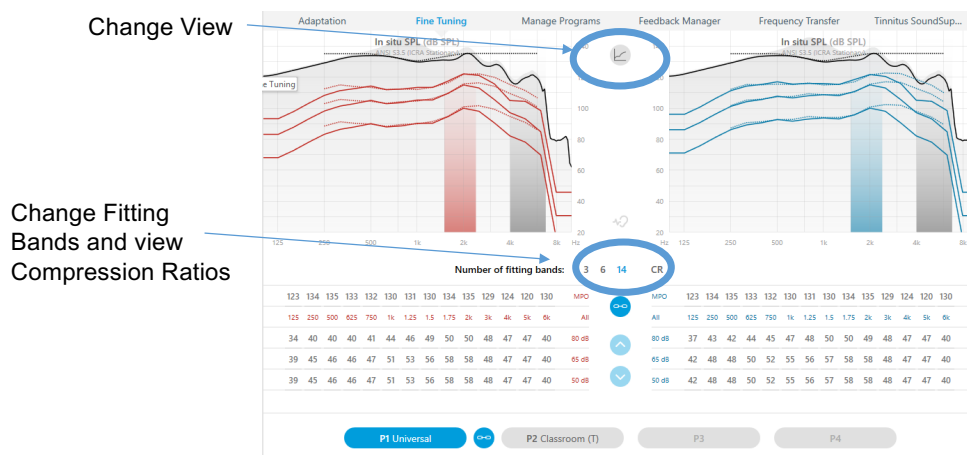


www.sonici.com

SONIC
Everyday Sounds Better

47

Programs

CELEBRATING
20
YEARS OF SOUND

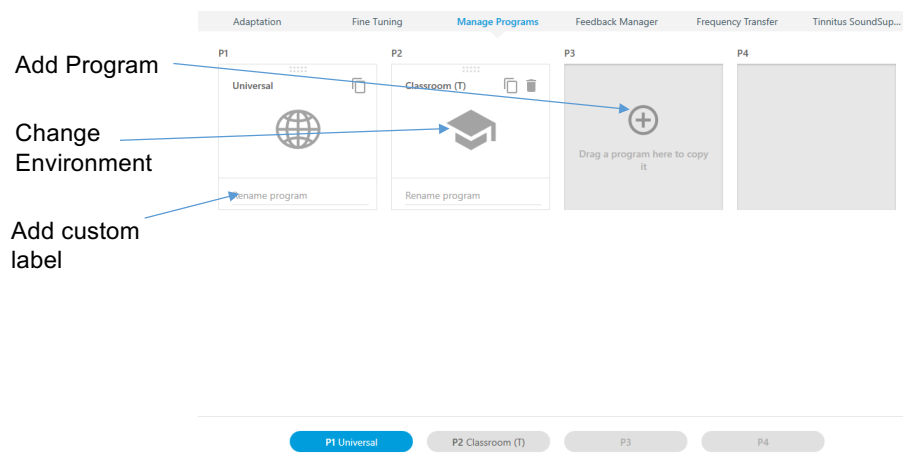


www.sonici.com

SONIC
Everyday Sounds Better

48

Feedback Manager

CELEBRATING
20
YEARS OF SOUND

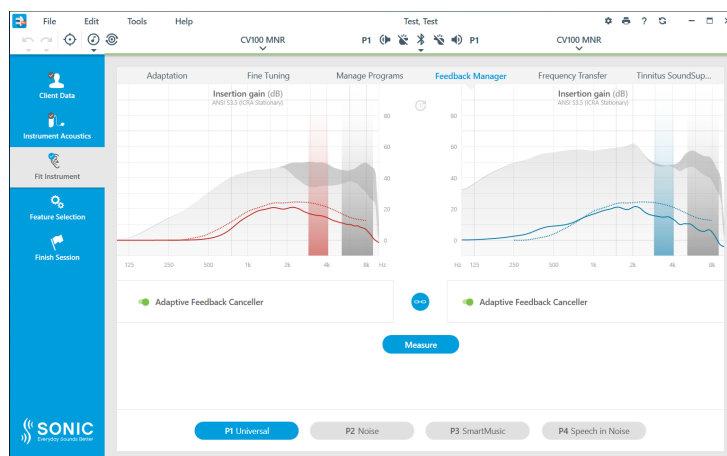

NEW in 2019!

**Adaptive
Feedback
Canceller PRO**

***Available in
Captivate and
Trek**

www.sonici.com

SONIC
Everyday Sounds Better

49

Frequency Transfer

CELEBRATING
20
YEARS OF SOUND

High Frequency
on/off

10 Destinations

7 Intensity levels



www.sonici.com

SONIC
Everyday Sounds Better

50

Tinnitus SoundSupport


CELEBRATING
20
YEARS OF SOUND

- Adjust Tinnitus signal:
 - White Noise, Pink Noise etc...
- Add modulation
- Add automatic level steering
 - Level of signal adjusts base on environment
- Turn off Hearing Aid microphone


www.sonicl.com

SONIC
Everyday Sounds Better

51

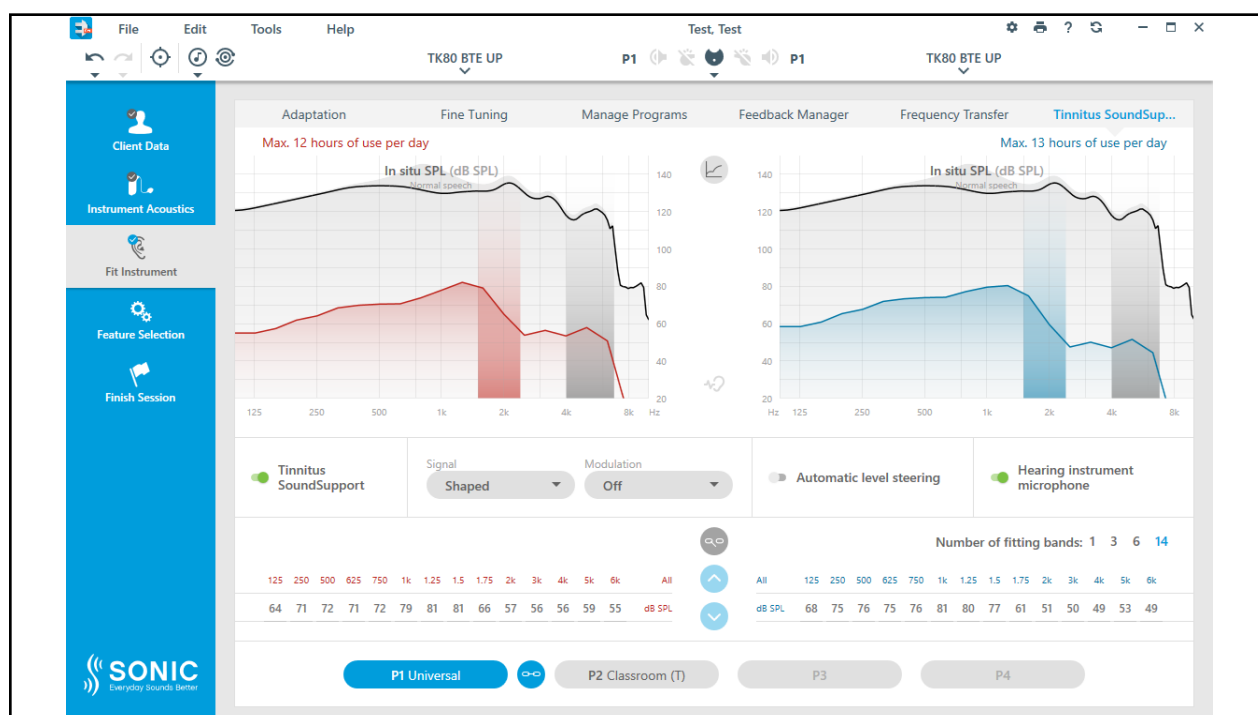

52

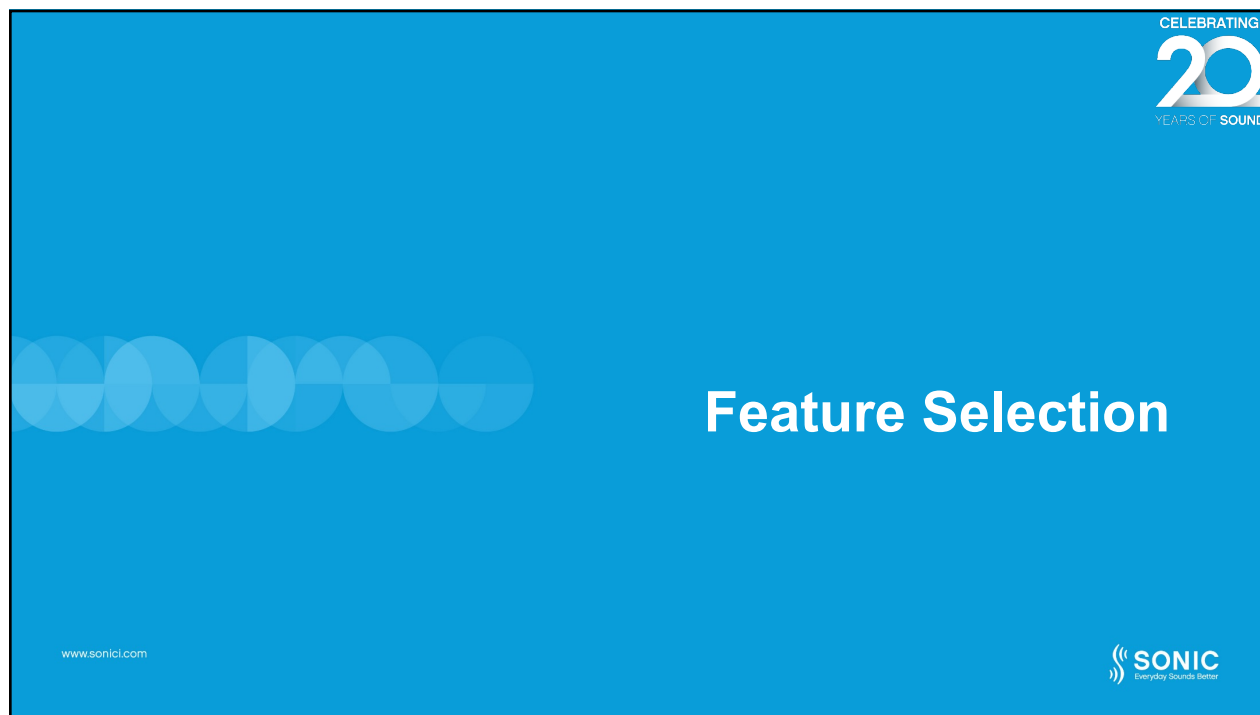

53

Feature Selection - General

Speech Variable Processing

- Envelope Focus
- Phoneme Focus

SmartCompress

- Compression Control: Speech in Noise Environment
- Gain Control: Noise Only Environment

Feature Selection

- P1 Universal
- P2
- P3
- P4

Speech Variable Processing

- Envelope Focus
- Phoneme Focus


SmartCompress

- Compression Control
- Gain Control

54

Feature Selection – Features

CELEBRATING
20
YEARS OF SOUND

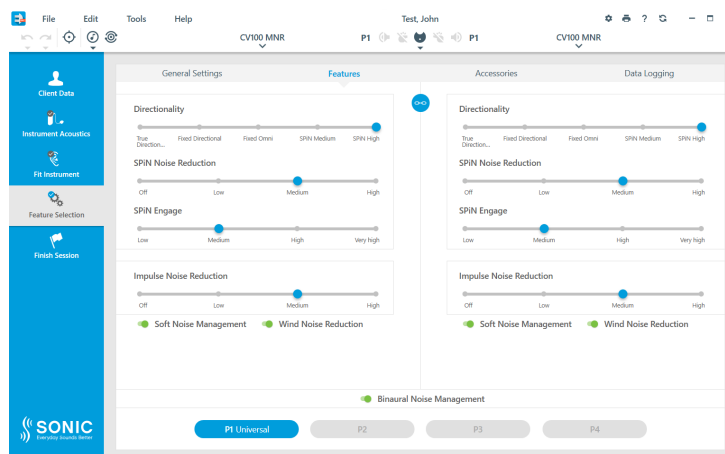

Directionality
SPiN Noise
Reduction
SPiN Engage
Impulse Noise
Reduction
Soft Noise
Management
Wind Noise
Reduction


www.sonicl.com

SONIC
Everyday Sounds Better

55

Feature Selection - Accessories

CELEBRATING
20
YEARS OF SOUND

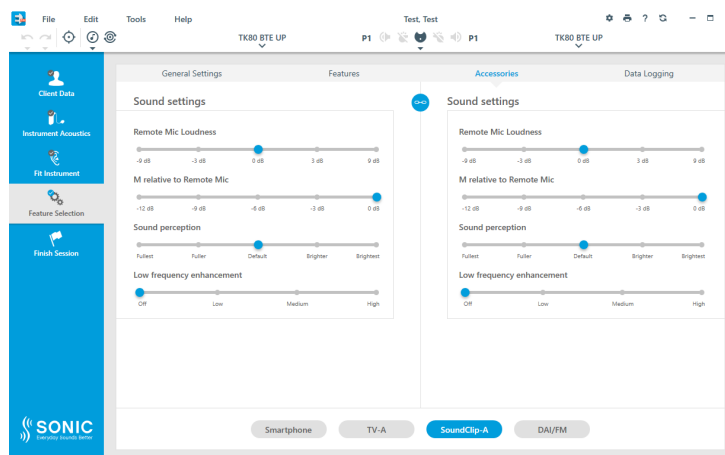


Sound Settings
Remote Mic
Loudness
Mic relative to
Remote Mic
Sound Quality
Low Frequency
Enhancement

www.sonicl.com

SONIC
Everyday Sounds Better

56

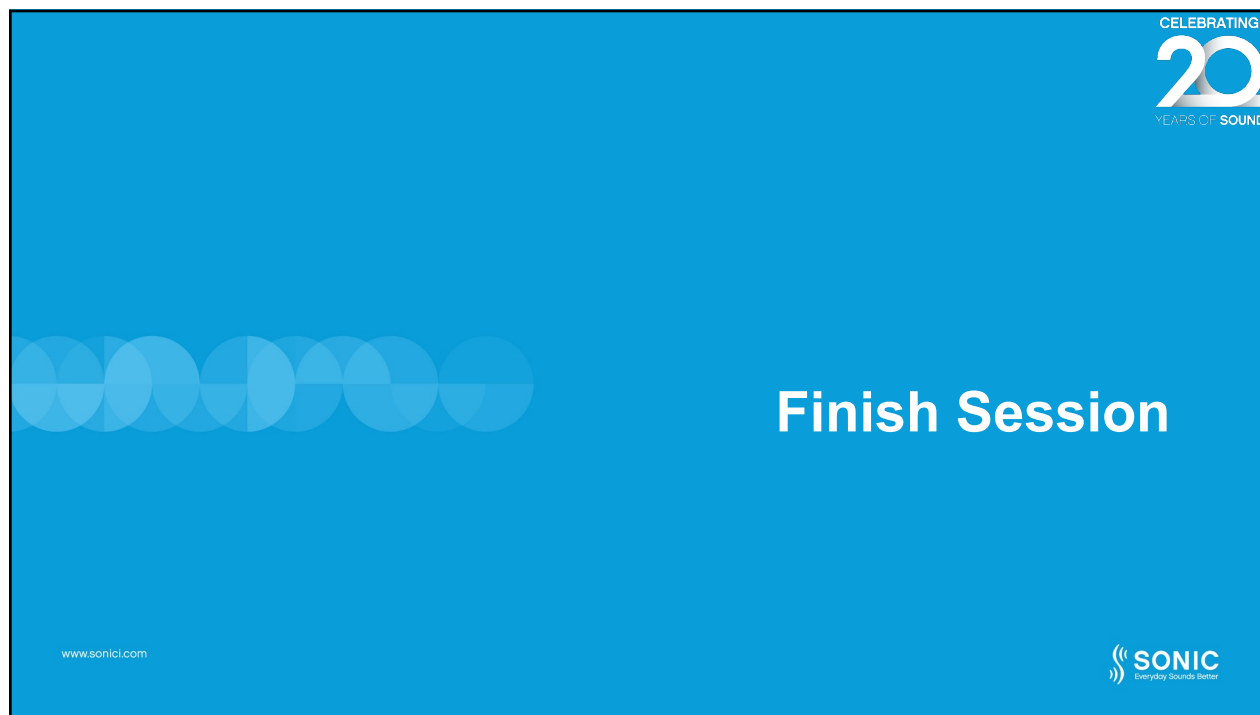

57

Finish Session – General Controls

CELEBRATING 20 YEARS OF SOUND

Volume Control
Program Change
Mute

New in 2019.2!
VC step size and VC range adjustments (Trek only)


www.sonici.com

SONIC
Everyday Sounds Better

58

Finish Session – Audible Indicators

CELEBRATING
20
YEARS OF SOUND

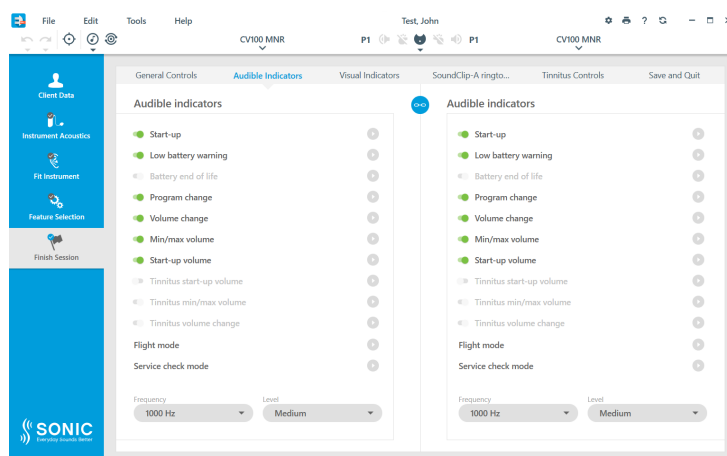

Demonstrate
Audible Indicators
Change Frequency
Change Level

www.sonici.com

SONIC
Everyday Sounds Better

59

Finish Session – Visual Indicators

CELEBRATING
20
YEARS OF SOUND


- Visual indicators are available for the following styles
 - Captivate miniRITE T R
 - Trek SP | UP
- Adjustable in EXPRESSfit® Pro 2019.2



www.sonici.com

SONIC
Everyday Sounds Better

60

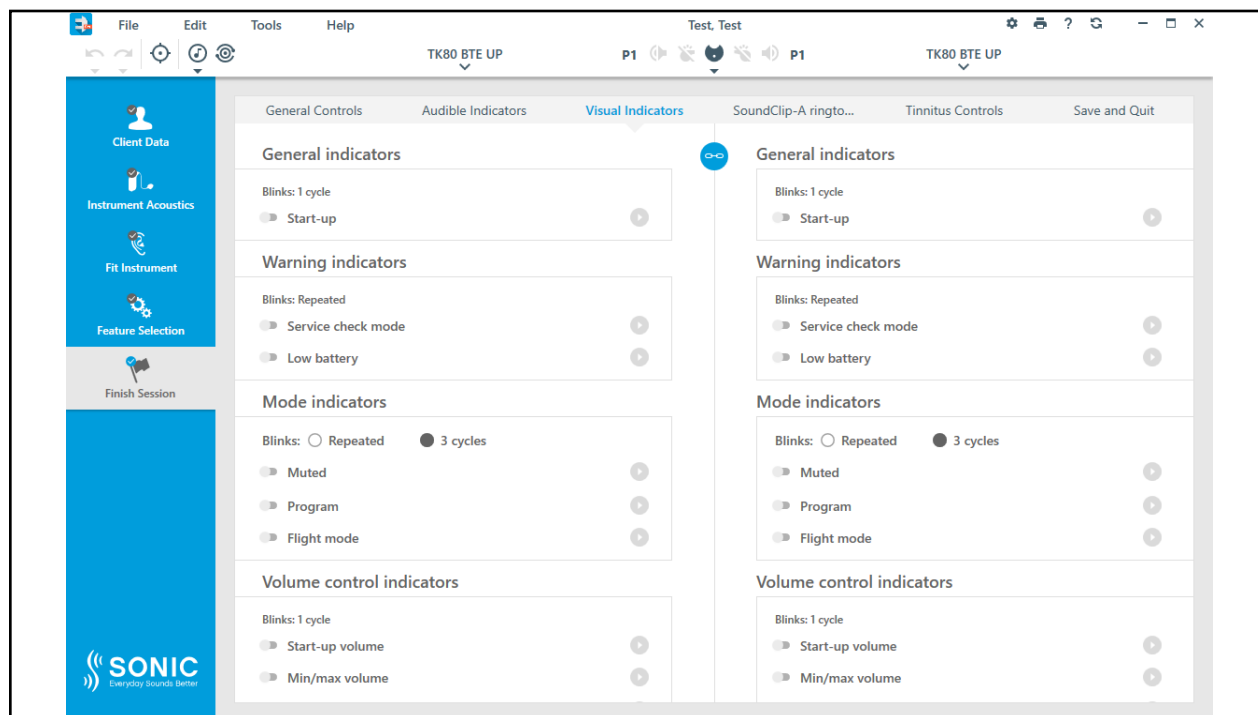

61

Finish Session – Tinnitus Controls

CELEBRATING 20 YEARS OF SOUND

Enable Tinnitus Volume Control

Set Range

Get Maximum Wearing Time data

www.sonici.com

SONIC
Everyday Sounds Better

62

Battery Status

CELEBRATING
20
YEARS OF SOUND


- The battery status screen is available for the Captivate miniRITE T R
- Battery Status
 - Battery Level
 - Battery Health



www.sonicl.com

SONIC
Everyday Sounds Better

63

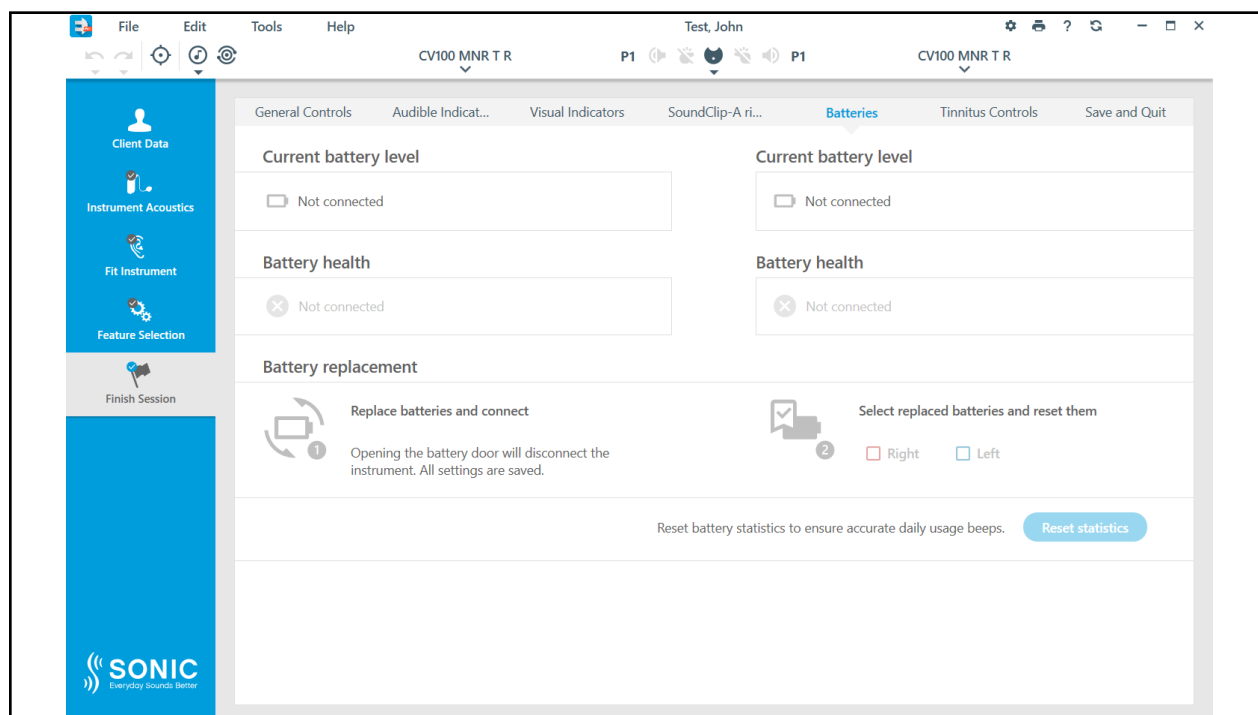

64

CELEBRATING
20
YEARS OF SOUND

Bimodal Fitting Support

New! in EXPRESSfit® Pro 2019.2

www.sonici.com

SONIC
Everyday Sounds Better

65

New in 2019.2!

Bimodal Fitting Support

- Downloadable PDF with advice to the professional on how to fit both cochlear implant and hearing aid technologies together
- Open in Help

Help

? Open help file

Bimodal fitting

About

Instructions For Use

Terms and Conditions

www.sonici.com

Bimodal flowchart


All unilateral cochlear implant recipients with usable residual hearing in the other ear are candidates for bimodal fitting. The flowchart provides an evidence-based, yet practical, method for fitting a hearing aid on a bimodal claim. The flowchart takes into account wideband fitting, restricted bandwidth fitting, use of frequency lowering and bandwidth lowering.

BOSS FLOWCHART
This bimodal fitting flowchart was developed by Carole Hays, AuD, CCC-A, Staff Audiologist at Ohio State University Hospital. Reprinted permission for use by Sonics, LLC.

www.sonici.com

SONIC
Everyday Sounds Better

66

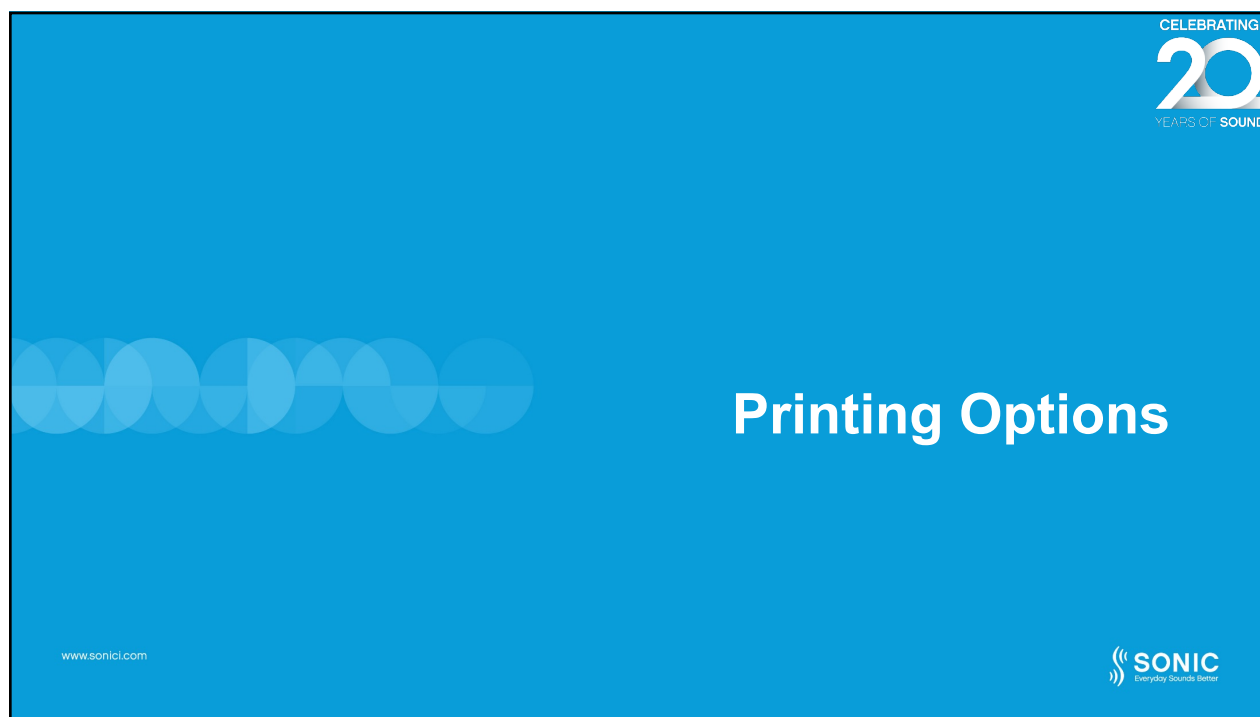

67

CELEBRATING 20 YEARS OF SOUND

Print Functions

- Client
 - Quick Guides
 - Hearing Instrument
 - SoundClip-A
 - Tinnitus settings
 - Customizable
 - Title Page option
 - Language selection
- Hearing Care Professional
 - Hearing Instrument settings
 - Customizable
 - HCP name
 - Company logo
 - Language Selection

www.sonicl.com

SONIC
Everyday Sounds Better


68

Quick Guide – Hearing Instrument

CELEBRATING
20
YEARS OF SOUND

Test Test
Hearing Instrument quick guide

Left hearing instrument **Right hearing instrument**



Turn the wheel
Volume up/down
Short press
Next program
Long press
Mute

Turn the wheel
Volume up/down
Short press
Next program
Long press
Mute

My hearing aid	L	R
Hearing instrument style	Enchant 100 ITEFS 85	Enchant 100 ITEFS 85
Serial number	Not present	Not present
Battery size	312	312
Wax filter	ProWax minifit	ProWax minifit
Acoustics	L	R
Vent	0.8 mm	0.8 mm
Available programs	P1: Universal	P1: Universal

When you press the pushbutton on one hearing instrument, the program in the other instrument will change as well.

Test Test
Hearing Instrument quick guide

Program description

P1: Universal

This program is personalized to your preferences and needs. It is the best choice for most listening situations and general instrument usage. Your instrument will automatically start in this program when turned on.

Volume control (P1)	L	R
Min/max volume	3 beeps	3 beeps
Preferred volume	2 beeps	2 beeps

When you adjust the volume in one instrument, the volume in the other instrument is also adjusted.

Mute

When you mute one hearing instrument, the second one will be muted as well.

Warnings	L	R
Battery low	3 beeps	3 beeps
Change battery	4 beeps	4 beeps

The SoundLink 2 app

The SoundLink 2 app enables you to control your hearing aids, choose what you want to hear and adjust the sound. The app offers a "Find my hearing aid" search feature, links to user instructions and low battery notification.

Search for the free SoundLink 2 app in Google Play or App Store.

Download on the App Store | GET IT ON Google Play

www.sonici.com


SONIC
Everyday Sounds Better

69

Quick Guide – SoundClip-A

CELEBRATING
20
YEARS OF SOUND

Test Test
SoundClip-A quick guide

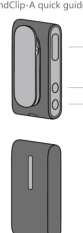

Short press
Accept/end call
Medium press (3 s)
Power on
Long press (6 s)
Power off

Short press
Volume up
Medium press (3 s)
Next program

Short press
Volume down
Medium press (3 s)
Mute

Available programs	L	R
P1: Universal	P1: Universal	P1: Universal
Ringtones	L	R
Mobile	On	On
Landline	On	On
BTDB00 USB	On	On

Test Test
SoundClip-A quick guide

The SoundLink 2 app

The SoundLink 2 app enables you to control your hearing aids, choose what you want to hear and adjust the sound. The app offers a "Find my hearing aid" search feature, links to user instructions and low battery notification.

Search for the free SoundLink 2 app in Google Play or App Store.

Download on the App Store | GET IT ON Google Play

www.sonici.com

SONIC
Everyday Sounds Better

70

Quick Guide - Tinnitus

CELEBRATING
20
YEARS OF SOUND

Test Text

Tinnitus quick guide

Your tinnitus settings

	L	R
Playing tinnitus relief sound	Apply a short press to change programs	Apply a short press to change programs
Available programs	P1: Shaped	P1: Shaped

When you press the pushbutton on one hearing instrument, the program in the other instrument will change as well.

Maximum wearing time per day

To ensure safe listening levels it is recommended that you limit the use of the Tinnitus program(s) each day based on the number of hours (h) listed in the table below.

Program	Start-up volume (Tinnitus)	Max volume (Tinnitus)
P1: Shaped	12 h	12 h



Mute

If you are in a program with Tinnitus SoundSupport turned on, the mute function will only mute the environmental sounds, but not the tinnitus relief sound.

The SoundLink 2 app

The SoundLink 2 app enables you to control your hearing aids, choose what you want to hear and adjust the sound. The app offers a "Find my hearing aid" search feature, links to user instructions and low battery notification.

Search for the free SoundLink 2 app in Google Play or App Store.

www.sonici.com

SONIC
Everyday Sounds Better

71

CELEBRATING
20
YEARS OF SOUND

First Fit Protocol

7 steps to success

www.sonici.com

SONIC
Everyday Sounds Better

72

First Fit Protocol

CELEBRATING
20
YEARS OF SOUND

1. Edit Client Info
 - Verify demographics and fitting rationale
2. Adaptation Manager
 - Default is 80%
3. Program Manager
4. Run Feedback Manager



www.sonici.com

SONIC
Everyday Sounds Better

73

First Fit Protocol

CELEBRATING
20
YEARS OF SOUND

5. Set Initial overall gain
 - Perform Real Ear Fit or...
 - Paper Crinkle test
6. Adjust for patient's own voice
7. Finish Session
 - Device Options and Audible Indicators
 - Save and Quit



www.sonici.com

SONIC
Everyday Sounds Better

74

Final Comments



- Thank you for joining us today for our course
 - EXPRESSfit® Pro 2019.2: Not Your Mama's Fitting Software
- Any Questions?
 - 888.423.7834
 - www.sonici.com
 - www.mysonici.com
 - Email: support@sonici.com

www.sonici.com



75



76