




1



2

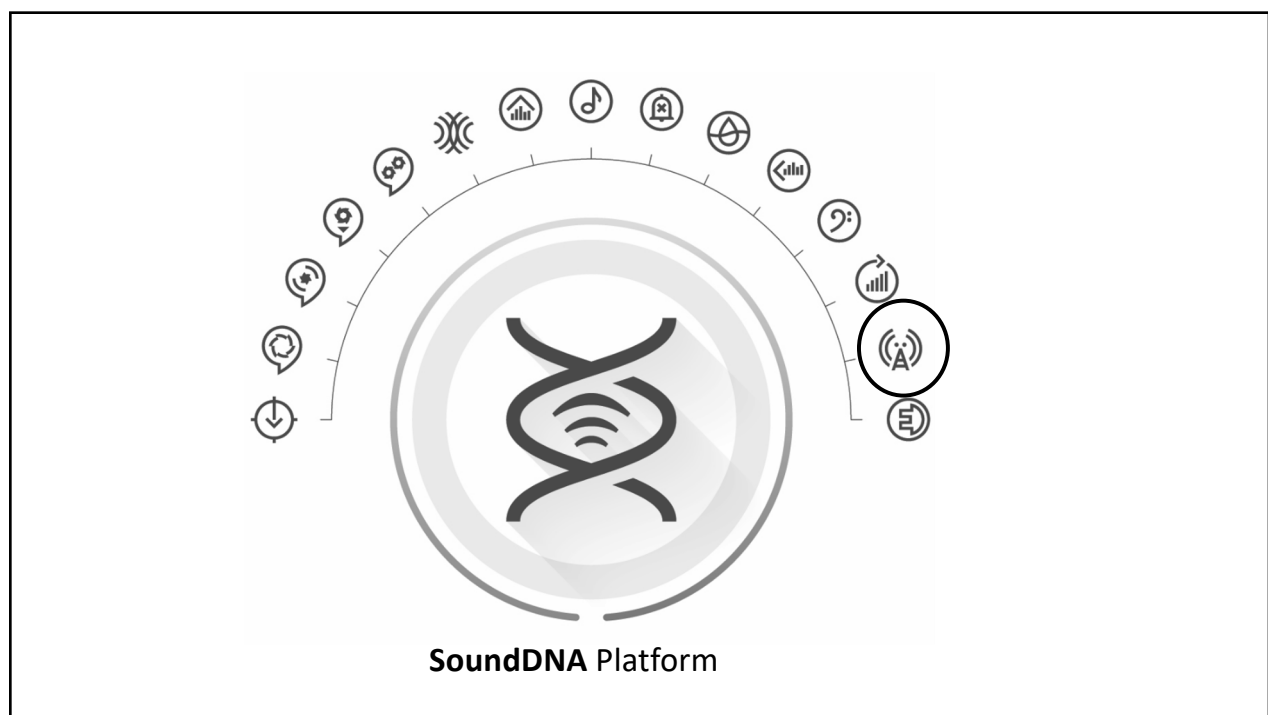


Dual-Radio System


Fast and direct wireless transmissions
with optimized battery life

3

Connectivity Portfolio

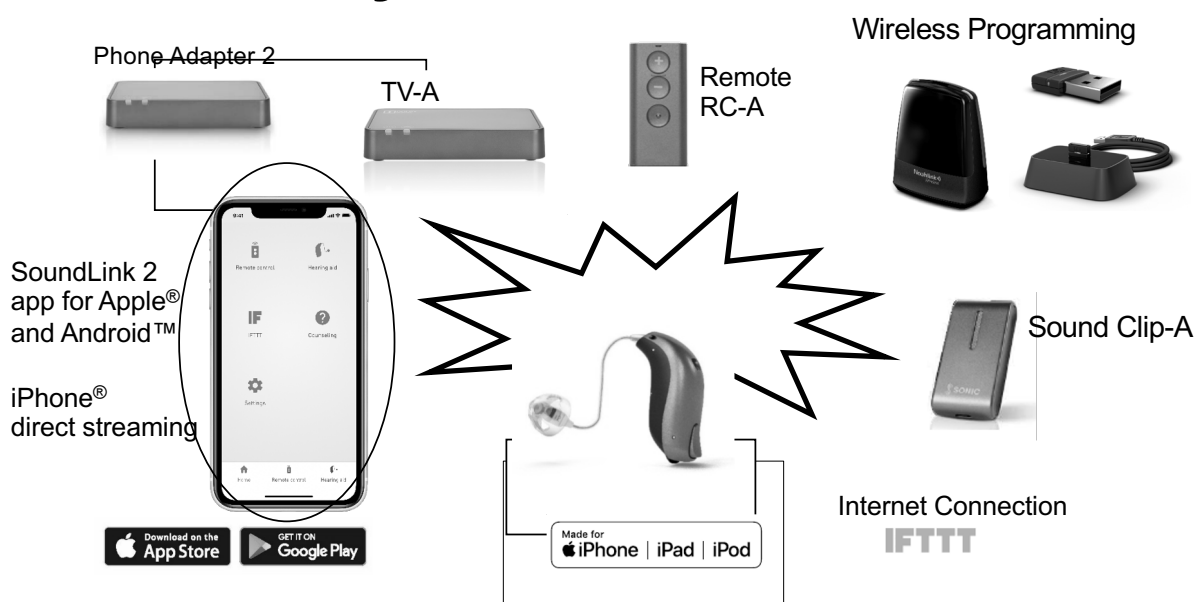

4

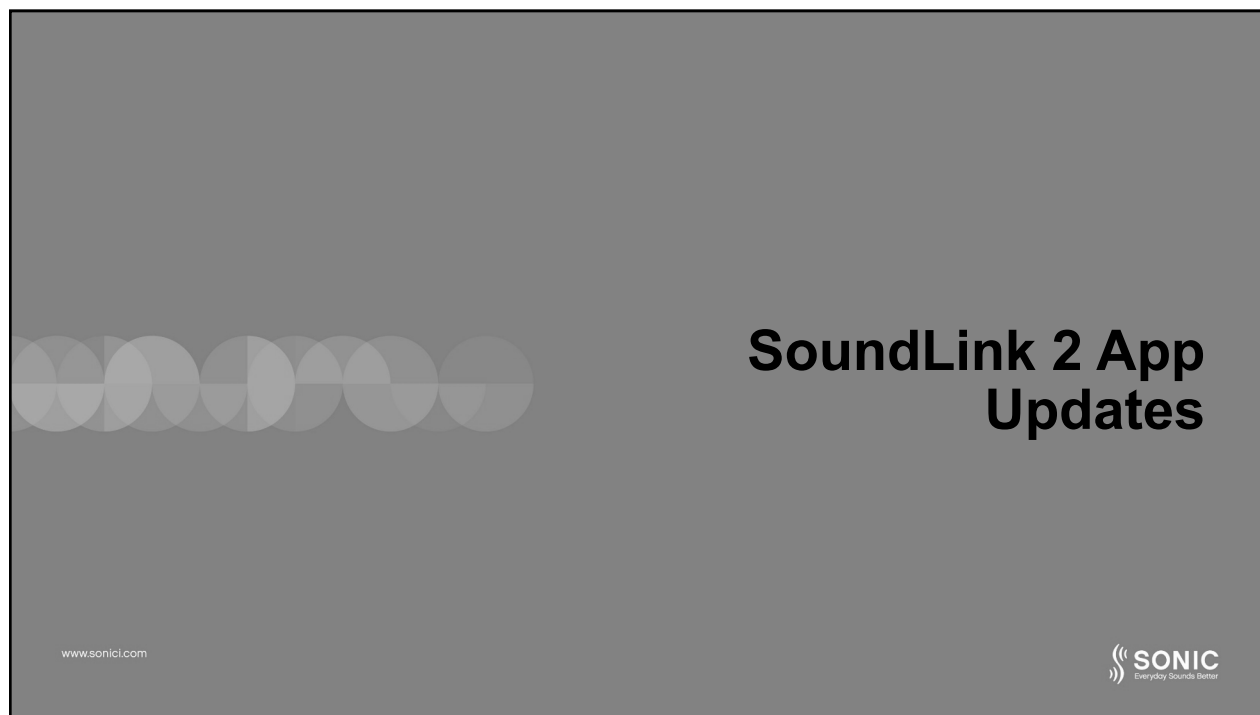


5


SoundLink 2 App - Updates

- New user interface and navigation flow
- Available for both iOS and Android
- Support for both Captivate 20 and 40
- DEMO mode
- Hearing aid info and data
- Instructions for use

www.sonici.com

 **SONIC**
Everyday Sounds Better


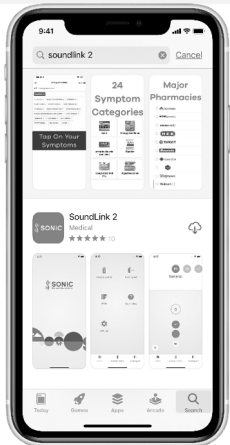
6

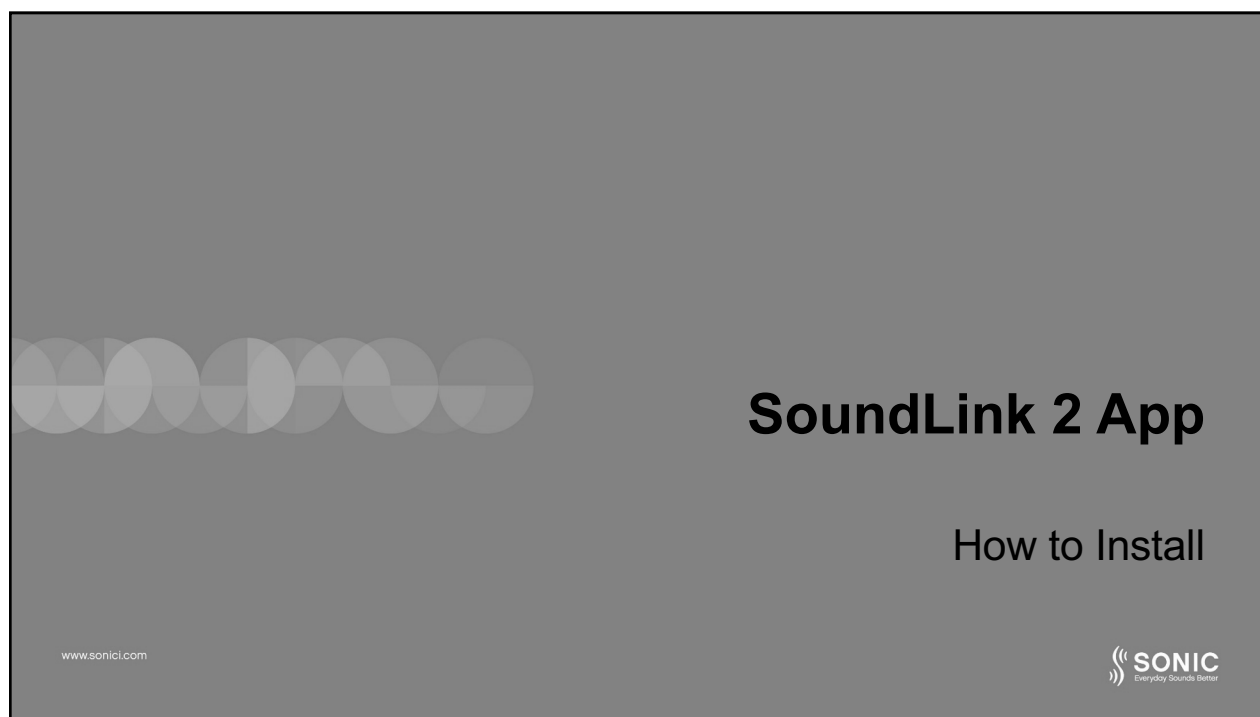

7

Installing the SoundLink 2 App

- Search SoundLink 2
 - iPhone → App Store
 - Android → Google Play

The iPhone screen shows a search for "SoundLink 2" with a single result from "Watch Apps". The Android screen shows a search for "soundlink 2" with a list of results, including "24 Symptom Categories", "Major Pharmacies", and the "SoundLink 2" app by SONIC, which is highlighted.


www.sonici.com

SONIC
Everyday Sounds Better

8


Installing the SoundLink 2 App

Select Download

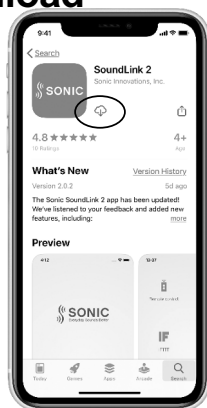

www.sonicl.com

App will install

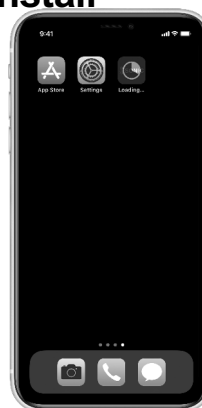

9

SoundLink 2 App

First time use

www.sonicl.com



10

First Time Use

- Start SoundLink 2 app from the home screen
 - Allow access to notifications and location services
- Ensure devices are connected
 - iPhone connects via Settings
 - Android connects within app




www.sonici.com



11

First Time Use

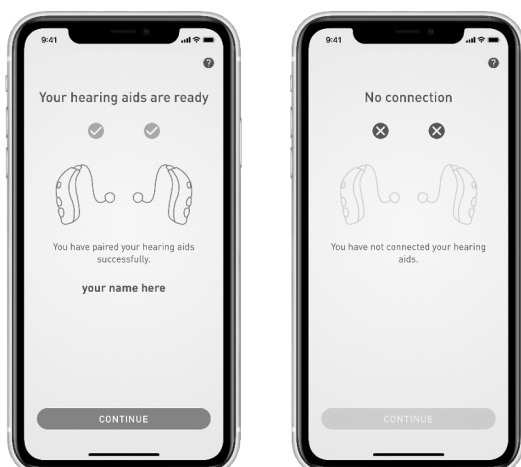


www.sonici.com




- App will initially show connection status
- If no connection retry pairing process
- Once connected app will become fully functional

12

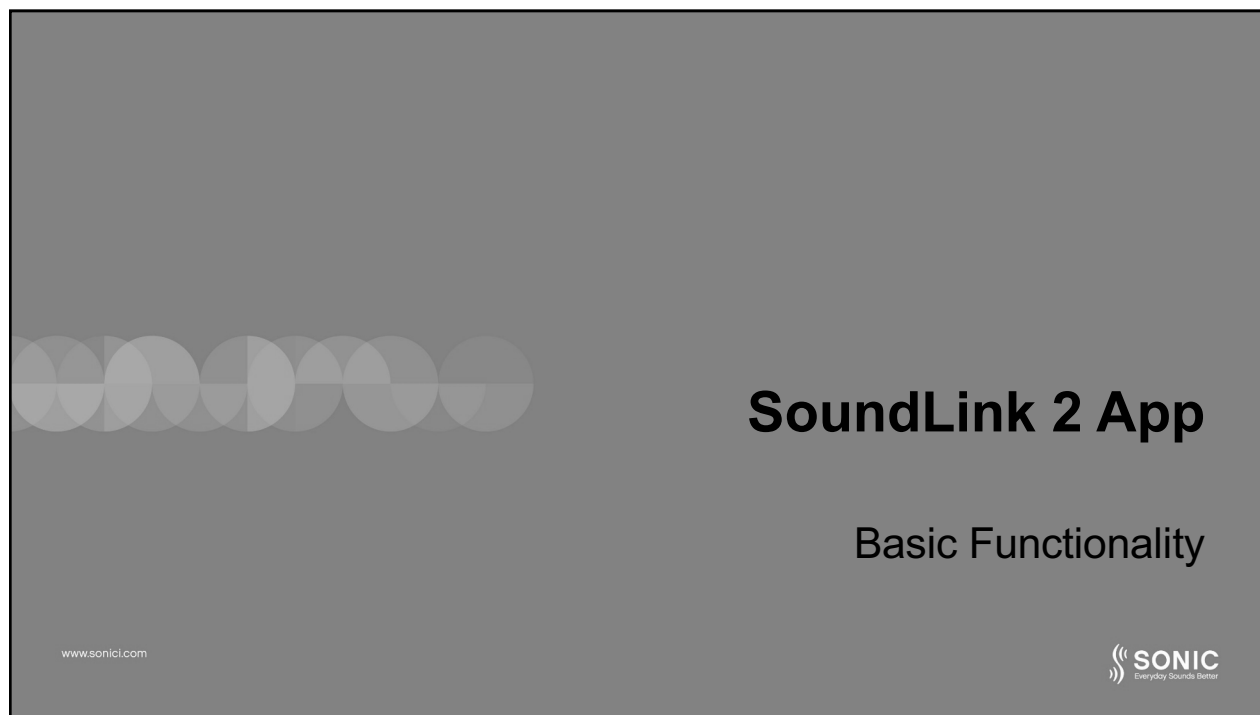

13

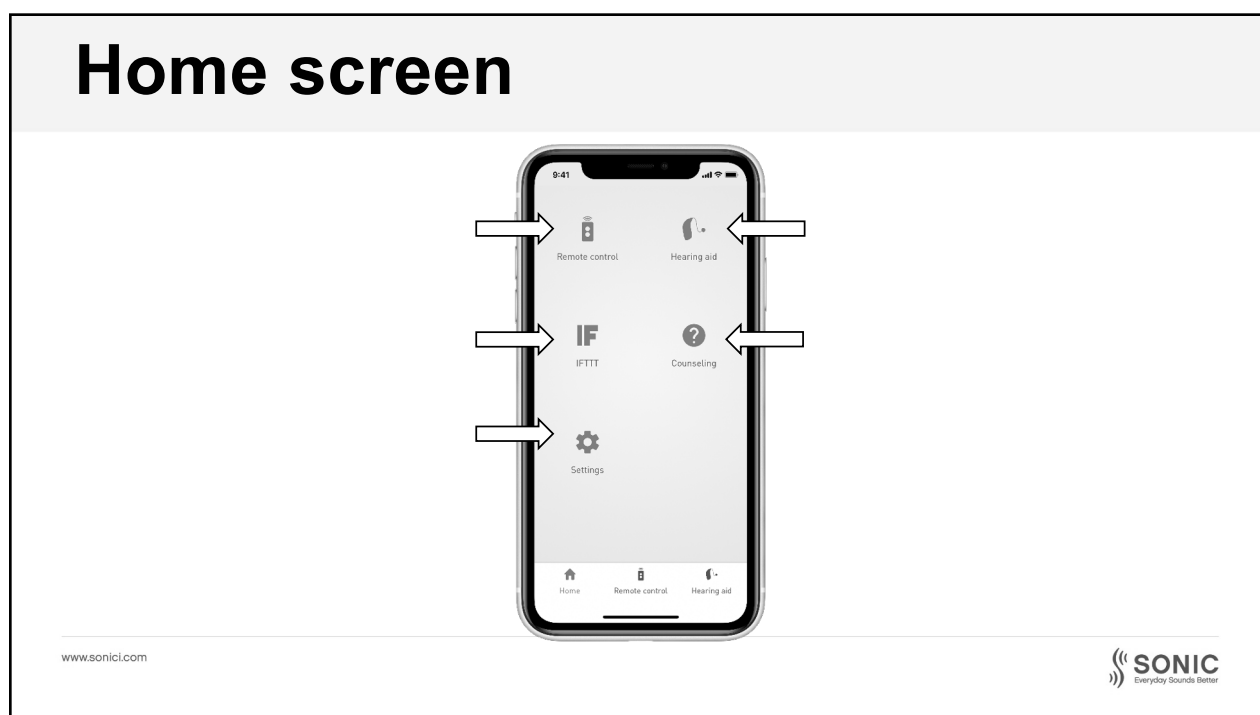

14

Remote control



www.sonicl.com

SONIC
Everyday Sounds Better

15

Remote control

- Program change
 - Active program highlighted
 - Touch desired program icon to change



www.sonicl.com

SONIC
Everyday Sounds Better

16

Remote control

- Volume change
 - + will increase
 - - will decrease
 - Mute
- Independent VC adjustments




www.sonicl.com

SONIC
Everyday Sounds Better

17

Hearing aid

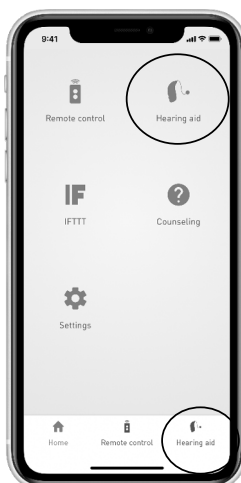


www.sonicl.com

SONIC
Everyday Sounds Better

18

Hearing aid

- Hearing aid information
 - Battery status
 - Serial numbers and firmware versions

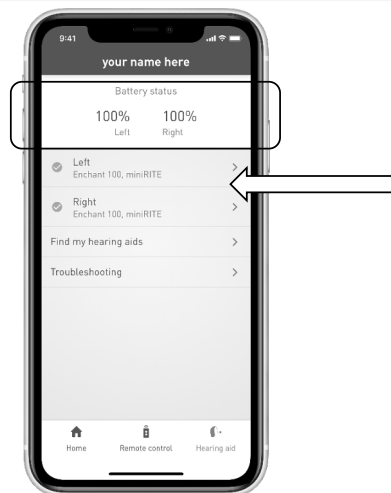


www.sonici.com

SONIC
Everyday Sounds Better

19

Hearing aid

- Find my hearing aids
 - Depends on location services (always allow)
 - Show last active connection on map
 - Proximity only available if connection is active

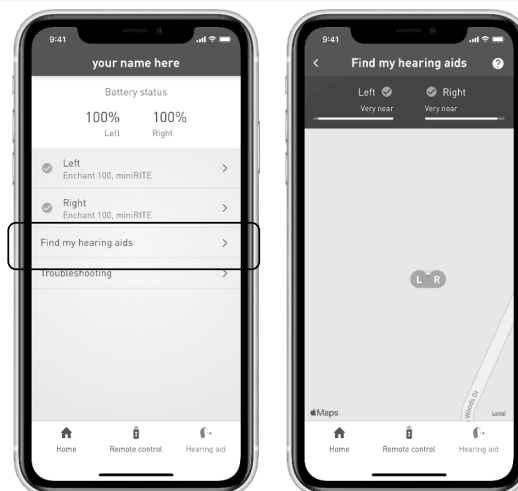


www.sonici.com

SONIC
Everyday Sounds Better

20

Hearing aid

- Troubleshooting
 - Basic tips for lost connections

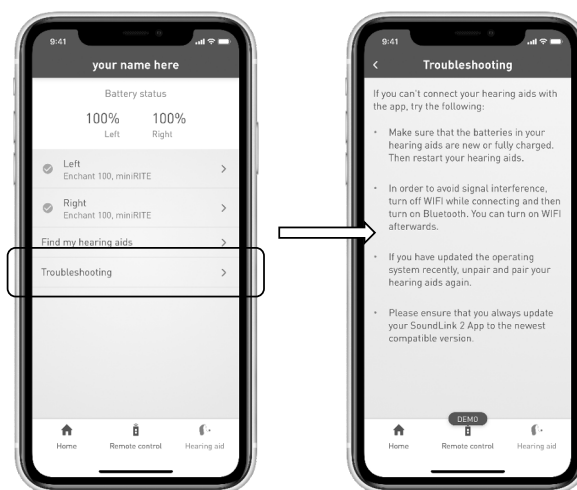


www.sonicl.com



21

Counseling

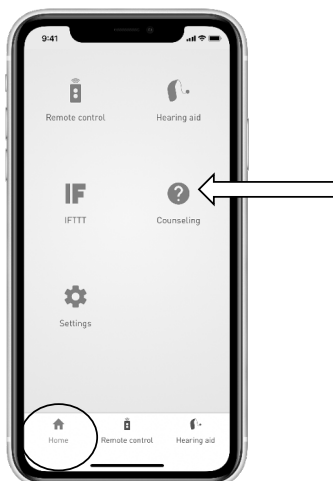


www.sonicl.com



22

Counseling

- Useful information and tips for your patients
- Can help with realistic expectations
- Communication strategies for difficult situations

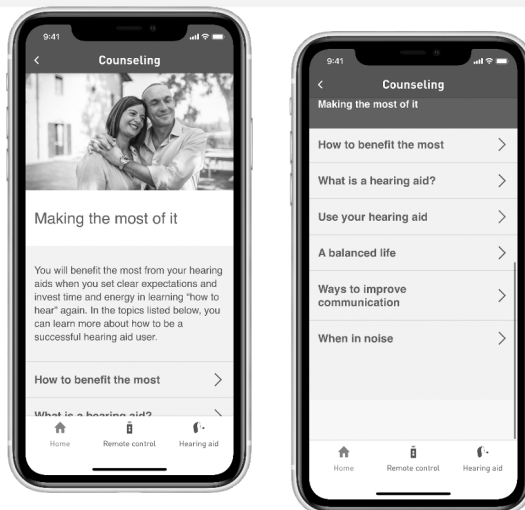

www.sonicl.com

SONIC
Everyday Sounds Better

23

Settings




www.sonicl.com

SONIC
Everyday Sounds Better

24

Settings

- My account
 - Allows for set up and access to IFTTT
- Battery low notification
 - Notifications through phone to advise of low battery
- Voice Prompts
 - For program change and battery level
- Haptic feedback
 - Phone vibrates with volume changes
- Additional options

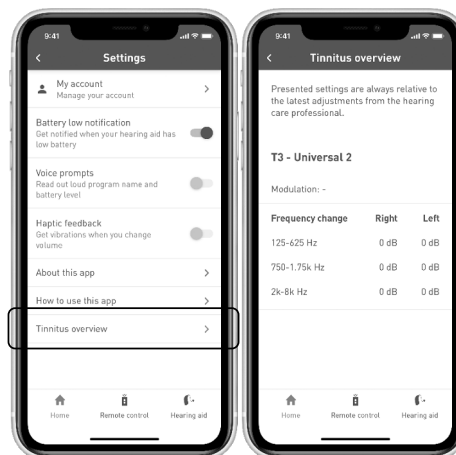

www.sonici.com



25

SoundLink 2 App

Tinnitus Options

www.sonici.com



26

Tinnitus Options

- Volume changes
 - Noise vs Surroundings
 - Increase, decrease and mute
- Advanced settings
 - Only available under 'Noise'



****Program must be added by the hearing care professional****


www.sonicl.com

SONIC
Everyday Sounds Better


27

Tinnitus options

- Advanced settings
 - Noise variations

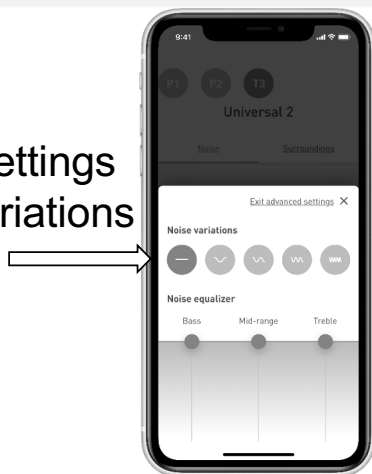

- Noise equalizer

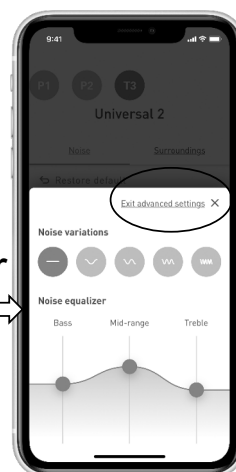


www.sonicl.com

SONIC
Everyday Sounds Better

28

Tinnitus options

- Restore default
 - Changes kept until defaults restored
- Tinnitus overview
 - Gives overview of changes
 - Found under 'Settings' from home screen

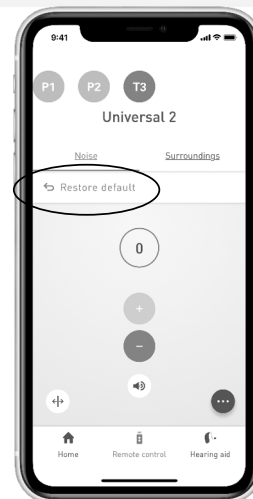

www.sonici.com

SONIC
Everyday Sounds Better

29

SoundLink 2 App

Live Listen from iPhone Mic and SoundClip-A

www.sonici.com

SONIC
Everyday Sounds Better

30

SoundClip-A

- Select SoundClip-A program
- Volume Adjustments
 - SoundClip-A
 - Surroundings




www.sonici.com



31

Live Listen

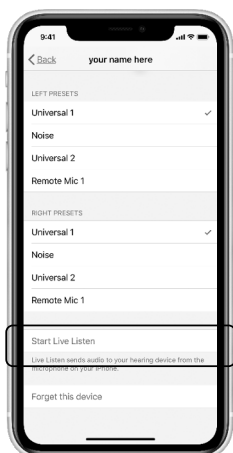


- Available via iPhone remote
 - Triple click of home or side button
- iPhone will act as a remote mic

www.sonici.com




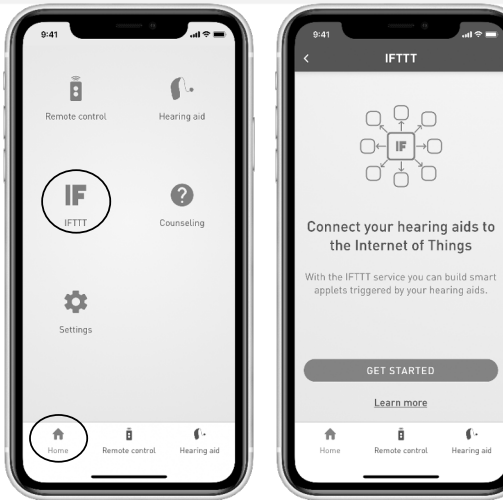
32

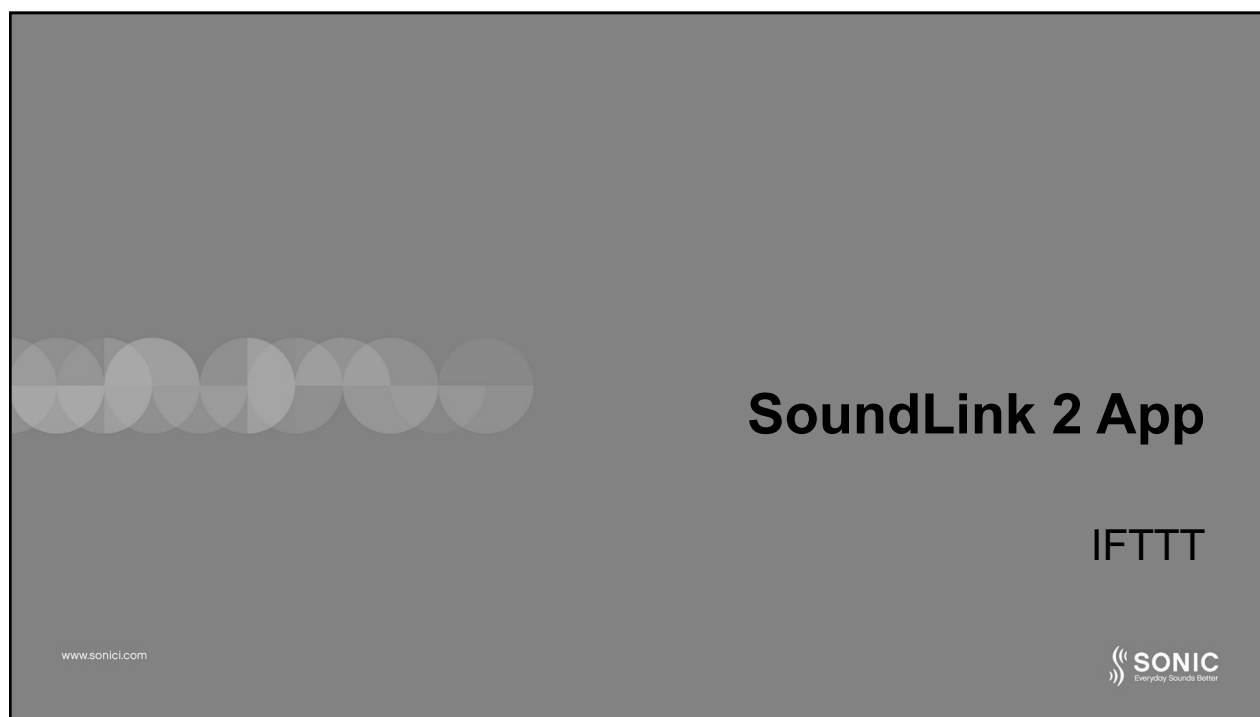


33

Access IFTTT

- From home screen touch IF to access IFTTT
 - Connect to hundreds of internet based services
 - Create Applets
 - <https://ifttt.com/go>


www.sonicl.com



34

IFTTT

- Create applets
 - Triggers
 - Actions
- For use with other connected devices

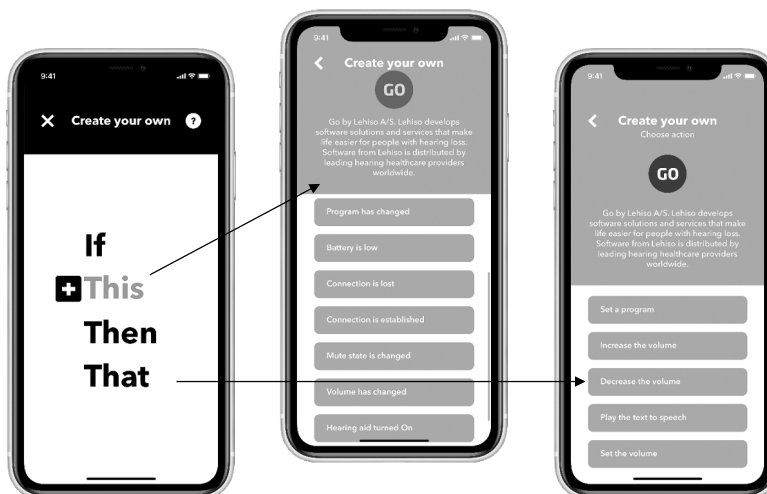

www.sonici.com



35

SoundLink 2 App

Summary

www.sonici.com



36

SoundLink 2 App - Summary

- Available for iPhone and Android
- Select program
- Adjust volume and mute
- Adjust streaming levels
- Adjust Tinnitus SoundSupport
- Find my hearing aid
- Access IFTTT
- Easy to use

www.sonici.com



37

Final Comments

- Thank you for attending our Sonic Spotlight Series: SoundLink 2!
- Any Questions?
 - 888.423.7834
 - www.mysonici.com
 - www.sonici.com

www.sonici.com



38



Bluetooth is a registered trademark owned by Bluetooth SIG, Inc, USA.

Apple, the Apple logo, iPhone, iPad, iPod touch, and Apple Watch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Android, Google Play, and the Google Play logo are trademarks of Google LLC.

© 2020 Sonic Innovations, Inc. All rights reserved.